Manufactured for:
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com
Made in China

Toilet Safety Rail with Adjustable Height with Adjustable Height



ASSEMBLY INSTRUCTIONS

- 1. Detach your toilet seat by first removing toilet seat bolts.
- 2. Place rail mounting flange on toilet and align holes.
- 3. Place toilet seat on top and align holes.
- 4. Realign bolts and tighten securely.
- Slide assist rails into mounting flange and secure with built-in snap buttons.
- 6. Set rail to desired width using the built-in snap buttons.
- 7. Set rail to desired height using the built-in snap buttons.

MAINTENANCE

Inspect the product weekly and tighten it as needed to ensure secure installation. Check all rubber tips and replace as needed.

CAUTION

Clean with solvent-free household cleaner.

WARNING

Before using this product, please ensure the full understanding of these instructions. If it is not properly assembled and installed, physical injury and damage could result. Make sure all built-in snap buttons are secured before use of product. If components are damaged or missing, contact your dealer immediately. Users with limited physical capacities must be assisted or supervised when using the product.

LIMITED LIFETIME WARRANTY

Your Dynarex product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions: Problems arising from normal wear; problems arising from failure to adhere to the product instructions; problems arising from misuse, negligence, accident or improper operation, maintenance or storage; problems arising from modifications or unauthorized repairs, parts or attachments; products where the serial number has been removed or defaced; problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

