## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS:

Provides support to heel to help provide pressure relief of Heel Spurs and Plantar Fasciitis. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

### CONTRAINDICATIONS: N/A

### WARNINGS AND PRECAUTIONS:

- The device must be prescribed and fitted by a suitably qualified healthcare professional.
  - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
  - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
  - Do not use over open wounds.
  - Do not use this device if it was damaged and/or packaging has been opened.
- NOTE:**Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INFORMATION:

A) Place heel cup into heel of shoe as far back as possible.

### USE AND CARE

- Rinse with cold water. Do not use soap.
- Lay on flat surface to dry. Do not wring out.
- Do not expose to direct heat sources.

### FOR SINGLE PATIENT USE ONLY.

 **NOT MADE WITH NATURAL RUBBER LATEX.**

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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MDSS GmbH