

Wrist Brace and Wrist Brace with Thumb Spica

NOTE: Instructions cover multiple product families, images may vary.



Figure 1



Figure 2



Figure 3

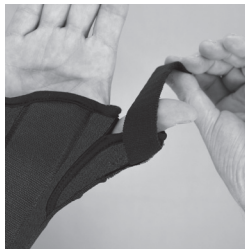


Figure 4

**NOT MADE WITH
natural rubber latex**

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Unfasten straps and slide affected hand into brace (Figure 1). Ensure stockinette panel is located on the top of the hand.
 2. Position palmar stay firmly against the palm of the hand. (Figure 2).
 3. Thread wrist straps through matching links (Figure 3).
 4. Pull and wrap straps around the wrist. Secure each strap when desired fit and level of compression is reached (Figure 3).
- NOTE:** For braces with thumb support, wrap thumb strap around thumb and fasten (Figure 4).
5. Trim wrist straps leaving approximately 1" – 2" of overlap.
 6. Adjust straps as needed when swelling of the hand, wrist or thumb changes.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.