LIMITED LIFETIME WARRANTY

Your Dynarex Product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs. parts or attachments
- Products where the serial number has been removed or defaced
- · Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

*This product meets or exceeds all federal safety and environmental regulations.

∧WARNING:

Cancer and Reproductive Harm www.P65Warnings.ca.gov



Reorder No. 10134

Offset Bariatric Cane

Reorder No. 10138 Bariatric Quad Cane with Large Base

R220719



Bariatric Cane





Manufactured for: Dynarex Corporation 10 Glenshaw Street | Orangeburg, NY 10962 USA • www.dvnarex.com | Made in China

FOR USERS TO READ BEFORE USE

For maximum safety, these precautions must be taken when using this product DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury may occur.

A WARNINGS/CAUTIONS

- · Use only as directed by a health care provider.
- · Length must be correctly adjusted to user's height.
- Spring button must be engaged and fully protruding through the adjustment hole and the collar must be tightened.
- Check that tips are not worn, ripped, cracked, or have excessive wear. If any of these conditions exist, replace the rubber tip immediately.
- The cane handle should be fully inserted into the hand grip.
- . Do not use if the user's weight is over 500 lb.
- Canes are designed to provide support, increased stability and assistance while walking. The canes are NOT designed to support the total weight of individuals.
- Do not use if user has any handicaps and/or disabilities or is taking any medication, etc. that a doctor, therapist and/or seller believes could cause interference with the safe use of the product.
- Do not use if there are any loose components, rattles, cracks, dents or other uncustomary characteristics in the metal, plastic, etc.
- · Do not subject the cane to sudden impact or jerking movements.
- Use the utmost care when using a cane. Watch out for potential safety hazards including, but not limited to, slippery, uneven or soft surfaces and objects in your path.

HEIGHT ADJUSTMENT

- Loosen the anti-rattle collar. Depending on the model, either turn the collar counterclockwise to loosen, turn the lock knob to loosen, or lift the plastic lever on the collar.
- 2. Press the snap button so that it enters the shaft of the cane.

- Slide extension up or down until snap button protrudes through an adjustment hole on handle end of cane.
 NOTE: An audible "CLICK" will be heard when snap button securely protrudes through the adjustment hole.
- Repeat steps 2-3 until cane is at height determined. PROPER CANE HEIGHT below.
- 5. Tighten the anti-rattle collar.

PROPER CANE HEIGHT

For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane:

- Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
- 2. Adjust cane so top portion of handle touches inside of wrist where it meets palm.

ADJUSTING QUAD CANE FOR RIGHT/LEFT USE

Quad canes (Reorder No. 10138) can be adjusted for either left or right hand use by following these easy steps:

- 1. Loosen the lock knob.
- Depress the spring button while rotating the handle 180 degrees.
- Engage the spring button in the hole that accommodates the user's height.
- Retighten the lock knob. NOTE: the 45-degree angled legs on quad canes should be placed farthest from the user's body when the cane is being used

CLEANING

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.