

 **Manufactured for:**
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com

Made in China



Tub Rail

Adjustable

LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

INSTALLATION INSTRUCTIONS

1. Clean bathtub thoroughly and wipe dry before applying rail. Contact surfaces must be dry and free of soapy residue to assure secure installation.
2. Loosen knob until rail is wide enough to slip on side of bathtub.
3. Place Tub Rail on bathtub with knob facing outside of tub and tighten until rail feels tight and secure
4. To test installation of rail, pull forward, back and side to side. If rail slips or moves, make sure surface is free of soapy film and is dry, then replace at desired location and retighten.

NOTE

Tub Rail requires retightening from time to time. Check periodically before use and retighten as needed.

HEIGHT ADJUSTMENT

After the Tub Rail is installed on the tub, remove the locking collars. Once removed, pull up or push down from the top of the handle to the desired height. Once the desired height is determined, making sure both holes line up, replace both the locking collars. Be sure the locking pins go through the tubing and protrude on the opposite side.

CAUTION

Weight Capacity: 300 lb. (136 kg)

Please use extreme care when installing on fiberglass tubs. **DO NOT OVERTIGHTEN** as the clamp may crack the fiberglass. This bathtub safety rail provides support and increased stability for an individual weighing up to 300 lb. This item is NOT designed to support the total weight of an individual. Use the bar for assistance only. The maximum horizontal capacity is 100 lb. (45 kg).