Manufactured for: **Dynarex Corporation** 10 Glenshaw Street Orangeburg, NY 10962 USA · www.dynarex.com Made in China

Elongated Raised Toilet Seat



with Detachable Arms







WEIGHT CAPACITY: 300 lb.

INCLUDED IN BOX



Elongated Raised Toilet Seat



Arm x2



Screw x2



Wing Nut x2




Washer (A)



Washer (B) x2

ASSEMBLY

- Turn the Elongated Raised Toilet Seat upside down. Locate the receiving brackets and find the "R" and "L" markings denoting the Right and Left sides.
- Inspect the arms. Find the "R" and "L" markings denoting the Right and Left sides.

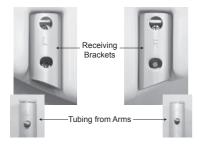

 Align the Right side arm with the Right side of the Elongated Raised Toilet Seat. Insert both ends of the arm into the receiving brackets.
Depress the brass push pin and slide the arm into the brackets. Ensure brass push-pin "snap" through the holes in the opposite fame.



4. Repeat step 3 for the left side.

INSTALLATION

- 1. Remove your toilet seat. (Figure 1)
- Place your Elongated Raised Toilet Seat on the toilet, place your toilet seat on top of the riser, lining up the holes of the riser and the seat with the holes on the toilet frame. (Figure 2)
- Put the enclosed bolts thru the holes, attaching the Elongated Raised Toilet Seat to the toilet frame with the nuts provided. (Figure 3)

WARNING! Before using your Elongated Raised Toilet Seat, push on the unit firmly to ensure that it is securely attached.

