

LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.



SYMBOL GLOSSARY

For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

R220701




WEIGHT CAPACITY: 650 LB.

Bariatric Commode Folding



ASSEMBLY INSTRUCTIONS


1. Unfold the sides of the commode frame and stand upright.
2. Attach front cross brace by depressing brass push pins and sliding brace into side frame socket. Repeat on the other side. Be sure brass push pins "snap" through holes.
3. Attach back brace by depressing brass push pins and sliding back brace into receptacle located on back of cross brace. Be sure brass push pins "snap" through holes.
4. Lower commode bucket or splash guard into bracket.
5. Attach seat and lid onto back of frame.

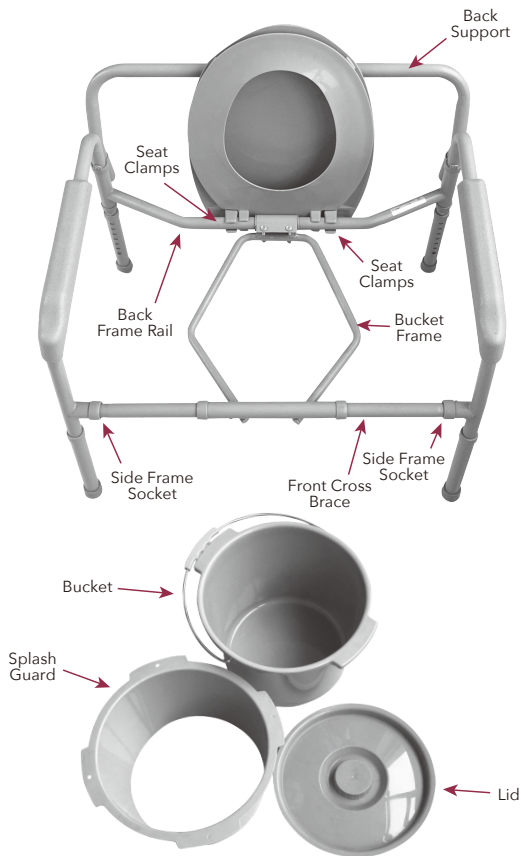

ADDITIONAL LEG HEIGHT

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes (see Figure 1).
2. Be sure legs are locked into place by pushing down on or by pulling legs.

FIGURE 1

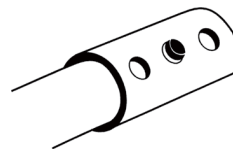

⚠ WARNINGS

- **DO NOT** use commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a healthcare dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
- Make certain that the height adjusting snap buttons fully protrude through the respective hole.
- Ensure that all components of the commode are secure and locked in place.
- Check the rubber tips on the legs for rips, wear or if they are missing. Immediately replace any or all if any imperfections exist.
- All four leg extensions with rubber tips **MUST** be level to the floor.
- Users with limited physical strength should be supervised or assisted while using commode.