

DynaRest™ Alternating Pressure Pump and Pad System



USER MANUAL

INTENDED USE

The DynaRest™ Alternating Pressure Pump and Pad System is intended to reduce the incidence of pressure ulcers while optimizing patient comfort.

PRODUCT FUNCTIONS

Pump

The functions of the pump are described below. Please refer to the figures of the pump.

(1) Power Switch

Turn ON/OFF the power, the pump will start/stop operation.

(2) Pressure Adjust Knob

Turn the soft/firm knob to set a comfortable pressure level.

(3) Air Pressure Valve

Connect the air tubes securely into the air pressure valve of the pump.



OPERATING INSTRUCTIONS

- Place pump on horizontal surface or hang it by means of the hook.
- Open the bubble pad on to the bed. Secure the pad to your mattress by tucking in the end flaps under your mattress at the head and foot areas.
- Connect air tubes securely into the air pressure valve of the pump. Ensure that the tubes are airtight and not kinked or twisted. Allow space for the bed rails to drop freely.
- Plug the pump into a suitable electrical power source and switch on.
The switch indicator should now illuminate.
- Allow 20-30 minutes for mattress to inflate fully.
- Cover with a sheet and tuck loosely.
- Use pressure adjustable knob to give maximum patient comfort.

ENVIRONMENT REQUIREMENTS

- Operation Temperature: 50°F~95°F (10°C~35°C)
- Storage Temperature: 5°F~122°F (-15°C~50°C)
- Operation Humidity: 20%~80% non-condensing
- Storage Humidity: 10%~90% non-condensing

CLEANING GUIDELINES

- Can be cleaned by wiping with a disinfectant solution.
- Do not immerse the pump in water.
- Do not heat or steam in autoclave.
- The mattress is aerated until it's fully dry.

TECHNICAL SPECIFICATIONS

Pump	Mattress
Power supply: 120V/60Hz	Size: 280 cm (L) × 90 cm (W) × 6.5 cm (H)
Air output: 5 liter/min.	130 Bubble cells
Pressure range: 70 mmHg – 110 mmHg	Without vents and with end flaps
Cycle time: 10 min.	Material: PVC (EN-71, low toxic)
Power consumption: 7W	Color: beige
Fuse: 1A	Repair kit included
Size: 29 cm (L) × 12 cm (W) × 10 cm (H)	
Weight: 1.4 kg	

WARRANTY

Your Dynarex DynaRest™ Alternating Pressure Pump and Pad System is warranted to be free of defects in materials and workmanship for One (1) year on the Pump and for Ninety (90) days on the Air Mattress, from original date of purchase, for use by the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

The Warranty shall not apply under the following conditions:

Problems arising from normal wear; Problems arising from failure to adhere to the product instructions; Problems arising from misuse, negligence, accident or improper operation, maintenance or storage; Problems arising from modifications or unauthorized repairs, parts or attachments; Products where the serial number has been removed or defaced; Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

 **Manufactured for:**
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com

Made in China

R190114



SYMBOL GLOSSARY
For an explanation of symbols
used in Dynarex packaging, visit
dynarex.com/symbols.php