

APPENDIX

QUICK-INFLATE PUMP Reorder **Nos. 10445-10446**

HOW TO QUICKLY INFLATE THE MATTRESS USING THE CPR VALVE & QUICK-INFLATE PUMP



 This mattress includes a Quick-Inflate Pump and nozzle attachment.



2. Attach the nozzle to the pump as shown.



3. Align the air holes on the CPR Valve. Insert the Quick-Inflate Pump into the CPR Valve. Turn on the Pump. The mattress will rapidly inflate in around 7-8 minutes.