Geri Chair Recliner \



3-Position • Blueridge













ASSEMBLY INSTRUCTIONS

- Slide the 1 piece back assembly (B) into the square tubes on the lower frame assembly (A) so that the holes in the back rest line up with the holes located on the horizontal brace of the lower frame assembly (A).
- Raise recliner lock bar (C) to adjust position of back assembly (B) until lock-release pull button (D) positively engages in storage position.
- 3. Insert screws through holes of the horizontal brace and tighten all screws with screwdriver.

NOTE: Make sure that the top screw on both sides of the coated square tubing from the back assembly are sufficiently secure before using recliner.

IMPORTANT SAFETY GUIDELINES:

- · Weight Capacity: 250 lb.
- Caster wheels should be locked during occupant transfer.



Figure 1

RECLINING INSTRUCTIONS

TO PLACE UNIT IN RECLINING POSITION

- 1. Ensure that recliner lock bar is in stored position.
- 2. Stand at either side of recliner.
- 3. Grasp armrest with one hand, and PVC push bar (F) with the other hand.
- Push down on push bar to recline the back to the first recline position. To achieve second recline position push on push bar again.
- 5. To raise upright, pull forward on push bar.
- 6. Set recliner lock bar (C).

RECLINING LOCK OPERATION

To Lock Recliner:

- 1. Place recliner into desired position.
- Pull out lock-release pull button (D) to recliner lock bar (C) from stored position.
- 3. Rotate recliner lock bar (C) down until notches are fully engaged over locking pins (E).

To Unlock Recliner:

- Lift recliner lock bar (C) up to disengage from locking pins.
- While holding lock-release pull button (D) out, rotate recliner lock bar (C) all the way up until completely against seat back.
- 3. Release lock-release pull button (D) to hold recliner lock bar (C) in stored position.

NOTE: Do not attempt to change position of chair with recliner lock bar engaged. This could cause damage to mechanism.

NOTE: The table can only be used in upright sitting position.

OPERATING INSTRUCTIONS

Articulating headrest may be adjusted to desired position by pulling forward or pushing back.

Operating Instructions for Table:

- 1. Underneath each arm, pull down and hold pull button and slide lock out keys forward on both sides.
- 2. Pick up table by circular table arm and slide into receiving tubes in front of each arm.
- 3. Table can slide on and off by keeping lock out key forward.

4. Table can be locked into one of five positions. Pull down the pull button. Hold the pull button and slide the key backwards. Proceed to the other arm and move that key backwards. Table will now be locked.

Storage:

The table stores in the small round tube underneath the left armrest, leaving the tabletop facing out.



Figure 2

MAINTENANCE/CLEANING

Every Three Months:

- · Check and clean casters.
- · Tighten nuts.
- · Check brakes on casters.
- · Check and clean lock recline mechanism.

WARNINGS/CAUTIONS

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment otherwise, injury or damage may occur.
- ALWAYS keep hands and fingers clear of moving parts to avoid injury.
- DO NOT use near an open flame or heat source, as all upholstery materials, even though treated with flame retardant, can ignite when exposed to open flame.
- Rear caster wheels MUST be locked during user transfer to or from recliner and while tilting and/or reclining or inclining (sitting up).
- · Ensure that casters are free of debris.
- DO NOT enter or exit the recliner when footrest is elevated.
- DO NOT sit on the footrest when recliner is extended. The recliner will tip and bodily injury may occur.
- DO NOT push or pull footrest to position the recliner.
- DO NOT place hands or feet into any openings when adjusting the recliner. Attendant or care provider should always verify placement of user's hands and feet prior to adjusting the recliner. Failure to do so may result in serious bodily injury.
- Operation of recliner with missing or broken footrest or recline mechanism guards may result in bodily injury.
 Periodic inspection of the footrest guard and recline mechanism guard MUST be made four times a year. Torn, cracked or otherwise damaged guards MUST be replaced prior to use. Recliner MUST be removed from service until replacement guards are installed.

- DO NOT lift the recliner by the footrest. Lifting by means of the footrest may result in injury to the user and/or damage to the recliner.
- DO NOT attempt to stop the recliner with the wheel locks, while in motion. Wheel locks are not brakes.
- Before attempting to transfer in or out of the recliner, every precaution should be taken to reduce the gap distance.
 Turn the recliner toward the object you are transferring onto. When transferring to and from the recliner, ALWAYS engage both wheel locks. Special care MUST BE taken with people that have physical limitations which may require an assistant.
- DO NOT traverse, climb or go down ramps or slopes greater than 9°.
- Test the recline feature of the recliner first without an occupant to ensure the locking lever and pins are operational. DO NOT operate the recliner if the locking lever or pins are not operational.
- ALWAYS make sure that the recliner is stable before using the recline option. Make sure the patient is properly positioned in the recliner before reclining or inclining to maintain maximum stability and safety.
- Excessive force or effort is never necessary to change the seating position of the chair. Do not operate recliner by applying force or pressure to leg rest.
- When returning the occupant of the recliner to the full upright position, more body strength will be required for approximately the last twenty degrees of incline (sitting up).
 Make sure to use proper body mechanics (use your legs) or seek assistance to avoid injury.
- Physical limitations of a user and larger occupants of the recliner can present conditions that can injure the assistant.
 When in doubt, assure that additional assistants are present before operating the recline feature.
- Check all parts for shipping damage. In case of damage, DO NOT use. Contact Carrier/Dealer for further instructions.

LIMITED WARRANTY

Your Dynarex Product is warrantied to be free of defects in materials and workmanship for a limited lifetime on the frame, and One (1) year on upholstery and all moving parts, from original date of purchase, for use by the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

The Warranty shall not apply under the following conditions:

- · Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments

- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

