Manufactured for: **Dynarex Corporation** 10 Glenshaw Street Orangeburg, NY 10962 USA • www.dynarex.com Made in China

Long-Term Care Bed dynarex



D500 5 Function Ultra Low Bed

User Manual














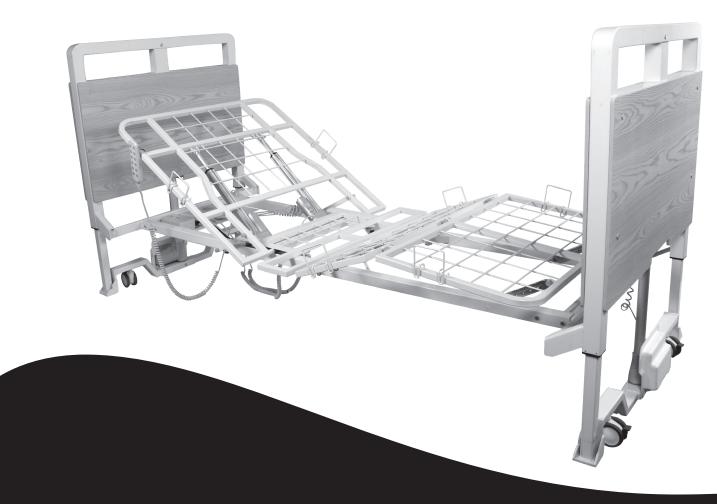

MARNINGS

- · Read all instructions before assembling or using the bed.
- DO NOT exceed maximum weight limitation of the bed.
 The maximum safe working load (SWL) of the bed is
 400 lb. The suggested patient weight is 320 lb. but
 varies based on the accessories on the bed.
- The beds are intended for users who fully understand the content of this manual and is not intended for use by children.
- DO NOT operate your bed until it is fully assembled and checked. To avoid injury, DO NOT attempt to remove the bed from the carton without assistance.
- DO NOT use accessories that are not designed or approved for use with the bed. Use only Dynarex approved parts and accessories. DO NOT make modifications to the bed without authorization from the manufacturer. Doing so will void the warranty. Serious harm or death may result from the use of improper parts or accessories.
- The bed is **NOT** a transport device. The wheels on the bed are for positioning the bed only.
- Ensure that the casters of the bed are locked at all times
 to prevent unintended bed movement. The bed should only
 be unlocked when the bed is moved. If bed is left unlocked
 unintended movement may occur which may cause damage to the resident or property damage.
- Moving the bed while the caster locks are engaged may damage the bed or the floor. Ensure all the casters are unlocked prior to moving the bed.
- · Never permit more than one person on the bed at one time.
- Ensure that the individual is properly positioned. The body weight must be evenly distributed over the sleeping surface. No body part should protrude outside the sleep surface. Caution should be used when transferring the individual to and from the bed.
- ALWAYS keep hands and feet away from the bottom of the bed and any moving parts to avoid injury. Due to the low bed clearance, extreme caution should be used as under bed space and/or moving parts can create crush or pinch points.
- DO NOT operate the bed with any items under the bed.
- ONLY use appropriate mattresses and ALWAYS adjust the mattress retainer system accordingly to ensure the mattress is fixed in place. The mattress must fit the frame properly to avoid any entrapment issues. The wrong size mattress may cause serious harm or death.
 For more information about entrapment, visit https://www.fda.gov/media/71460/download.
- Inspect the casters every six months to check for tightness and wear.
- DO NOT, under any circumstances, cut or remove the ground prong from any plug. Some devices are equipped with three-prong (ground) plugs for protection against possible shock hazards. Where a two-prong wall receptacle is encountered, it is the personal responsibility and obligation of the customer to contact a qualified electrician to have the two-prong receptacle replaced with a properly grounded three-prong wall receptacle in accordance with the regional

- and national regulations. If you must use an extension cable, use **ONLY** a three-wire extension cable having the same or higher electrical rating as the device being connected.
- Cables should be routed and secured properly to ensure they are not damaged during normal operation. DO NOT use if any cable is cut, frayed or loosely connected to the device. Cables may be damaged by inappropriate handling, e.g. by kinking, shearing or other mechanical damages. The cables are only replaceable by authorized Dynarex service personnel.
- When moving the bed ensure the power cord and pendants are secured. Ensure that the casters do not roll over the cables.
- DO NOT attempt to open the pre-sealed actuator or obtain local service, for it will VOID the warranty and might result in damage. Consult your dealer or manufacturer for further information.
- This bed is not intended to be used in an oxygen rich environment.
- DO NOT allow any part of the bed to become wet or submerged in water.
- DO NOT use near fire or explosive gases.
- The hand pendant cord may be a source of entanglement. Patients with decreased mental acuity should not have access to the pendant as serious harm or death may result. An optional staff control embedded into the footboard is available.
- For every 2 minutes of continuous use of the actuator, the bed actuator must not be in use for 18 minutes.
- The use of the Trendelenburg and Reverse Trendelenburg may have adverse medical effects. These functions should be used under the supervision of medical personnel.
- To reduce the possibility of damage to walls or other property, the ensure at least a 18" between the wall or other objects and the headboard and footboard before engaging the Trendelenburg / Reverse Trendelenburg positions.
- To engage the Trendelenburg or Reverse Trendelenburg position, raise the bed to its highest position first before making the adjustment to position. Ensure there are no obstruction around the bed before adjusting the position.

WHEN USING RAILS OR ASSIST BARS:

- Before operating, ensure that the bed rails/bars are assembled and installed correctly as instructed.
- DO NOT apply side pressure to the bed rails/bars as it can deform or damage the bed and/or bed rails/bars. Serious harm or death may result from improper use of the bed rails/bars.
- **DO NOT** use the bed rails/bar as a push handle for moving the bed. This could deform or break the bed rails.
- **DO NOT** use the bed rails/bar if proper installation cannot be achieved. Contact your dealer or manufacturer.
- When lowering the bed below 12.5" ensure the bed rails/ bars are raised. Lowering the bed below 12.5" with the bed rails/bars in a lowered position may damage the bed.

DECLARATION – ELECTROMAGNETIC EMISSIONS

Radio Frequency Interference (RFI) influences most electronic equipment. Caution should be exercised regarding the use of portable communications equipment in the area around such equipment. If RFI causes erratic behavior, shut the bed off

immediately. Leave it off while the transmission is in progress.

Interference to electronic equipment may occur in the vicinity of devices marked with this symbol:



Information about Electromagnetic Emissions

Guidance and Manufacturer's Declaration - Electromagnetic Emissions

The care bed is intended for use in the electromagnetic environment specified below. The customer or user of the care bed should ensure that it is used in such an environment.

Emitted interference	Compliance	Electromagnetic Environment - Guidelines
RF emissions according to CISPR11	Group 1	The care bed uses RF energy only for its internal functioning. Therefore the RF emissions are very low and it is unlikely that nearby electronic devices will be disturbed.
RF emissions according to CISPR11	Class B	The care bed is designed for use in all establishments including domestic establishments and those determined to be directly connected to a public supply network that supplies buildings used for residential purposes.
Emissions of harmonics according to CISPR11	Class A	
Emissions of voltage fluctuations/Flicker according to IEC 61000-3-3	Complies	

Information about Electromagnetic Interference Immunity

Guidance and Manufacturer's Declarations - Electromagnetic Interference Immunity

The care bed is intended for use in the electromagnetic environment specified below. The customer or user of the care bed should ensure that it is used in such an environment.

Interference Immunity Certification	IEC 60601 Test Level	Compliance Level	Electromagnetic Environ- ment - Guidelines
Electrostatic Discharge (ESD) according to IEC 61000-4-2	± 6 kV Contact discharge ± 8 kV Air discharge	± 6 kV Contact discharge ± 8 kV Air discharge	Floors should be wood, concrete or ceramic tile floors. If the floor is covered with synthetic material, the relative humidity should be at least 30%.
Electrical Fast Transients/Bursts according to IEC 61000-4-4	± 2 kV For power lines ± 1 kV for input and output lines	± 2 kV For power lines ± 1 kV For input and output lines	The quality of the supply voltage should be equivalent to that of a typical business or hospital environment.
Surges according to IEC 61000- 4-5	1 Kv Voltage phase- phase conductor 1 Kv Voltage phase- ground conductor	± 1 Kv Voltage phase-phase conductor 1 Kv Voltage phase-ground conductor	The quality of the supply voltage should be equivalent to that of a typical business or hospital environment.
RF emissions according to CIS-PR11	Class B		The care bed is designed for use in all establishments including domestic establishments and those determined to be directly connected to a public supply.

DECLARATION - ELECTROMAGNETIC EMISSIONS (Continued)

Information about Non Life Support Devices Electromagnetic Interference Immunity

Guidance and Manufacturer's Declarations - Non-Life-Support-Devices Electromagnetic Interference Immunity

The care bed is intended for use in the electromagnetic environment specified below. The customer or user of the care bed should ensure that it is used in such an environment.

Interference Immunity Certification	IEC 60601 Test Level	Compliance Level	Electromagnetic Environ- ment - Guidelines
Conducted RF interferences according to IEC 61000-4-6 Emitted RF interferences according to IEC 61000-4-3	3 V eff 150 kHz-80 MHz 3 V/m 80 MHz-2.5 GHz	3 V eff 3 V/m	Portable and mobile radios, including cables, should not be used closer to the care bed than the recommended working clearance that is calculated by the equation for the appropriate frequency. Where P is the Power of the transmitter in Watts (W) according to specifications of the transmitter manufacturer and D is the recommended working clearance in meters. Field strengths from fixed RF transmitters should, at all frequencies, according to a site survey, a-Note p.5 be lower than the level of agreement be b- Note.p.5. In the vicinity of equipment, bearing the following symbol, interference.

Note 1: At 80 and 800 MHz, the higher frequency range must be taken.

Note 2: These guidelines may not apply in all situations. The propagation of electromagnetic waves is affected by absorption and reflection from structures, objects and persons.

a) Field strengths from fixed transmitters, such as base stations of mobile telephones and land mobile radios, amateur radio, AM, FM radio and TV broadcast cannot be predicted with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey is recommended. If the field strength at the location of the care bed exceeds the specified compliance level above then the care bed should be monitored with respect to its normal operation. If abnormal performance is observed, it may be necessary to take additional measures, such as reorienting or relocating the care bed.

b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

DECLARATION - ELECTROMAGNETIC EMISSIONS (Continued)

Working Clearances

Recommended working clearances between portable and mobile RF communications equipment and the care bed

The care bed is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the care bed can help to prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the care bed as recommended below, according to the maximum output power of the communication device.

Output Power of Transmitter in Watts (W)	Working clearance according to transmission frequency (In meters - M)		
	150 kHz to 80 MHz at 3 V/m	80 MHz to 800 MHz at 3 V/m	800 MHz to 2.5 GHz at 3 V/m
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters not rated in the list above, the working clearance can be determined using the equation, which belongs to the transmitter, where P is the nominal output of the transmitter in Watts (W) according to specifications of the transmitter manufacturer.

Note 1: An additional factor of 10/3 is applied when calculating the recommended working clearance between transmitters in the 80 MHz to 2.5 GHz frequency range in order to reduce the probability that a mobile/portable communications device unintentionally brought into the patient area could lead to interference.

Note 2: These guidelines may not apply in all situations. Propagation of electromagnetic waves is affected by absorption and reflection from structures, objects and persons.


TECHNICAL SPECIFICATIONS

	Bed Height	2.5"-25.6"
OPERATING RANGE	Back Rest Angle	0°-70°
	Knee Break Angle	0°-27°
RATIN	Additional Manual Leg Lift	Yes, various positions
OPER	Trendelenburg/Reverse Trendelenburg	± 12°
	Auto Contour	Yes
WEIGHT	Weight Capacity	400 lb.
WEI	Safe Working Load (SWL)	400 lb.
ax (O	Sleeping Surface Dimension (Width × Length)	36" × 80"
WEIGHTS & DIMENSIONS	Caster Size	3"
VEIGH	Gross Weight	253 lb.
- 0	Net Weight	231 lb.
	Power Supply (V/PH/Hz)	100-240V/1ph/50-60Hz
	Power Consumption (Max)	200 W
LECTRICS	Number of Motors	5
	Protection Class	IPX4
3	Insulation Class	II Type B
	Duty Cycle	Interruption 10%, Max. 2 min./18min.
	Battery backup	Yes, optional
	Operating Temperature	40°F-105°F (5°C-40°C)
MENT	Storage Temperature	15°F-120°F (-10°C-50°C)
ENVIRONMENT	Operation & Storage Humidity	20%-80%
EN	Operation & Storage Atmospheric Pressure	700-1060 hPa
	Operation & Storage Altitude	≤ 3000 m
	Noise Level	< 50db(a)


ASSEMBLY INSTRUCTIONS

1. UNPACKING THE BED


- 1. Unpack the components from the shipping cartons.
- 2. Remove any black zip ties holding the components to the Bed Assembly. White zip ties should not be removed.
- 3. Upon unpacking your bed, the package should contain all the components shown below.
- 4. Inspect the bed components for damage. If the bed is damaged, do not use the bed and contact the manufacturer.

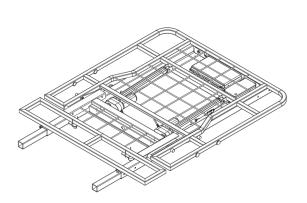


Head End Bed Deck × 1

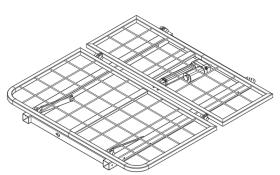

Foot End Bed Deck × 1

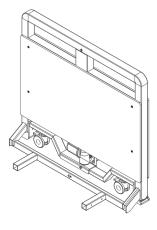

Headboard × 1

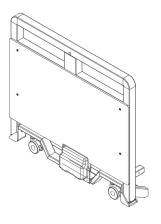

Footboard × 1



Mattress Retainer × 8



Hand Pendant Clip × 1



Transport Bar × 2



M10 Hex Bolt × 16




Allen Wrench (size 5mm) × 1

2.1 ASSEMBLING THE BED


The bed comes in with the sections (head board, foot board, upper bed deck and lower bed deck) stacked. Carefully lay each section on the floor.

2.2 ASSEMBLING THE HEADBOARD AND FOOTBOARD

- 1. Position two long pieces of cardboard supports on the floor shoulder width apart.
- Carefully tilt the head board onto the cardboard supports. Position the plastic actuator casing at the lower center of the head board in between the two supports.

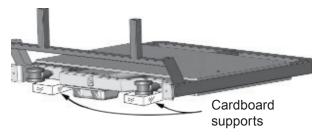



- 3. Rotate the head end bed deck vertically. Aligned and slot the bed deck onto the head board. Secure the assembly with two socket screws on each side.
- 4. Tilt the assembly back on its foot.
- 5. Repeat the same procedure for foot board installation.

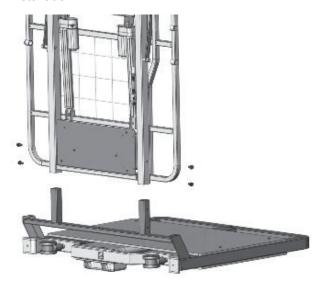

2.3 ASSEMBLING THE BED DECK


- 1. With both the head and foot end assemblies standing on their feet, align and join the two decks together.
- Secure the two decks together with two socket screws on each side.

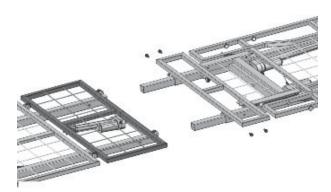

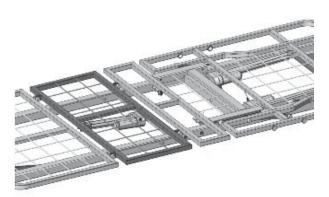


2.4 INSTALLING THE MATTRESS RETAINING SYSTEM


- Snap the mattress retainer onto the bed frame and push it down firmly. The mattress retainers for the sides will be longer than the ones for the bed ends. Ensure the mattress retainer sits flushed with the edge of the bed.
- Repeat the procedure until all six mattress retainers are installed. There should be one retainer on either side of the bed and two on both ends of the bed.

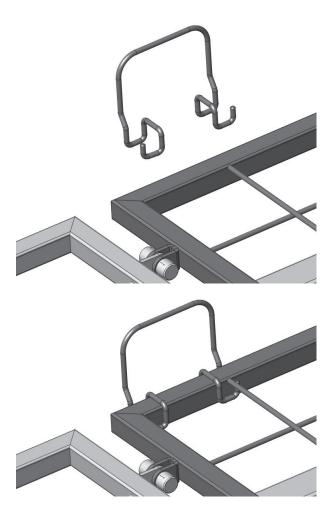

2.5 INSTALLING THE HAND PENDANT CLIP

Clip the hand pendant clip over the headboard and push it down firmly.

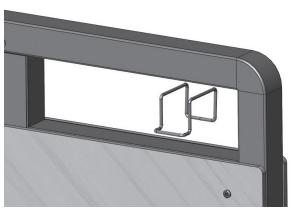


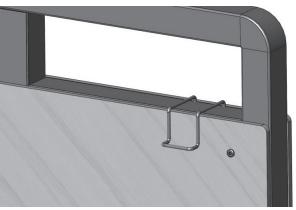

2.6 CONNECTING CABLES TO THE CONTROL BOX

Motor cables will need to be connected to the control box, located at the center of the bed. The motors for the backrest control and head section height adjustment should already be connected.


1. Open the control box locking lid by pressing in the sides and lift it up, to expose the connection ports.

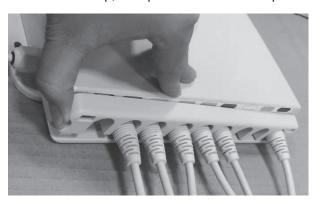

 The cables and the port connections are colorcoded. Route the cables to their matching port. When running the cables, ensure they are neatly tied along the bed frame and do not interfere with the bed operation.




- 1. Backup Battery (if applicable)
- 2. Backrest Motor 1
- 3. Backrest Motor 2
- 4. Headboard Motor
- 5. Footboard Motor
- 6. Knee-break Motor
- 7. Underbed Lighting (if applicable)
- 8. Nursing Control (if applicable)
- 9. Hand pendant

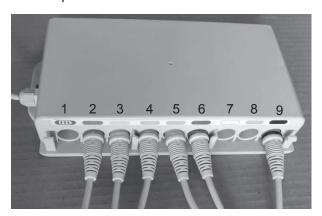

Connect all cables and reinstate the locking lid, ensure there is an audible click.

2.7 INSTALLING BATTERY BACKUP

- 1. Switch the power supply off when installing the backup battery.
- 2. Affix the backup battery with nuts and bolts to the left of the control box, on the same holding plate.

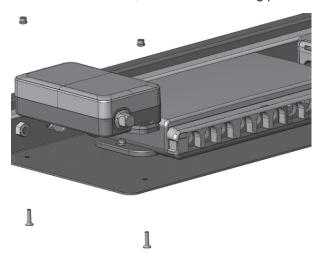

- 3. Open the control box locking lid by pressing in the sides and lift it up, to expose the connections ports.
- Connect the backup battery cord to the control box. The connection port is indicated with a battery symbol above.
- 5. Close the locking lid. There should be an audible click. Switch the power supply back on.

2.8 INSTALLING THE RAILS OR ASSIST BARS

- 1. Before installing the rails or bars, the bed should be in the flat position.
- 2. Unclip the metal snap ring and remove the metal pins from the rail mounting brackets.



3. Remove cover for the appropriate holes.



4. Place the rail bracket onto the edge of the bed decking. Align the rail bracket mounting holes with the bed deck mounting holes.



5. Re-insert the metal pins from the outside edge of the bed through the mounting holes and secure with the snap rings on the inside edge of the bed. Make sure the snap rings are positioned in the groove of the metal pins.




- 6. If the bed rail or bar has additional allen screws, tighten the allen screws next to the metal pins.
- 7. Repeat the same procedure to install the other rail.

EQUIPMENT OPERATION

1. POSITIONING THE BED


To position the bed unlock the casters. Once the bed is in its desired location, lock all casters in place.

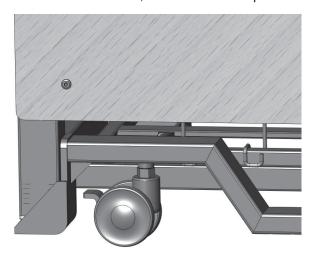

2. USING THE HAND PENDANT

The UP arrow (\triangle) indicates raising the corresponding parts of the bed.

The DOWN arrow (∇) indicates lowering the corresponding parts of the bed.

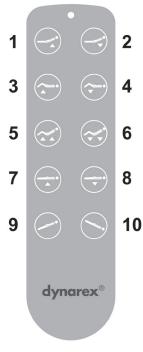

Use button 1 and 2 to adjust the backrest up or down.

Use button 3 and 4 to adjust the kneebreak up or down.

Use button 5 and 6 to adjust the backrest and kneebreak up or down simultaneously. (Auto Contour feature)

Use button 7 and 8 to raise or lower the bed platform.

Use button 9 and 10 to tilt the bed into reversed Trendelengburg and Trendelenburg positions.

2.1 ENTRAPMENT SAFETY MEASURE

The bed is equipped with an Entrapment Safety Measure. If the bed platform is higher than 12.5" from the floor, long press button 8 will lower the bed to 12.5". To continue lowering the bed, release and long press button 8 again. Ensure no body parts or obstacles are within the bed's moving range. Ensure the bed rails/bars are raised.

2.2 RESETTING THE BED

Where a connection fault is identified and rectified, the control system will require a reset in order to operate again. If the fault occurred on the motors for platform profiling, lower the corresponding motor to the lowest position for reset; where the fault occurred on the motors for bed height adjustments, raise the bed to the highest position for reset.

If the bed is not functioning, follow these simple steps below.

- Connect all components according to the system diagram.
- 2. Turn on unit.
- Check that all the Power indicator light on the control box is lit.
- Reset all functions before use:
 Back Reset Press and hold button #2 for 10 seconds (nothing will happen)
 Leg Reset Press and hold button #4 for 10 seconds (nothing will happen)
 Lift Reset Press and hold button #7 to raise the bed to the highest position and continue to hold for 10 seconds (blinking will stop)

EQUIPMENT OPERATION (Continued)

3. ADJUSTING THE EXTENDED HEEL LIFT

The end foot section can be adjusted manually to further alter the kneebreak angle.

- Grip the end section of the bed platform. Lift up slowly to the preferred height. Ensure there is an audible click indicating that the deck is locked in place within the Ratcheting Bracket.
- 2. To lower the end deck to its original position, lift up to the highest position and slowly lay the bed platform back down.



4. FOLDING AND TRANSPORTING THE BED

The bed can be disassembled and transferred in a compact manner using the transport bars.

- Reverse the bed installation procedure to disassemble the bed into four sections, the headboard, the footboard, the upper and lower bed decks. Keep the screws and washers.
- 2. Position the transport bar with the deck connection brackets facing upwards. Insert a transport bar on each of the headboard connection brackets.
- 3. Insert the footboard into the other side of the transport bar.
- 4. The folded bed can be transported with the casters on the head and footboard. Reverse the procedure to reassemble the bed.



EQUIPMENT OPERATION (Continued)

5. OPERATING THE BED RAILS

METAL SWING RAIL (REORDER NUMBER 12950)

Raise Bed Rail: Pull the bed rail upwards until the rail is securely locked in place.

Lower Bed Rail: Squeeze the release lever at the bottom corner of the bed rail and slowly lower the bed rail to its lowest position.



COMPOSITE SWING RAIL (REORDER NUMBER 12951)

Raise Bed Rail: Pull the bed rail upwards until the rail is securely locked in place.

Lower Bed Rail: Press the release button and slowly lower the bed rail to its lowest position.



ASSIST BAR (REORDER NUMBER 12952)

Parallel Assist Bar Alignment: Lift up and rotate the bed rail until it is parallel to the bed edge and release the bed rail. Ensure it is securely locked in place.


Perpendicular Assist Bar Alignment: Lift up and rotate the bed rail until it is perpendicular to the bed edge, then release the bed rail. Ensure it is securely locked in place.



PIVOT ASSIST BAR (REORDER NUMBER 12953)

Raise Bed Rail: Pull the bed rail upwards until the rail is securely locked in place.

Lower Bed Rail: Pull the release knob and slowly lower the bed rail to its lowest position.

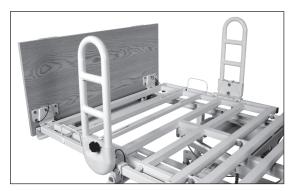

TROUBLESHOOTING GUIDE

SYMPTOMS	POSSIBLE FAULTS	POSSIBLE SOLUTIONS
Bed idle when hand pendant buttons are pressed.	Adjustment may be at the maximum or minimum position.	Check if other buttons are working.
	Power cord not connected or damaged.	Ensure the power cable is properly connected to the electrical socket and the bed.
	Hand pendant or actuator connectors are loose.	Ensure tight connection of all connectors to control box.
	Load is too heavy.	Ensure the load weight is within the designated capacity. Reduce the load.
	Control system has not been reset after faults were fixed.	Reset the bed (see page 10).
Blinking Remote	Power on but motors are not functioning	Reset the bed (see page 10).
Adjustment via hand pendant or Staff Control only partially occurs and stops.	The bed may be overloaded.	Ensure the load weight is within the designated capacity. Reduce the load.
	The bed may be constrained from moving.	Check for obstructions.
	Actuators connection may be loose or damaged.	Ensure actuator is properly secured to the bed.
	Thermal shut down may have occurred.	Wait at least 20 minutes before trying the functions again. The bed is intended to be used for 2 min every 20 min.
	If battery backup is installed, the battery may be low.	Plug bed into appropriate power source.
Bed not responding to hand pendant or Staff Control.	The lockout function may be engaged.	Ensure that the lockout function is not engaged on the Staff Control.
	Cables may be loose.	Ensure all cables are fully inserted.
		Ensure the bed is plugged into appropriate outlet.
		Unplug bed for 60 seconds and plug back in.
	Thermal shut down may have occurred.	Wait at least 20 minutes before trying the functions again. The bed is intended to be used for 2 min every 20 min.

TROUBLESHOOTING GUIDE (Continued)

Casters/Brakes noisy or stiff.	Debris or fluff in bearings.	Clean or replace casters.
Noisy or dry sound from pivot points.	Needs lubrication.	Lubricate your bed.
Unusual noise from actuator.	Actuator is worn, damaged or spindle is bent.	Replace the actuator. Contact your supplier.

LIMITED WARRANTY

Your Dynarex Product is warrantied to be free of defects in materials and workmanship for Fifteen (15) years on structural Steel, Three (3) years on electrical and mechanical, and One (1) year on all other parts and components from the original date of purchase. This item was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- · Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments

- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

SERVICE RECORD

DATE	PERFORMED BY	CONDITION REPORT

SERVICE RECORD

DATE	PERFORMED BY	CONDITION REPORT