ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The user should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

The Knee Immobilizer provides effective preoperative, postoperative or post-injury knee immobilization. It is comfortable to wear, and it provides durable, rigid immobilization, also suitable for Patella dislocations and Pre-operative pain conditions.

PERFORMANCE CHARACTERISTICS:

Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS:

Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS:

Do not use over open wounds. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. Do not use this device if it was damaged and/or package has been opened.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

1. Remove medial and lateral stay pockets. Align the posterior stays with the curve of the back of the leg. (Fig. 1)
2. Position the medial & lateral stays along the midline of the leg. Foam shell may be trimmed with scissors if necessary. (Fig. 2)
3. Starting at the bottom or distally, secure all straps and tighten. (Fig. 3)
4. Straps may be adjusted for a secure fit. Straps should be snug but not constrict.

CLEANING INSTRUCTIONS:

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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