NitriDerm[®] Ultra Blue

Nitrile Exam Gloves Series 157

Exceptionally soft, low modulus nitrile synthetic compound is very comfortable for extended periods. Very good donning properties. Slightly tacky exterior finish provides excellent gripping properties. Tested to provide maximum protection from fentanyl exposure.

Characteristics: Non-Sterile, Powder-Free, Blue Color

SIZE	ITEM NO.
XS	157050
S	157100
М	157200
L	157300
XL	157350
XXL	157400

XS-XL: 100 Gloves/Box. 10 Boxes/Case

XXL: 80 Gloves/Box, 10 Boxes/Case



Features:

- Flat Stacking = Gloves Dispense Easily, One by One
- Non-Latex = No Latex Allergens
- Finger Textured = An Improved Wet/Dry Grip
- Low Modulus = A Softer, More Comfortable Fit

Tested for use with Chemotherapy Drugs

Manufactured in compliance with multiple international standards, including ASTM D6319, ASTM D5151, and ASTM F1671

Tested for use with Chemotherapy Drugs as per ASTM D6978 Standard Practice for Assessment of Medical Gloves to Permeation by Chemotherapy Drugs Minimum Breakthrough Detection Time in Minutes:				
Azacytidine (25.0 mg/ml)	>240		Ifosfamide (50.0 mg/ml)	>240
Carboplatin (10.0 mg/ml)	>240		Irinotecan (20.0 mg/ml)	>240
Carmustine (3.3 mg/ml)	23.3		Methotrexate (25.0 mg/ml)	>240
Cisplatin (1.0 mg/ml)	>240		Mitomycin C (0.5 mg/ml)	>240
Cyclophosphamide (20.0 mg/ml)	>240		Mitoxantrone (2.0 mg/ml)	>240
Dacarbazine (10.0 mg/ml)	>240		Oncovin (1.0 mg/ml)	>240
Docetaxel (10.0 mg/ml)	>240		Oxaliplatin (5.0 mg/ml)	>240
Doxorubicin Hydrochloride (2.0 mg/ml)	>240		Paclitaxel (6.0 mg/ml)	>240
Epirubicin (2.0 mg/ml)	>240		Thiotepa (10.0 mg/ml)	58.2
Etoposide (20.0 mg/ml)	>240		Vincristine Sulfate (1.0 mg/ml)	>240
Fluorouracil (50.0 mg/ml)	>240		Vinorelbine (10.0 mg/ml)	>240
Gemcitabine (38.0 mg/ml)	>240	Also Tested:	Fentanyl Citrate (100 mcg/2ml)	>240

Please note that Carmustine and Thiotepa have extremely low permeation times of 23.3 minutes and 58.2 minutes respectively
Caution: In compliance with ASTM D6978, the testing conditions used are intended to approximate the
worst-case conditions for clinical use. Testing was conducted on single layer glove material. It is the users'
responsibility to determine the applicability of these gloves for their intended use with chemotherapy drugs.

Tensile Strength & Elongation	Before Aging	After Accelerated Aging
Tensile Strength (Mpa)	28.3	28.7
ASTM Requirement Min. (Mpa)	14	14
Elongation (%)	657	582
ASTM Requirement Min. (%)	500	400

Average Length	9.5 in ◆ 240 mm
Average Palm Thickness	3.5 mil ◆ 0.09 mm
Average Finger Thickness	4.4 mil ◆ 0.11 mm





Product Solutions You Trust

View this product on our website:

