


NITRO (RTL10266, RTL10266BK, RTL10266WT & RTL10266RG)

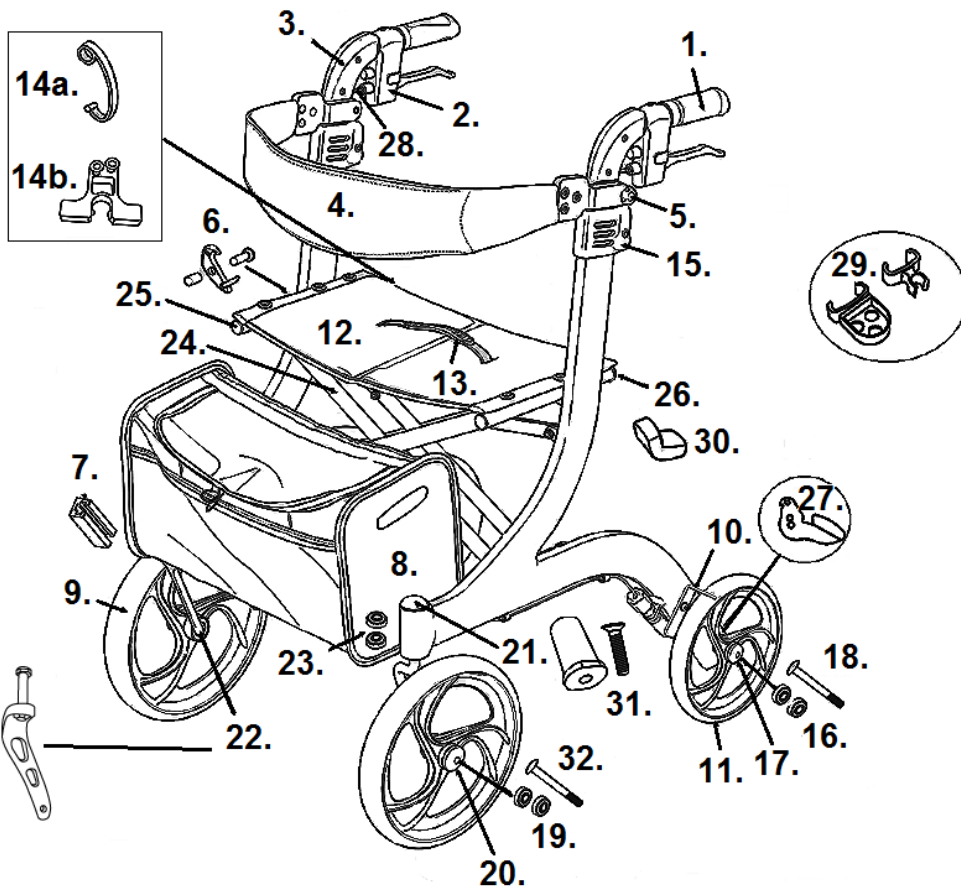

- | | |
|--|--|
| 1. Handgrip Left- 1026602
Handgrip Right- 1026603 | 12. Seat Upholstery & Hardware Black- 1026608
Seat Upholstery & Hardware Grey- 1026608-G |
| 2. Handbrake Side Mount 10266 Left- 1026601-RN [4]
Handbrake Side Mount 10266 Rt- 1026601-RNR [4]
Handbrake Side Mount 10266BK Left- 1026601-G [4]
Handbrake Side Mount 10266BK Rt- 1026601-GR [4]
Handbrake Side Mount 10266WT Left- 1026601-W [4]
Handbrake Side Mount 10266WT Rt- 1026601-WR [4]
Handbrake Side Mount 10266RG Left- 1026601-W
Handbrake Side Mount 10266RG Rt- 1026601-WR
Cable Only 48"- 1026632 [4] | 13. Folding Strap Assembly 10266- 1026609-FSR
Folding Strap Assembly 10266BK- 1026609-FSG
Folding Strap Assembly 10266WT- 1026609-FSW
Folding Strap Assembly 10266RG- 1026609-FSW |
| 3. Handlebar Side Mount (ea)- 1026630N [4] | 14a. Folding Locking Latch 10266- 1026626-R
Folding Locking Latch 10266BK- 1026626-G
Folding Locking Latch 10266WT- 1026626-W |
| 4. B/rest Assy. 10266, 10266WT & 10266RG- 1026604-B
Backrest Assembly 10266BK- 1026604-G | 14b. Folding Locking Latch- 1026644 [1] |
| 5. Backrest Knob & Bolt 10266- 1026605-R
Backrest Knob & Bolt 10266BK- 1026605-G
Backrest Knob & Bolt 10266WT- 1026605-W
Backrest Knob & Bolt 10266RG- 1026605-W | 15. Back Height Adjustment Latch 10266- 1026607-R
Back Height Adjustment Latch 10266BK- 1026607-G
Back Height Adjustment Latch 10266WT- 1026606-W
Back Height Adjustment Latch 10266RG- 1026606-W |
| 6. Safety Lock- 1026638 | 16. Rear Axle Bearings (pr)- 1026613 |
| 7. Side Frame Bag Mount & Screws (pr)- 1026611 | 17. Rear Hub Cap- 1026619 |
| 8. Bag Side Frame Mount 10266BK- 1026610-G
Bag Side Frame 10266- 1026610-R
Bag Side Frame 10266WT- 1026610-W
Bag Cross Frame Mount 10266- 1026633-R [5]
Bag Cross Frame Mount 10266BK- 1026633-BK [5]
Bag Cross Frame Mount 10266WT- 1026633-WT [5]
Bag Cross Frame Mount 10266RG- 1026633-WT | 18. Rear Axle- 1026631 |
| 9. Caster & Fork Assembly Style #1 Left- 1026625
Caster & Fork Assembly Style #1 Right- 1026624
Caster & Fork Assembly Style #2 Lt- 1026625-AN [7]
Caster & Fork Assembly Style #2 Rt- 1026624-AN [7]
Caster Style #1- 1026617
Caster Style #2- 1026617-AN [3] | 19. Caster Bearings Old Style (pr)- 1026618
Caster Bearings New Style (pr)- 1026618-AN [3] |
| 10. Wheel Lock Assembly Metal Left- 1026615
Wheel Lock Assembly Metal Right- 1026616
Wheel Lock Assembly Plastic Left- 1026641 [2]
Wheel Lock Assembly Plastic Right- 1026642 [2] | 20. Caster Hub Cap Old Style- 1026614
Caster Hub Cap New Style- 1026614-AN [3] |
| 11. Rear Wheel Left- 1026612-L
Rear Wheel Right- 1026612-R | 21. Dust Cap (pr)- 1026620 |
| | 22. Fork Assembly Old Style Left- 1026622
Fork Assembly Old Style Right- 1026621 |
| | 22. Fork Assembly New Style Left- 1026622-AN [3]
Fork Assembly New Style Right- 1026621-AN [3] |
| | 23. Fork Stem Bearings (pr)- 1026623 |
| | 24. Cross Frame Side Frame Mt Bag- No longer available.
Cross Frame Cross Frame Mount Bag- 1026635 [5]
Cross Frame Bolt- 1026646 |
| | 25. Front End Cap Side Frame Mt Bag (pr)- 1026628
25. Front End Cap Cross Frame Mt Bag (ea)- 1026634 [5] |
| | 26. Cross Frame Rear End Caps (pr)- 1026628 |
| | 27. Step Tube Left- 1026636
Step Tube Right- 1026637 [6] |
| | 28. Brake Cable Guide- 1026639 |
| | 29. Cane Holder- 892 |
| | 30. Seat Rail Pad- STDS2A1807 |
| | 31. Cross Brace Screw & Insert- 1026643 |
| | 32. Front Axle- 1026645 [3] |

[1] Serial # 2A1606300234 & higher

[2] Serial # 2A1510302380 & higher

[3] Serial # 2A1606301542 & higher

[4] Serial # 2A1301..... & higher

[5] Serial # 2A1406..... & higher

[6] Serial # 2A14081025.. & higher

[7] Red Nitro Serial # 2A1606301542 to 2A1909301640

White Nitro Serial # 2A1606301542 to 2A1909301841

Black Nitro Serial # 2A1606301542 to 2A1909301841