

instructions

# FabTRAC<sup>®</sup>

## CERVICAL TRACTION




- 1 Pick up head halter by the two metal 'D' rings on overhead straps, holding them with one hand.



- 2 Place head through the middle of the head halter with chin resting in chin cup. The Velcro loop straps hanging at the back of the head halter should be on the outside of the halter.



- 3 Fasten Velcro loop straps to the Velcro hook on the chin cup.

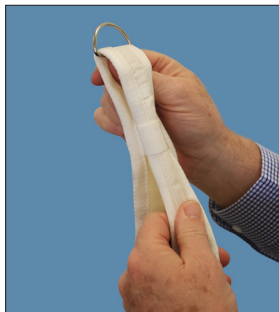

- 4 Adjust head halter size by moving overhead straps up and down at 'D' rings.



# FabTRAC®

## CERVICAL TRACTION



Fabrication Enterprises Inc.  
250 Clearbrook Rd, Suite 240  
Elmsford, NY 10523 (USA)  
tel: +1-914-345-9300 • 800-431-2830  
fax: +1-914-345-9800 • 800-634-5370  
FabEnt.com



AJW Technology Consulting GmbH  
Breite Strasse 3  
40213 Düsseldorf (Germany)