


Deluxe Head Immobilizer

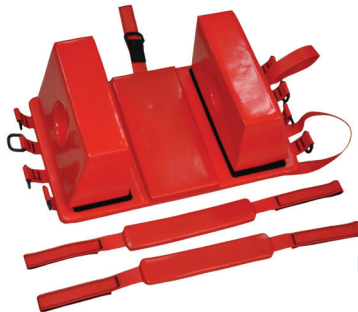

- STEP 1** Disassemble the head immobilizer by removing both support pads and loosening all straps. Position baseplate below the center hole at the head-end of the backboard. If the backboard does not have a center hole, position the backboard accordingly.
- STEP 2** Fasten the straps by running them through the slots along the side of the backboard and placing them directly to the securing strips. If there are no slots, run straps underneath backboard and fasten.
- STEP 3** If the backboard does have a center hole located at the head-end of the backboard, run the top strap through the hole and loop back through the buckle and tighten. If there is no center hole, leave strap fastened to itself.
- STEP 4** Make sure that all straps are tightened and that the baseplate is stationary on the backboard. Position the support pillows around the patient's head with the flat side of the pillow flush against the ears.
- STEP 5** Once the support pillows are fastened, center a forehead/chin strap over the patient's forehead and secure each side by running each end of the strap through the "D" rings and looping them back to fasten. Use of the second forehead/chin strap should be determined by an authorized professional. Intended for use only by authorized EMS personnel.



 **Manufactured for:**
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com

Made in China

For More Emergency
Medical Service Products,
go to www.dynarex.com.

© 2018 Dynarex Corporation.

R180913