


# The specialized blade for neuro and trauma.

# Safe for patients. Easy for you.

Clip more hair in a single pass and give your patients a safe, comfortable clipping experience with the widest specialty blade on the market.

 Quickly and effectively removes hair in trauma cases.

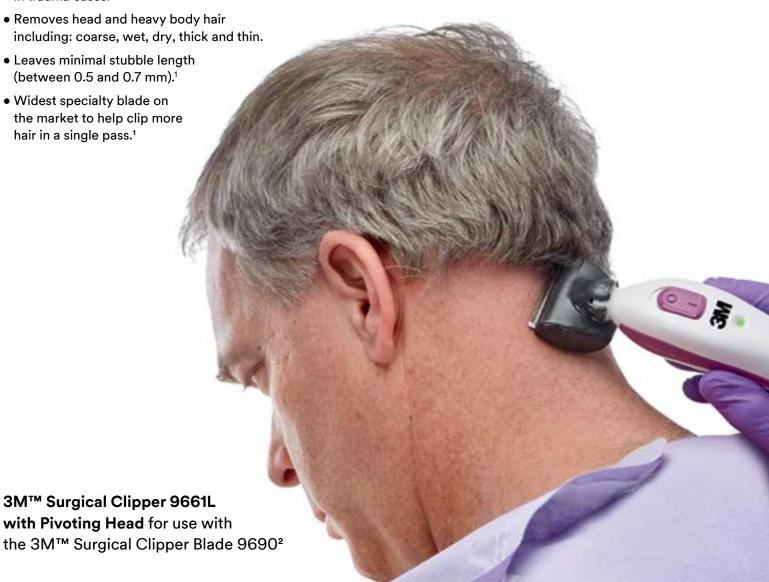

# Easy to charge, use and clean.

# Changing blades



#### **Attach**

Remove backing from the blade packaging, hold the clipper at a 45-degree angle and snap on single-use blade 9690.

Note: An audible snap indicates that the blade is securely attached.



#### Remove

Use your thumb to "pop off" the used blade into a sharps container.

Note: Always use a new blade for each patient to avoid cross-contamination.

# Clipping



#### Push

Position the clipper like a pencil with the clipper blade flat or at a slight angle on the skin surface. Gently push the clipper forward, against the direction of the patient's hair growth.

Note: Do not use excessive pressure to push the clipper down on the patient's skin.

### **Cleaning options**



#### Submerse<sup>3</sup>

With the clipper in the "off" position and the blade removed, soak the clipper in water. Ensure clipper is thoroughly dry before recharging or reusing.



#### Or rinse

With the clipper in the "off" position and the blade removed, rinse the handle under running water. Ensure clipper is thoroughly dry before recharging or reusing.



#### Then wipe

With the clipper in the "off" position and the blade removed, wipe the clipper handle with a disinfectant.

Note: If choosing to rinse or submerse, rinsing or submersing should be done prior to disinfecting.



## Charging

Before initial use, charge clipper for 4 hours. After each use, store the clipper handle in the "off" position in the plugged-in charge stand. Clipper provides approximately 160 minutes of run time on a 4-hour recharge. When charged, the light will be green. When battery needs recharging, and during recharge, the light will flash amber. Refer to package insert for warnings and more detailed instructions.

For more information about the 3M™ Surgical Clipper 9661L or other 3M Health Care products:

- Contact your local 3M representative
- Call the 3M Health Care Customer Helpline at 800.228.3957
- Visit 3M.com/medical

Catalog Number	Product Description	Items per Box	Boxes per Case
9660	3M™ Surgical Clipper Blade 9660	50	1
9661L	3M™ Surgical Clipper 9661L with Pivoting Head	1	1
9662L	3M™ Surgical Clipper Charger for 9661L	1	1
9667L	3M™ Surgical Clipper Starter Kit 9667L: 3M™ Surgical Clipper 9661L and 3M™ Surgical Clipper Charger 9662L	1	1
9690	3M™ Surgical Clipper Blade 9690 (for head and heavy body hair)	20	1



3M Health Care
3M Infection Prevention Division
3M Center, 2510 Conway Ave
St. Paul, MN 55144-1000
U.S.A.
1800 228 3957
www.3m.com/medical

<sup>1</sup>Verified by 3M testing.

<sup>2</sup>The 9690 blade, when used with the 3M<sup>™</sup> Surgical Clipper 9661L, is non-pivoting.

<sup>3</sup>Meets IPX7 rating – submersible in water no deeper than one meter for up to 30 minutes.

Please recycle. Printed in the U.S.A. © 3M 2016. 3M is a trademark of 3M. All rights reserved. 70-2011-5771-9