Welch Allyn[®]

FlexiPort[®] Blood Pressure Cuffs

Flexiport: Soft, Vinyl, and Reuse

Trimline[™] Blood Pressure Cuffs

Trimline: Soft, Reuse, Isolation

Two-Piece Reusable Blood Pressure Cuffs

Directions for use

723529 DIR 80019996 Ver. C Revised 2016-12


CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician or licensed healthcare practitioner. This device should be used by trained personnel.

Standards

This device is designed to function within the limits prescribed by:

- AAMI/ANSI/ISO 81066-1:2007 specification for non-invasive sphygmomanometers - Part 1: Requirements and test methods for non-automated measurement type

Cuff connections and symbols

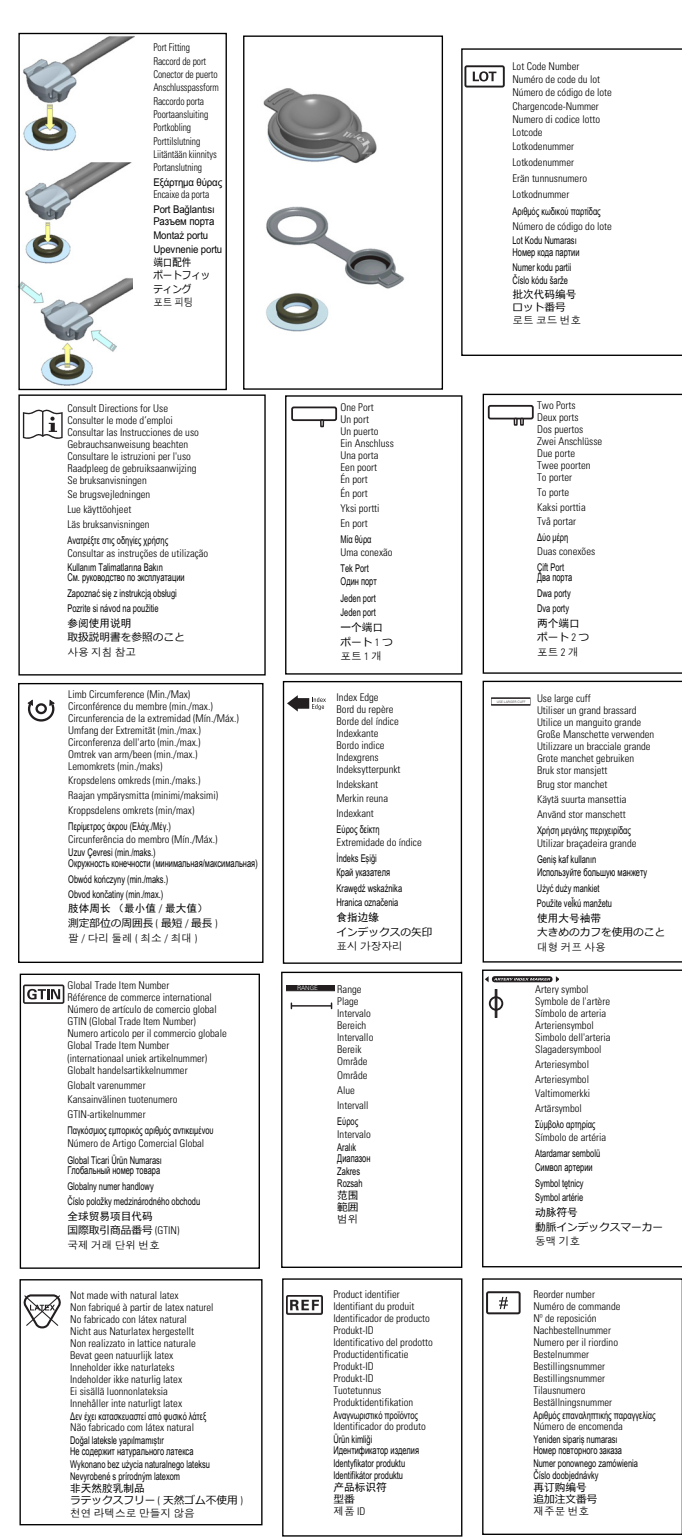ENGLISH

Introduction

Intended use

Welch Allyn FlexiPort through Adult Blood Pressure Cuffs are noninvasive blood pressure cuffs intended for use in conjunction with non-automated and automated sphygmomanometers to determine blood pressure in pediatric through adult patients.

Contraindications

Welch Allyn pediatric through adult Blood Pressure cuffs are contraindicated for neonate use.

Warnings

A warning statement in this manual identifies a condition or practice which, if not corrected or discontinued immediately could lead to injury, illness, or death.

WARNING: Possible measurement error. Use only approved blood pressure cuffs and accessories; substitution might result in measurement error.

WARNING: Inaccurate measurement risk. Only use the cuff when the artery index marker falls within the printed range indicated on the cuff, otherwise erroneous readings may result.

WARNING: Patient injury risk. Never install Luer lock connectors on Welch Allyn blood pressure tubing. Using these connectors on blood pressure cuff tubing creates the risk of mistakenly connecting this device to a patient's intravenous line and introducing air into the patient's circulatory system.

WARNING: Do not apply cuff to areas on patient where skin is delicate or damaged. Check cuff site frequently for irritation.

WARNING: Allow space for 1 to 2 fingers between patient and cuff.

WARNING: Do not apply cuff to limbs used for IV infusion.

WARNING: Minimize cuff movement and limb motion during readings.

WARNING: Ensure an airtight seal at all connection points prior to use.

Cautions

A caution statement in this manual identifies information within the manual to avoid equipment failure.

CAUTION: Do not press cuff with a hot iron.

CAUTION: Do not inflate cuff unless the hook and loop is closed.

CAUTION: Do not allow foreign debris to ingress into tubes or port on cuff.

CAUTION: Do not use steam or heat to sterilize the cuff, or tubing.

CAUTION: Do not exceed 250 mmHg with high size disposable cuffs at or above 30 °C/86 °F.

CAUTION: Intravenous Systems (IV) - Do not connect cuffs with luer lock connectors to intravenous fluid systems or fluid may enter the cuff.

Operation

Use the Welch Allyn Blood Pressure cuff the same as a traditional blood pressure cuff. The blood pressure cuff works with manual and automated sphygmomanometers.

Select cuff size appropriate for the patient's arm circumference. The applicable range, in centimeters, is printed on each cuff.

Operational Pressure Range: 0 to 300 mmHg

NOTE: The "Artery Index Marker" on the cuff should fall within the "Range" indicated on the cuff. If the artery index marker falls short of range, use a larger cuff to ensure accurate results. If the artery index marker is past the range, use a smaller cuff to ensure accurate results.

Cleaning and Low-level disinfection for One Piece Blood Pressure Cuffs (FlexiPort and Trimline) and Two-Piece Blood Pressure Cuffs

WARNING: Cleaning and disinfection procedures must be conducted by persons trained in medical device cleaning and disinfection.

WARNING: Consult the cleaning and germicidal cleaner agents' manufacturer's instructions for their proper use and germicidal efficacy.

CAUTION: Washing Plug or FlexiPort Cap must be installed into cuff tubing prior to cleaning or damage to cuff bladder may occur.

CAUTION: Use only the cleaning or germicidal cleaner agent types listed or damage to cuff may occur.

CAUTION: Repeated reprocessing may cause degradation of device; follow inspection procedures to assure integrity of device.

CAUTION: Do not aggressively scrub cuff as damage to cuff markings and/or cuff closure integrity may occur.

Materials

- Neutral pH enzymatic-based cleaning detergent.
- Bleach-based germicidal cleaner suitable for use on healthcare equipment and capable of low-level disinfection. For example: A cleaner including 1:10 dilution of bleach (6500 ppm sodium hypochlorite) and detergent. Reference EPA-registered disinfectants: <http://www.epa.gov/oppad001/chemindex.htm>
- Clean or sterile cloths, soft brush, soaking tray, and potable rinse water (softer preferred).

Cleaning and low-level disinfection for One Piece Blood Pressure Cuffs (FlexiPort and Trimline)

Preparation for cleaning

Install FlexiPort cap onto cuff port or washing plug into cuff tubing. See below for appropriate plug or cap per cuff.

- Washing Plug REF: 5082-163 for One-Piece Style Reusable, Disposable Soft and Isolation blood pressure cuffs, except Neonate
- Washing Plug REF: 5082-198 for Disposable Soft Neonate blood pressure cuffs only
- FlexiPort Cap REF: 5082-159 for FlexiPort Style Reusable, Soft and Vinyl disposable blood pressure cuffs

Cleaning only (cuffs and connector accessories)

1. Prepare neutral pH enzymatic cleaning detergent solution per manufacturer instructions.
2. Immerse or soak cuff and accessories in solution.
3. Soft brush all surfaces of the cuff and accessories to remove visible soil. Repeat as necessary.

Clean and disinfect (cuffs)

1. Clean: Thoroughly saturate (spray or immerse) all surfaces of the cuff and accessories with germicidal cleaner.
 - Soft brush to remove visible soil.
 - Water rinse.
 - Damp dry and inspect.
2. Disinfect: Thoroughly re-saturate (spray or immerse) all surfaces of the cuff and accessories with germicidal cleaner.
3. Soft brush all surfaces. Allow a 5 minute wet contact time or longer, if directed by the germicidal cleaner manufacturer. Do not exceed 10 minutes of wet contact time.

After cleaning or cleaning and disinfection

1. Thoroughly water rinse.
2. Damp dry with a clean cloth.
3. Remove Washing Plug or FlexiPort Cap and allow to air dry.
4. Inspect cuff for deterioration, adequate closure integrity, and inflate to assess for leaks. Do not use if any abnormalities are found.

Cleaning and low-level disinfection for Two-Piece Reusable Blood Pressure Cuffs

Preparation for cleaning

1. Insert Washing Plug REF 5082-163 into cuff bladder tubing.
2. Remove bladder from cuff sleeve.

Cleaning only (cuff and bladder)

1. Prepare neutral pH enzymatic cleaning detergent solution per manufacturer instructions.
2. Immerse or soak cuff and bladder in solution.
3. Soft brush all surfaces of the cuff and bladder to remove visible soil. Repeat as necessary.

Clean and disinfect (cuff and bladder)

1. Clean: Thoroughly saturate (spray or immerse) all surfaces of the cuff and bladder with germicidal cleaner.
 - Soft brush to remove visible soil.
 - Water rinse.
 - Damp dry and inspect.
2. Disinfect: Thoroughly re-saturate (spray or immerse) all surfaces of the cuff and bladder with germicidal cleaner.
3. Soft brush all surfaces. Allow a 5 minute wet contact time or longer, if directed by the germicidal cleaner manufacturer. Do not exceed 10 minutes of wet contact time.

After cleaning or cleaning and disinfection

1. Thoroughly water rinse.
2. Damp dry with a clean cloth.
3. Remove Washing Plug and allow to air dry.
4. Reinstall bladder into cuff sleeve.
5. Inspect cuff for deterioration, adequate closure integrity, and inflate to assess for leaks. Do not use if any abnormalities are found.

Environmental specifications

NOTE: Comply with regional law when non-automated sphygmomanometer or accessories are discarded.

Storage temperature 20°C to 55°C (4°F to 131°F)

Storage relative humidity 15% to 95% (noncondensing)

Operating temperature 10°C to 40°C (50°F to 104°F)

Operating relative humidity 15% to 90% (noncondensing)

Legal Statement

Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal and improper use of the product, that may result from failure to properly use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual. Welch Allyn[®] and FlexiPort[®] are trademarks of Welch Allyn, Inc.

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