## **Application Instructions Ankle Foot Orthosis**

**Notice:** This device is intended for the use only under the direction of a medical professional. If you experience any pain, swelling, sensation changes or any unusual reactions while using this product, consult your medical professional immediately.

## How to use:

1 - Sit in a chair. Place the foot into the brace as shown.





- 2 Wrap the calf strap around the leg
- 3 Put on a lace up shoe.

## Use and care:

The brace may be washed in warm water with a mild detergent. Rinse thoroughly and towel dry.

## Fitting chart

Size	Men's shoe size	Women's shoe size
small	5-7.5	3-6
medium	7.5-10	6-9
large	10-13	9+



1-888-769-6084



**EMERGO EUROPE**Molenstraat 15
2513 BH, The Hague
The Netherlands
Tel: +31 (0)70 345 8570
Fax: +31 (0)70 346 7299



Sky Medical Inc. 5229 NW 108<sup>th</sup> Avenue Sunrise FL 33351 USA