
1

2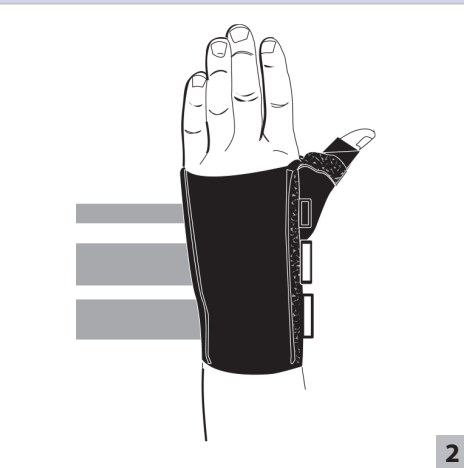

3

## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

### INTENDED USE/INDICATIONS

- Immobilisation of wrist and thumb due to tendonitis, Carpal Tunnel Syndrome (CTS) or deQuervain's syndrome.

### PERFORMANCE CHARACTERISTICS

Providing immobilization or controlled movement of the limb or body segment.

### CONTRAINDICATIONS

- None

### WARNINGS AND PRECAUTIONS

- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS

- 1) Loosen straps and slip hand into support with thumb resting in abduction stay. The palmar stay should be resting on palm and wrist area of the hand.  
**Note:** Straps are malleable and may be contoured to fit individual needs.
- 2) Wrap and secure thumb strap.
- 3) Starting at the hand, feed straps through "D" rings and secure, working up the arm.

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**NOT MADE WITH NATURAL RUBBER LATEX.**

**FOR SINGLE PATIENT USE ONLY**

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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MDSS GmbH