
**Cushion-Top Bed Rail Pads** 

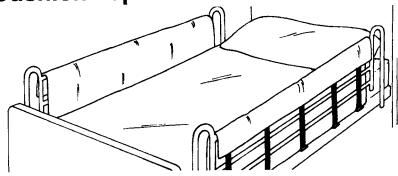


Reorder#	Description	Length	Unit
☐ 401210	Cushion-Top Bed Rail Pads	60"	Pair
□ 401220	Cushion-Top Bed Rail Pads	72"	Pair
□ 401230	Cushion-Top Bed Rail Pads	80"	Pair

## **Purpose**

A new design for patient safety that cushions the side and top of the bed rail. Protects against contact with metal side rails while reducing the injuries associated with entrapment between the rails.

## Easy to Use

- 1. Select the proper length for your bed rails—60, 72, or 80 inches. Place the pads over the rails as shown in the illustration at the top.
- 2. Thread each Velcro® strap through D-ring at bottom of the bed rail pad. Press the "hook" tab against the "loop" material to secure. See illustration at right.

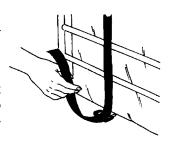

## **Easy to Care For**

Skil-Care's Cushion-Top Bed Rail Pads wipe clean with a damp cloth. A mild spray-type cleanser may be used. Do not use a cleanser that contains bleach. If lint or other particles accumulate in the Velcro® hook material, they may be removed with a stiff brush.

