

ECLIPSE®

PROBE COVER


Disposable, latex-free protective cover
for medical ultrasound probe/transducers.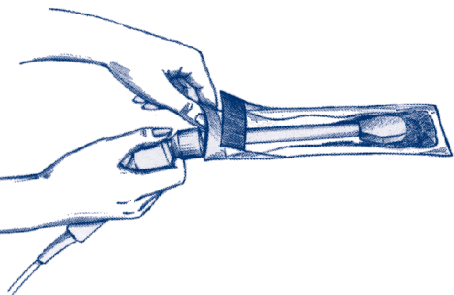

Hold inner white tab while inserting a clean, dry probe/transducer head into pre-gelled ECLIPSE Probe Cover. Gently slide probe until the head reaches the tip of the probe cover.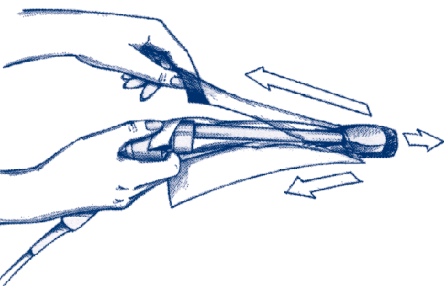

Hold top printed blue tab and gently pull towards you to release the protective outer packaging.

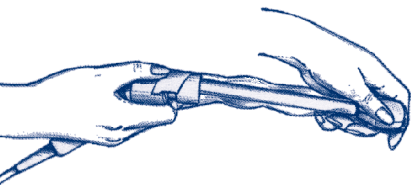Smooth ECLIPSE Probe Cover over the probe/transducer head to eliminate air bubbles and cover the probe lens with gel, (if air bubbles remain in lens area, gently smooth them away with your thumb). The probe/transducer is now protected and ready for normal use.

For use as a protective cover for medical ultrasound probes/transducers. For professional use only. Use Protex® Ultra Wipes or Protex® Spray to pre-clean probes/transducers prior to high-level disinfection. Refer to ultrasound equipment manufacturer's instructions for cleaning ultrasound probe/transducer between patient uses. This product is not sterile and is not intended to be sterilized.

PARKER LABORATORIES, INC.
286 Eldridge Road
Fairfield, NJ 07004 USA

Phone: (973) 276-9500
Fax: (973) 276-9510
Web: www.parkerlabs.com

Toll Free: (800) 631-8888
E-mail: parker@parkerlabs.com

ECLIPSE®

PROBE COVER

Eclipse® Probe Cover - Latex-free protective cover for medical ultrasound probes/transducers, pre-gelled inside with Aquasonic® 100 Ultrasound Gel. ② For Professional Use only.
Available: 3.25"/1.73"W x 9.5"L (83mm/44mm x 241mm) and
2.5"/1.73"W x 9.5"L (64mm/44mm x 241mm)

⚠ This product is not sterile and is not intended to be sterilized.

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