


STEP 1

Open up the Elbowlift so that the hook-and-loop strap is pulled away from the suspension pad.

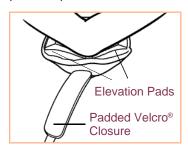


Care of Elbowlift:

When Elbowlift becomes soiled, hand washing is recommended.


Machine washing Elbowlift in a standard washer is an acceptable alternative to hand washing. The hook-and-loop strap should be closed and placed into a net laundry bag prior to machine washing.

Application Guide

STEP 2 Place tip of patient's elbow over the centering hole with the foam insert pads aligned under the arm and forearm.

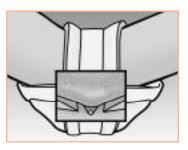

STEP 3 Close the strap across the elbow using the hook and loop closure. Fit the Elbowlift so it is snug but not tight.

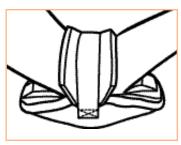

IMPORTANT: Remove Elbowlift every eight hours and check the skin or wound status.

For the prevention or treatment of heel ulcers, use and and





Suspension Boots.



DM Systems, Inc.

1316 Šherman Ave. • Evanston, IL 60201 Tel: 800-254-5438 • Fax: 847-328-9561 E-mail: info@dmsystems.com • www.elbowlift.com

© 2001, DM Systems, Inc. All rights reserved.