

Percussors/Accessories

Applicator #54-AP209

Right Angle Directional-Stroking® Adaptor for Directional-Percussion™ and Directional-Stroking®

Applicator #54-AP209 is used when a full percussive effect from the entire surface of an applicator is desired. This full percussive effect is obtained because the face plate of the head assembly describes a circular motion imparted to it by an eccentric cam inside the head. By screwing one leg of the percussion adaptor into the face plate, and affixing an applicator to the other, the oscillatory effect produced by the face plate is changed into a Directional-Stroking/Directional Percussion effect.

Never use this Directional-Stroking® adaptor at speeds in excess of 35 CPS.



Applicator #54-AP210

Soft Sponge Rubber 2-5/8" Diameter

Applicator #54-AP210 is a small sponge applicator designed for use on children or small, sensitive body parts. It is ideal for gentle massage of tender or delicate areas.



Applicator #54-AP212

Soft Sponge Rubber 3-1/2" Diameter

Applicator #54-AP212 is a large sponge applicator designed for gentle massage over sensitive body parts.



Applicator #54-AP214

Fine sponge with Plastic Back

Applicator #54-AP214 includes one fine sponge and the plastic backing that attaches to the head assembly.

Applicators #54-AP214FS and #54-AP214FC are fine and coarse sponge replacements designed for exfoliation. Each ordering code includes ten (10) sponges.

Applicator #54-214FS or #54-214FC



Applicator #54-AP215

Multiple Prong Applicator

Applicator #54-AP215 is designed for vigorous massage and surface stimulation over large body areas.



Applicator #54-AP216

Four-Ball Firm Rubber

Applicator #54-AP216 is designed for deep, heavy-pressure massage of major muscle groups such as back, thighs, shoulders and buttocks.



Applicator #54-AP217

Half-Ball Firm Rubber

Applicator #54-AP217 is a large, half-ball designed for working body depressions and for localized effects in postural drainage. It is most effective when applied to the soles of the feet or hands to stimulate circulation in the limbs, and to relax muscles.



Applicator #54-AP222

Blunt-Tipped Firm Rubber

Applicator #54-AP222 is designed for working body depressions and the soles of the feet and hands. It can also be used for trigger-point therapy.



Applicator #54-AP223

Two-Ball Firm Rubber

Applicator #54-AP223 is used in a manner similar to applicator #54-AP216, but because of its narrower width, it fits more easily over the cervical and is well suited to straddle the spinal column, when used for back massage. It is well-adapted for manipulation of the smaller joints, because the balls can be worked into the flesh while the oval plate connecting the balls is high enough to override intervening bony structures.



Applicator #54-AP225

Skin and Scalp Surface

Applicator #54-AP225 is designed for vigorous massage and surface stimulation over large body areas and the scalp.



Applicator #54-AP227

Pointed-Tip Firm Rubber

Applicator #54-AP227 is designed to reduce trigger points; for reflexology; and for wherever the thumb would be used in muscle-goading techniques.



Applicator #54-AP229

Large Round Firm Rubber

Applicator #54-AP229 is 5" in diameter and designed for deep massage of the back, buttocks and thighs. Excellent for postural drainage when used with #54-AP209 (Right Angle Directional-Stroking® Adaptor).



Applicator #54-AP230

Curved Flex Sponge & Frame

Applicator #54-AP230 is designed for use on the neck, shoulders, legs, arms, back, and buttocks. Extend foam life by using #54-AP236 or #54-AP236D applicator covers. The Curved Flex Sponge Rubber is available by itself as #54-AP231. The frame is also available by itself as #54-AP232.



Applicator #54-AP231

Curved Flex Rubber Sponge

Applicator #54-AP231 is the sponge part of Applicator #54-AP230.



Applicator #54-AP232

Curved Flex Sponge Frame

Applicator #54-AP232 is the plastic frame part of Applicator #54-AP230. Not to be used without Applicator #54-AP231.



Applicator #54-AP242

Suction Applicator

Suction applicator for use in reducing cellulite. To be used with Applicator #54-AP209 and with the GX-99®, Cellutec®, Contour®, Fleximatic®, or TherAssist®.



Applicator #54-AP250

Vibration Applicator

For use only with the Flimm-Fighter® or Vibracare®.



Applicator #54-AP251

Adult Soft Cupping Applicator

For use only with the Flimm-Fighter® or Vibracare®.



Applicator #54-AP252

Pediatric Soft Cupping Applicator

For use only with the Flimm-Fighter® or Vibracare®.



Applicator #54-AP253

Soft Foam Applicator

For use only with the Flimm-Fighter® or Vibracare®.



Percussors/Accessories



Applicator #54-AP331

Adaptor Ring

Applicator #54-AP331 is designed to protect applicator threads against wear and tear. This applicator is included with Applicator #54-AP209 (Right Angle Directional-Stroking® Adaptor).

Applicator #54-AP509

Micro Right Angle Directional-Percussion® Adaptor

Applicator #54-AP509 is used when a full percussive effect from the entire surface of an applicator is desired. By screwing on leg of the percussion adaptor into the faceplate, and affixing an applicator to the other, the oscillatory effect produced by the faceplate is changed into a full percussion effect.

For use with the G5® Microhead™ only.

Applicator #54-AP510

Micro Soft Sponge 1-3/4" Diameter

This micro polyurethane sponge applicator is designed for use on children, elderly, or sensitive areas of the body. For postural drainage this applicator may be attached to Applicator #54-AP509 for a full percussive effect.

For use with the G5® Microhead™ only.

Applicator #54-AP514

Fine Micro Sponge with Plastic Back

This is a plastic back with Velcro attached upon which to affix either a soft sponge or coarse sponge. These sponges are easily replaced. A reusable micro cover helps to extend the usefulness of the sponge.

Applicators #54-AP514FS and #54-AP514FC each contain ten (10) fine or coarse sponges.

For use with the G5® Microhead™ only.

Applicator #54-AP522

Micro Rounded Firm Rubber

This rounded one-ball applicator provides a firmer touch than Applicator #54-AP510 for postural drainage and is ideal for relaxing muscles in difficult to reach areas.

For use with the G5® Microhead™ only.



Applicator #54-AP525

Micro Scalp and Skin Surface

This multi-fingered applicator stimulates the circulation of the skin and provides a gentle relaxation massage. This applicator is not intended for use in postural drainage.

For use with the G5® Microhead™ only.



Applicator #54-AP529

Micro Flat Firm Rubber

Like the macro applicator #54-AP229, this micro applicator is designed for postural drainage. This applicator may be affixed to Applicator #54-AP509 percussion adaptor for full percussive effect.

For use with the G5® Microhead™ only.



G5® Applicator Packages

Professional Therapy Package -

(#54-PRO-PKG). The Professional Therapy Package includes the following applicators:

- #54-AP212 - Large Sponge - 3-1/2"
- #54-AP215 - Multiple-Prong
- #54-AP216 - Four Ball
- #54-AP227 - Pointed-Tip
- #54-AP229 - Large Round Firm Rubber
- #54-AP230 - Curved Sponge
- #54-AP235D - Medium Disposable Applicator Covers (50)
- #54-AP236D - Large Disposable Applicator Covers (50)



Respiratory Therapy Package -

(#54-RT-PKG). The Respiratory Therapy Package includes the following applicators:

- #54-AP209 - Right Angle Percussion Adaptor
- #54-AP210 - Small Sponge - 2-5/8"
- #54-AP212 - Large Sponge - 3-1/2"
- #54-AP229 - Large Round Firm Rubber
- #54-AP230 - Curved Sponge
- #54-AP236 - Large Plastic Applicator Cover (1)
- #54-AP234D - Small Disposable Applicator Covers (50)
- #54-AP235D - Medium Disposable Applicator Covers (50)
- #54-AP236D - Large Disposable Applicator Covers (50)

Percussors/Accessories

Spa Treatment Package - (#54-SPA-PKG).
The Spa & Beauty Applicator Package includes the following applicators:



- #54-AP209 - Right Angle Percussion Adaptor
- #54-AP212 - Large Sponge - 3-1/2"
- #54-AP214 - Fine Sponge w/Back
- #54-AP214FF - 10 Fine Sponges
- #54-AP214FC - 10 Course Sponges
- #54-AP215 - Multiple Prong
- #54-AP216 - Four-Ball
- #54-AP225 - Skin & Scalp
- #54-AP227 - Pointed-Tip
- #54-AP230 - Curved Sponge
- #54-AP242 - Suction
- #54-AP235D - Medium Disposable Applicator Covers (50)
- #54-AP236D - Large Disposable Applicator Covers (50)

Professional Portable Package - (#54-PRO-PORT-PKG). The Portable Applicator Package comes standard with the G5 Pro Power and G5 GBM hand-held devices, and includes the following applicators:

- #54-AP216 - Four Ball
- #54-AP223 - Two Ball
- #54-AP227 - Pointed-Tip
- #54-AP229 - Large Round Firm Rubber
- #54-AP230 - Curved Sponge
- #54-AP235DHF - Medium Disposable Applicator Covers (25)
- #54-AP236DHF - Large Disposable Applicator Covers (25)



Home Care Therapy Package - (#54-HC-PKG).
This package is for use only with the G5 Flimm-Fighter and G5 Vibracare.

- #54-AP234MIN - Small Disposable Applicator Covers (6)
- #54-AP250 - Flat Round Applicator
- #54-AP251 - Adult Soft Cupping Applicator
- #54-AP253 - Soft Foam Applicator

Pediatric/Geriatic Home Care Package - (#54-PGHC-PKG). This package is for use only with the G5 Flimm-Fighter and G5 Vibracare.

- #54-AP234MIN - Small Disposable Applicator Covers (6)
- #54-AP250 - Flat Round Applicator
- #54-AP252 - Pediatric Soft Cupping Applicator
- #54-AP253 - Soft Foam Applicator

Micro Therapy Package - (#54-MCT-PKG). The Micro Therapy Applicator Package is for use only with the G5 Gemini or G5 Vibraport.



- #54-AP509 - Micro Right Angle Adaptor
- #54-AP510 - Micro Soft Sponge Rubber
- #54-AP514 - Micro Fine Sponge w/Back
- #54-AP514FS - 10 Micro Fine Sponges
- #54-AP514CS - 10 Micro Coarse Sponges
- #54-AP522 - Micro Blunt-Tip
- #54-AP525 - Micro Skin & Scalp
- #54-AP529 - Micro Round Firm Rubber
- #54-AP237 - Micro Applicator Covers (6)

**Micro applicators are approximately 1/4 to 1/2 the size of the regular applicators.*

G5® Accessories

Self Application Kit



#54-SAPKG

Wall Mount Bracket



#54-ACWM

Caster Stand



#54-AP50

Accessory Tray



#54-AP52B (black)
#54-AP52W (white)

G5® Brand Travel Bag



#54-AP240

Applicator Covers



Disposable Applicator Covers



Part #	Description/Qty.
#54-AP234	Small App. Covers (Pkg. of 6 Plastic Covers for Applicators #54-AP210, #54-AP217 & #54-AP253)
#54-AP235	Medium App. Covers (Pkg. of 6 Plastic Covers for Applicators #54-AP212, #54-AP216 & #54-AP229)
#54-AP236	Large App. Covers (Pkg. of 6 Plastic Covers for Applicator #54-AP230)
Part #	Description/Qty.
#54-AP234D	Small Disp. Covers (50 Small Covers for Applicators #54-AP210, #54-AP217, #54-AP251, #54-AP252 & #54-AP253)
#54-AP234MIN	Small Disp. Covers (6 Small Disp. Covers for Applicator #54-AP253)
#54-AP235D	Medium Disp. Covers (50 Medium Covers for Applicators #54-AP212, #54-AP216 & #54-AP229)
#54-AP236D	Large Disp. Covers (50 Large Covers for Applicator #54-AP230)
#54-AP237	Extra Small App. Covers (Pkg. of 6 Plastic Covers for Micro Applicators)