

Homecraft Steel Folding Commode

0815-61-919



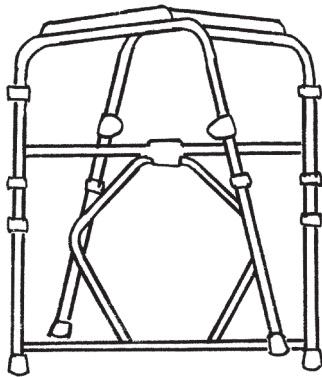


Illustration Keys

- A. Commode Frame
- B. Cross Brace, front
- C. Cross Brace, back
- D. Push Pins
- E. Side Frame Socket
- F. Back Connector of Cross Brace
- G. Commode Bucket
- H. Commode Seat & Lid

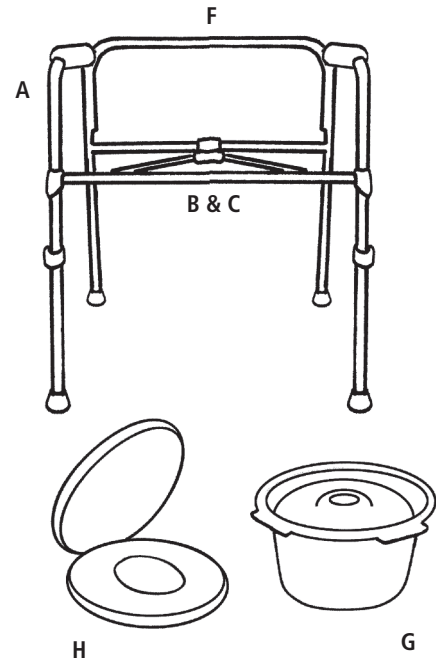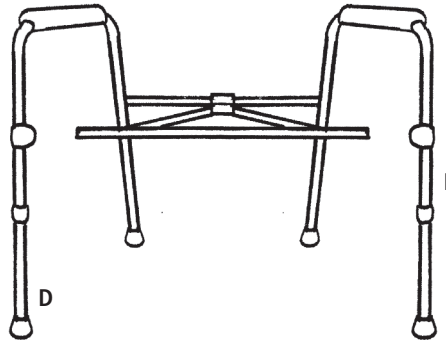

Before Assembly

Check that all components are included in packaging and that the parts have not been damaged during handling and shipping. Contact us if you find damage.

Read the following instructions and make sure you understand them. For clarification of any step, consult a caregiver, technical professional, family or friend.

Assembly Instructions

Open up the frame and set legs on the floor. Alter the position of the sides so that they are 90-degrees to the back. Depress the brass push pins and slide the front cross brace into the socket of the sides of frame. Place the push pins over the holes on the side socket and release. Make sure the pins are pushed all the way through the holes to ensure the front of cross brace and sides are attached correctly. Depress the push pins and slide the back brace of the frame through the holder on the back of cross brace. Again make sure the pins are "snapped" into the holes. Set the commode bucket into the cross brace and attach the commode lid to back of frame by snapping it securely on the frame. To fold up the com-

mode depress the push pins to release the cross brace for the frame of the commode.

Finish assembly by setting the leg height. Decide how high you want the legs, make sure the height is the same on all of the legs and insert buttons on legs into the corresponding holes. This will lock the leg at your desired height, and you can check the positioning by trying to move legs up or down.

Care

Use a mild cleaning product—no abrasive, scouring pads or solvents including lacquer thinner, turpentine or acetone.

Precautions

Supervision or assistance should be provided for users who are weak or tend to be imbalanced. Do not exceed the maximum user weight capacity of 300 lbs.

Before usage, check the following: All legs are on the floor when open and set up. Snap buttons on legs protrude through the proper holes so that they are secured at an even height. The toilet seat is down before positioning. Periodically inspect the rubber tips on the legs and that all nuts, bolts and screws are tight. If tips are torn or missing, replace them immediately.

Ensure that all push buttons are locked in place before use.

Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



Patterson Medical

1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547

Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310

www.pattersonmedical.com

© July 2011 Patterson Medical Products, Inc.
081561919LIT



Authorized European Representative Patterson Medical, Ltd.

Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom
Phone: 08444 124 330
Fax: 08448 730 100
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784