
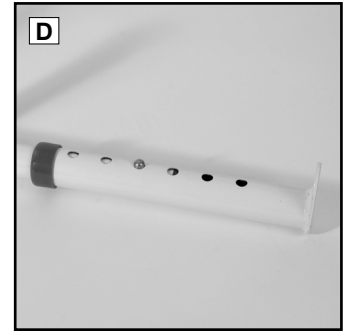
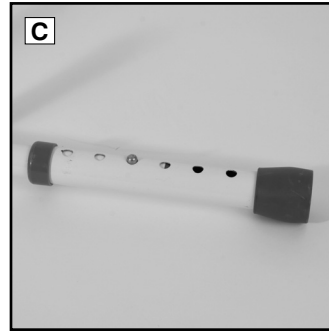
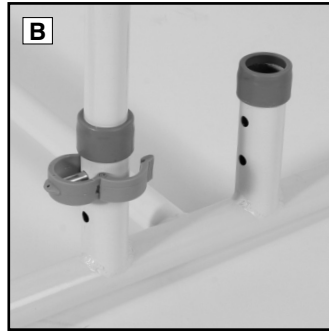
New Code	Old Code	
<b>09 107 9987</b>	AA2202	Steel Version
<b>09 107 9995</b>	AA2203	Aluminium Version
<b>09 108 0001</b>	AA2204	Steel Floor Fixing Feet
<b>09 108 0019</b>	AA2205	Aluminium Floor Fixing Feet

New Code	
<b>09 117 2378</b>	Bariatric Steel Toilet Surround Rail
<b>09 117 2386</b>	Bariatric Aluminium Toilet Surround Rail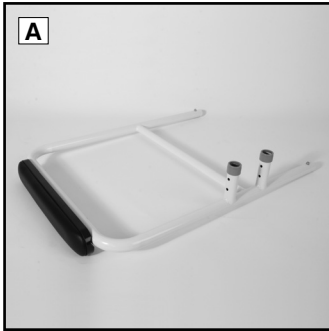

## Item Description

This sturdy toilet surround rail is both height and width adjustable making it suitable for a variety of situations. Available in either steel or aluminium, it has padded handgrips for additional comfort and support. Supplied flat packed, it is easy to assemble without the need for tools.

## Intended Uses

To compensate for a disability by helping support the user while standing up and sitting down.

## Instructions for Installation and Use

1. Lay one of the side frames **(A)** on the floor and position one of the cross bars **(B)** in the extended rod.
2. Align the hole on the cross bar with the hole on the extended rod and insert the pin clip.
3. Repeat step 1 and 2 with the second cross bar.
4. Lay the second frame on the floor and put the first frame assembly onto it. Line up the cross bars with the extending rods and insert the pin clips. Turn the frame so the feet are up and secure the cross bars.

## Adjusting the height

5. With the frame lying on one side, press in the brass studs and slide the leg sleeves **(C)** to the desired height. Ensure that all of the legs are adjusted to the same height and the brass studs are securing in the required holes.

## Using the floor-fixing feet

After the correct height and width has been determined for the safe use of the user, the toilet surround rail can be attached permanently to the floor. Once the frame has been attached to the floor it can no longer be adjusted in height or width. Attachment of the frame to the floor should be done by a qualified tradesman.

Remove the leg sleeves with the rubber ferrules **(C)** and attach the floor-fixing feet **(D)** in their place.

With the frame properly positioned, mark the position of the holes in the feet onto the floor. Remove the frame and check the area to be sure the floor is sound and there are no obstacles (such as plumbing and wiring). Drill holes and use suitable fasteners to screw down the feet.

## Care and Cleaning

The surround rails can be cleaned with mild detergent and water. Must be dried after cleaning and before use.

## Cautions

- Before use, ensure that all the legs are at the same height before putting weight on the frame
- Ensure that the top rail cross tube is below the top of the toilet seat
- The surround rails are designed to take a maximum user weight of 190kg/ 420lbs/ 30stone

Maximum user weight 

Please retain these instructions for future reference.

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New Code	Old Code	
09 107 9987	AA2202	Version acier
09 107 9995	AA2203	Version aluminium
09 108 0001	AA2204	Pied de fixation au sol en acier
09 108 0019	AA2205	Pied de fixation au sol en aluminium

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New Code	
09 117 2378	Bariatric Steel Toilet Surround Rail
09 117 2386	Bariatric Aluminium Toilet Surround Rail

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