

### **Lo-Boy Whirlpools**

Division of Acorn Engineering Company®









# L Series 90 Gallon Stationary Whirlpool

L-90-S



L-90-S

Whitehall L-90-S Stationary Whirlpool is recommended for treatment of extremities, hip and back area and allows spaces for exercising.

Whirlpool Tank is 60"L x 24"W x 18"D and is seamless welded construction, fabricated from heavy gauge, type 304 stainless steel, polished to a satin finish. Tank rim is reinforced with a concealed stainless steel rod. Tank bottom is seamless, coved (rounded) design for effective cleaning to minimize bacteria buildup. Tank is also provided with an extended stainless steel skirt flush to the floor line to conceal internal drain piping. A 2" Combination Drain and Overflow, and a Filler Spout are provided.

**Turbine Raising and Lowering Device** permits adjustment of desired height and direction of water agitation towards the area of the body to be treated. It is spring balanced and provided with a locking device.

**Turbine Assembly** is equipped with Underwriters Laboratory listed 1/2 HP jet pump motor protected with an ABS plastic shell, automatic thermal overload protector, and lifetime-sealed bearings. All parts contacting the water are stainless steel, chrome plated brass or bronze.

**Thermometer** is provided with a dual scale, adjustable calibration, and a stem inserts into a retainer to prevent rattle during whirlpool agitation.

### **GUIDE SPECIFICATIONS**

Provide Whitehall Stationary Whirlpool (specify model number and options) with a 90-gallon capacity, 60"L x 24"W x 18"D. Whirlpool shall be fabricated from heavy gauge, type 304 stainless steel. Construction shall be seamless welded and exposed surfaces shall be polished to satin finish. Tank shall have a reinforced rim and shall be fitted with a stainless steel skirt flush to the floor line. Bottom of tank shall be a seamless, rounded coved design to minimize bacteria build up. Provide a 2" combination drain and overflow, and a filler spout. Provide a turbine assembly with raising and lowering device which functions as an agitator and permits adjustment of desired height, direction of water agitation, and can be locked into place. Turbine assembly shall be UL listed 1/2 HP jet pump motor with automatic thermal overload protector and lifetime-sealed bearings. Provide a thermometer with a stem retainer to prevent rattle when whirlpool is in operation.

Please visit www.whitehallmfg.com for most current specifications.






Whitehall Mfg. • P.O. Box 3527 • City of Industry, CA 91744 • (800) 782-7706 • (626) 968-6681 • Fax (626) 855-4862

Updated: 10/03/18



## **Lo-Boy Whirlpools**

#### **MODEL NUMBER AND OPTION SELECTIONS Suffix Options** (See Whirlpool Options Section for Details) Adjustable Low Chair □I -ALC Series ☐ -ALS Adjustable Low Stool ✓ L Lo-Boy Series □ -ARS Arm Rest Support □ -AS3 Adjustable Suspension Seat **Capacity** ☐ -CTS3 Combination Table with Side Seats ✓ -90 Gallons ☐ -EG Enviro Glaze Color Powder Coating -**Type** Specify Color:\_ ✓ -S Stationary Whirlpool (Special Finishes Consult Factory) ☐ -GFI Ground Fault Interrupter **Turbine Electrical Requirements** ☐ -HR Head Rest, Adjustable ☐ 115VAC / 50Hz ☐ 115VAC / 60HZ Pump Strainer Cover -PCS ☐ 230VAC / 50Hz ☐ 230VAC / 60HZ -PHS2 Patient Handling System, Adjustable ☐ -RHG Rubber Hand Grips □ -RS Reclining Seat Country of Use ☐ -SDO Separate Drain and Overflow Assembly ☐ -TTS2 Tank Top Seat For Mixing Valve: Refer to supplemental MXT15 sheet for details.

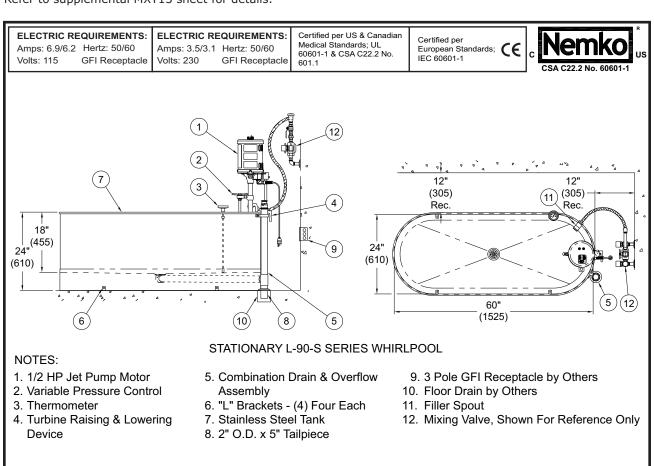

All dimensions are nominal and subject to manufacturer's change without notice. Whitehall assumes no responsibility fo use of void or superseded data © Whitehall Manufacturing, Member of Morris Group International.

Please visit **www.whitehallmfg.com** for most current specifications.

_	
$\overline{\wedge}$	WARNING: Cancer and Reproductive Harm - www.P65Warnings.ca.gov

	5 5		
SELECTION SUMMARY & APPROVAL FOR MANUFACTURING			
Model Number & Options	Quantity		
Company	Date		
Approval for Manufacturing/Signature			

Whitehall Mfg. • P.O. Box 3527 • City of Industry, CA 91744 • (800) 782-7706 • (626) 968-6681 • Fax (626) 855-4862