

# Zip-IQ TT Test Tube Centrifuge

The LW Scientific Zip-IQ TT centrifuge is an impressive 6-place test tube centrifuge for spinning blood, urine, and other fluids in 3ml to 15ml test tubes. This attractive unit is very small, conserving valuable counter space in the lab, but also sturdy and heavy with suction-cupped feet for stability. The digital controls allow speed and time to be adjusted with the touch of a finger and will show real-time speed and time remaining. This centrifuge is surprisingly quiet making the lab environment much more peaceful...you won't even realize this unit is running. The powerful, maintenance-free, brushless motor spins at 5,000rpm (2,500g) producing serum in under 10 minutes. Slow the speed to 2,000rpm (400g) to spin urines in 5 minutes. The auto-brake stops the rotor in 10 seconds, and the lid pops open automatically upon completion. The Zip-IQ TT is the smart choice for any clinic or laboratory.



## ADVANTAGES

- Small footprint conserves counter space
- Smooth, whisper-quiet operation
- Digital time and speed for accuracy
- 6-place angled rotor spins common 3ml-15ml test tubes

## FEATURES

- Angled 6-place test tube rotor with inserts (3ml to 15ml tubes)
- Max tube size 16.8mm diameter x 123mm tall
- Digital LCD displays speed, time, lid open/close
- Auto-lid-open at end of cycle
- Auto-brake stops rotor in 10 seconds
- Suction-cupped feet to prevent slipping
- Brushless maintenance-free motor
- Auto-switching AC adapter power supply
- Zero-RPM interlock – cannot open while spinning

## SPECIFICATIONS

- Digital variable speed: 300-5,000rpm in steps of 100 rpm
- Maximum RCF= 2,500g
- Digital timer: 15 seconds to 99 minutes
- 100-240V 50/60Hz auto-switching power supply
- 24-volt brushless DC motor
- 53 dBA +/- 3 dBA @ 18"-24"

## INCLUDES

- 6-place angled rotor for 7ml-15ml tubes (16mm x 100mm+)
- 6 black inserts for 5ml-6ml tubes (13mm x 100mm+)
- 6 white inserts for 3ml-4ml tubes (13mm x 75mm)
- 24 volt DC 4 amp AC power adapter (100-240 volt AC 50/60Hz input)

## Zip-IQ TT Test Tube Centrifuge

Model #	Description
ZIC-06AD-15T3	Digital, 6-place angled test tube rotor (3ml-15ml), 5,000 rpm, 0-lock, 90-240 volt

## WEIGHT AND DIMS

- Lid Open Height: 11.75" (298.45mm)
- Height: 6.2" (158mm)
- Length: 8.3" (212mm)
- Width: 10.6" (270mm)
- Weight: 9.9 lbs (4.5 kg)

**Boxed Dimensions:**  
Height: 8.75" (203.2mm)  
Length: 11" (279.4mm)  
Width: 15" (381mm)  
Weight: 12.25 lbs (5.56kg)



GrayMed  
PH: 61 3 86091732  
Email: [sales@graymed.com.au](mailto:sales@graymed.com.au)  
Web: [www.graymed.com.au](http://www.graymed.com.au)



P 770.270.1394  
F 770.270.2389  
865 Marathon Parkway  
Lawrenceville GA 30046