

29 Wells Avenue Yonkers, New York 10701

LATERAL BODY SUPPORT

Reorder #	Cover Fabric	Height
□706010	Synthetic Sheepskin	12¾
□706011	Polyester	12¾"
□706012	Spray-and-Wipe Nylon	12¾"
□706015	Synthetic Sheepskin	14"
□706016	Polyester	14"
□706017	Spray-and-Wipe Nylon	14"

ELEVATED-ARM LATERAL BODY SUPPORT

Reorder #	Description	Cover Fabric	Height
□706020	Right Arm Elevation	Synthetic Sheepskin	12¾"
□706021	Left Arm Elevation	Synthetic Sheepskin	12¾"

PURPOSE

Both versions of the Lateral Body Support - nonelevated and elevated-arm-are designed to improve sitting posture and reduce restraint dependecy. The elevatedarm model is recommended for residents with edema. All Lateral Body Supports may be used in the wheelchair or geri-chair.

TO USE

1A. Lateral Body Support (Nonelevated) Select the correct height Lateral Body Support for your resident. For small and average size residents, use the 12³/₄" - high support. For tall residents, use the 14"-high support. NOTE: if your resident is seated on a cushion, you may need to use the higher support.

1B. Elevated-Arm Lateral Support Because of the angle of elevation, this device is available only in a 12³/₄" height. This height is measured at the lowest point of the incline. Small residents may require a seat cushion when using the Elevated-Arm Lateral Support.

2. Position the Lateral Body Support on the inside of the chair armrest. Place the support on the side of the chair toward which your resident leans. The standard support



may be used on either the right or left side of the chair. The elevated-arm support must be ordered specifically for right- or left-leaning residents.

3. Secure the tie cords to the chair as shown in the illustration. The method for securing the support to the chair is the same for both versions. CAUTION

914-963-2040

These devices are not restraints! The determination as to the appropriateness of any version of the Lateral Body Support for a particular resident is the responsibility of the facility and its professional and medical staff.

- 4. Seat resident as far back in the chair as possible and place resident's arm in the trough. If additional arm stabilization is required, use the hook-and-loop arm strap.
- 5. Check resident periodically. Reposition as necessary.

LAUNDERING

Synthetic sheepskin and polyester covers are designed to be removed for laundering. Covers may be washed and dried at temperatures up to 180°F (80° C). Bleaching is not recommended. Make certain to remove hook-and-loop arm strap before laundering. Nylon covers may be cleaned with a mild spray-type cleaner and a damp cloth.

Reorder #	Cover Fabric	Size	Fabric
□784070	Nonelevated Lateral Body Support	12¾	Sheepskin
□784071	Elevated Right Arm Lateral Body Support	12¾"	Sheepskin
□784072	Elevated Left Arm Lateral Body Support	12¾"	Sheepskin
□784075	Nonelevated Lateral Body Support	14"	Sheepskin
□784077	Nonelevated Lateral Body Support	12¾"	Polyester
□784049	Nonelevated Lateral Body Support	14"	Polyester
□784081	Nonelevated Lateral Body Support	12¾"	Nylon
□784083	Nonelevated Lateral Body Support	14"	Nylon



Advanced design for nursing and therapy.™