

Homecraft Lightweight Shower Chairs



Please read these instructions carefully and retain for future reference

Intended Use

To compensate for a disability by providing a sitting surface for a person taking a shower

Assembly Instructions

- 1 Remove the contents from box.
- 2 Place the unit in upright position.
- 3 Locate the front seat support tube (the front and back seat support tubes are lower than the side support tubes).
- 4 Place the front seat caches (A) under the front seat support tube.
- 5 Lower the seat down and snap the back seat clips (B) into place.
- 6 Adjust the legs to the desired height, ensuring they are aligned equally (with the same height adjustment) and the buttons of the spring pins for each leg locate fully into position and protrude through the holes.

Care and Cleaning

- Regularly inspect all parts carefully for damage or wear, checking the security of any adjustable fittings.
- All parts can be wiped clean using a solution of mild detergent or household disinfectant.
- Dry thoroughly after each use.

	Old Code	New Code
Shower Chair	AA1605	09 107 5753
Shower/Commode Chair	AA1606	09 107 5761



Maximum user weight





Cautions

- Ensure the chair is securely in place before putting weight on it.
- Use of a slip-resistant bath mat in conjunction with the shower chair is strongly recommended.
- Cover the chair with a towel to reduce slipperiness.
- Ensure the users feet are placed firmly on the floor when using the chair.



Note: The ferrules on these chairs are made from natural rubber and contain latex

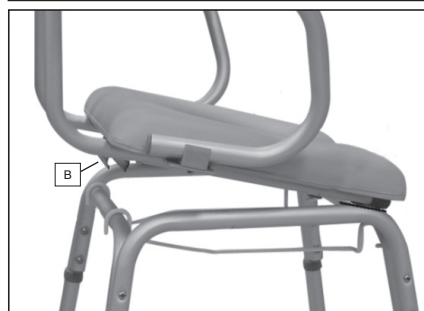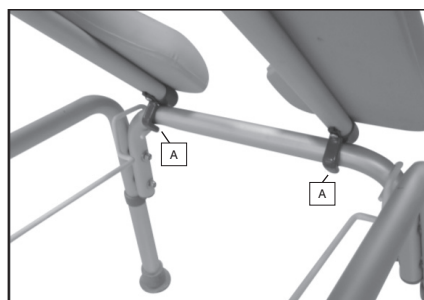

Manufactured for Patterson Medical Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK: Tel: 08444 124 330 Fax: 08448 730 100
www.pattersonmedical.co.uk

International: Tel: +44 1623 448 706
Fax: +44 1623 448 784

Patterson Medical Products, Inc.

28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938
Tel: 1-800-323-5547 Telefax: 1-800-547-4333

Patterson Medical Australia

PO Box 1006, North Ryde, NSW Australia 2113.
Tel: 1300 473 422 Fax: 1300 766 473

Patterson Medical France

Kinetec SAS, Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15 Fax: 03 24 52 90 34



PATTERSON
MEDICAL