

## 800.551.7096 | www.bionix.com

## IGLOO® WOUND IRRIGATION SHIELD - USER INSTRUCTIONS

- 1. Prep and drape the wound in a sterile fashion. Prepare a basin with sterile saline or other desired irrigant solution.
- 2. Fill a standard luer lock syringe or other delivery device with the irrigation solution.
- 3. Attach the Igloo Wound Irrigator to the tip of the syringe or other delivery device.
- 4. Place the Igloo Wound Irrigator over the wound, lightly touching the skin of the patient. For large, spacious (or chronic) wounds, place Igloo just above the wound surface.
- 5. Irrigate the wound in the usual fashion; syringe may need to be refilled one or more times as needed.
- 6. After use, discard the Igloo Wound Irrigator appropriately.

## IGLOO® WOUND IRRIGATION SYSTEM - USER INSTRUCTIONS

- 1. Prep and drape the wound in a sterile fashion. Prepare a basin and sterile saline bottle.
- 2. Straighten irrigation tubing.
- 3. Twist the Igloo Irrigation System adapter onto saline bottle until tight. Irrigation tubing should reach bottom of bottle.
- 4. Attach the Igloo Wound Irrigation Shield to the tip of the irrigation wand.
- 5. Place the Igloo Wound Irrigation Shield over the wound, lightly touching the skin of the patient. For large, spacious (or chronic) wounds, place Igloo just above the wound surface.
- 6. Irrigate the wound in the usual fashion.
- 7. After use, discard the Igloo Wound Irrigation System appropriately.





MADE IN



STERILE R

