


Nitro Tall

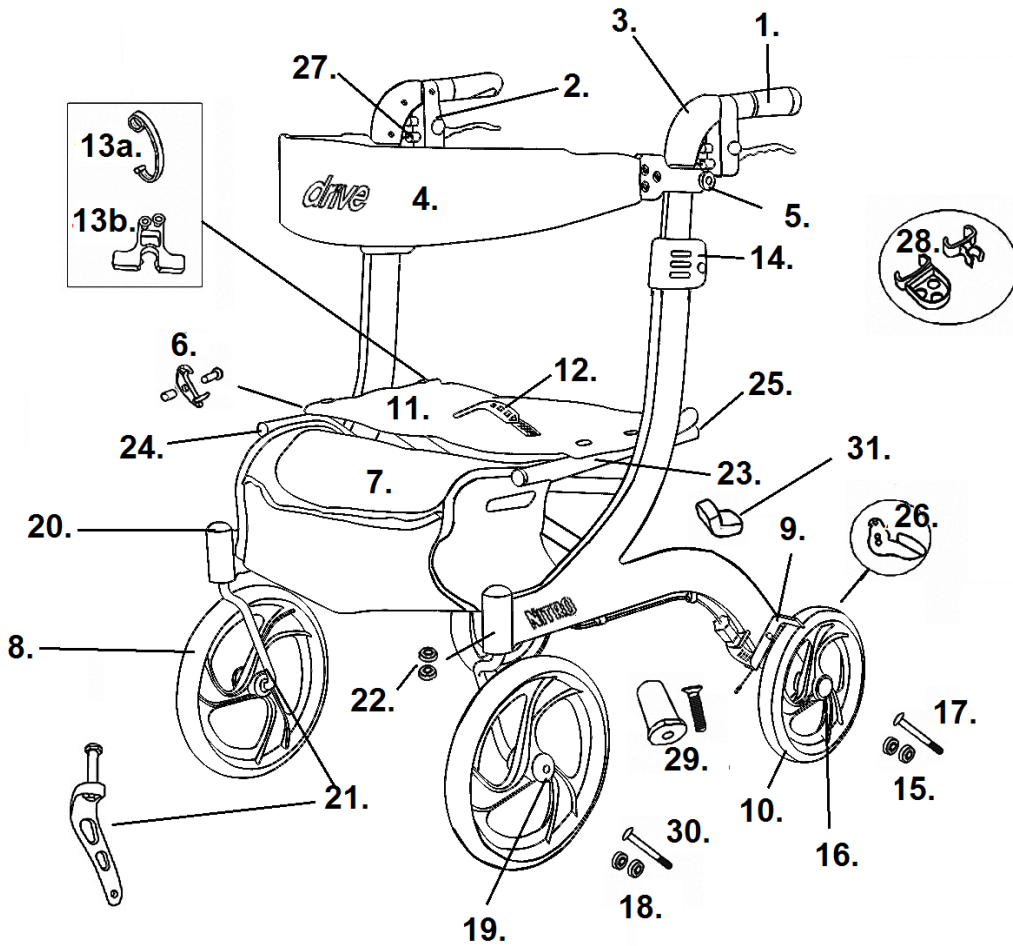

- | | |
|---|--|
| <p>1. Handgrip Left- 1026602
Handbrake Right- 1026603</p> <p>2. Hand Brake RTL10266-T Lt- 1026601-RN-T
Hand Brake RTL10266-T Rt- 1026601-RNR-T
Hand Brake RTL10266BK-T Lt- 1026601-G-T
Hand Brake RTL10266BK-T Rt- 1026601-GR-T
Hand Brake RTL10266WT-T Lt- 1026601-W-T
Hand Brake RTL10266WT-T Rt- 1026601-WR-T
Cable Only 55.5"- 1026632-T</p> <p>3. Handle Bar- 1026630N</p> <p>4. Backrest Assembly RTL10266-T- 1026604-B
Backrest Assembly RTL10266WT-T- 1026604-B
Backrest Assembly RTL10266BK-T- 1026604-G</p> <p>5. Backrest Knob & Bolt RTL10266-T- 1026605-R
Backrest Knob & Bolt RTL10266WT-T- 1026605-W
Backrest Knob & Bolt RTL10266BK-T- 1026605-G</p> <p>6. Safety Lock- 1026638</p> <p>7. Tote Bag RTL10266-T- 1026633-R
Tote Bag RTL10266BK-T- 1026633-BK
Tote Bag RTL10266WT-T- 1026633-WT</p> <p>8. Caster & Fork Assembly Style #1 Left- 1026625
Caster & Fork Assembly Style #1 Right- 1026624
Caster & Fork Assembly Style #2 Left- 1026625-AN [5]
Caster & Fork Assy. Style #2 Right- 1026624-AN [5]
Caster Style #1- 1026617
Caster Style #2- 1026617-AN [4]</p> <p>9. Wheel lock Assembly Metal Left- 1026615
Wheel lock Assembly Metal Right- 1026616
Wheel lock Assembly Plastic Left- 1026641 [2]
Wheel lock Assembly Plastic Right- 1026642 [2]</p> <p>10. Rear Wheel Left- 1026612-L
Rear Wheel Right- 1026612-R</p> <p>11. Seat/Hdwr. RTL10266-T (Black)- 1026608-TALL
Seat/Hdwr. RTL10266WT-T (Grey)- 1026608-TALL-G
Seat/Hdwr. RTL10266BK-T (Grey)- 1026608-TALL-G</p> | <p>12. Folding Strap Assy. 10266-T- 1026609-FSR-TALL
Folding Strap Assy. 10266BK-T- 1026609-FSG-TALL
Folding Strap Assy. 10266WT-T- 1026609-FSW-TALL</p> <p>13a. Folding Locking Latch 10266-T- 1026626-R
Folding Locking Latch 10266BK-T- 1026626-G
Folding Locking Latch 10266WT-T- 1026626-W</p> <p>13b. Folding Locking Latch- 1026644 [3]</p> <p>14. Back Height Adj. Latch RTL10266-T- 1026607-R
Back Height Adj. Latch RTL10266BK-T- 1026607-G
Back Height Adj. Latch RTL10266WT-T- 1026606-W</p> <p>15. Rear Axle Bearings (pr)- 1026613</p> <p>16. Rear Hub Cap- 1026619</p> <p>17. Rear Axle- 1026631</p> <p>18. Front Axle Bearings (pr)- 1026618
Front Axle Bearings (pr)- 1026618-AN [4]</p> <p>19. Front Hub Cap Old Style Fork- 1026614
Front Hub Cap New Style Fork- 1026614-AN [4]</p> <p>20. Dust Cap (pr)- 1026620</p> <p>21. Fork Assembly Old Style Left- 1026622
Fork Assembly Old Style Right- 1026621
Fork Assembly New Style Left- 1026622-AN [4]
Fork Assembly New Style Right- 1026621-AN [4]</p> <p>22. Fork Stem Bearings (pr)- 1026623</p> <p>23. Cross Frame- 1026627-TALL
Cross Frame Bolt- 1026646</p> <p>24. Cross Frame Front End Cap (ea)- 1026634</p> <p>25. Cross Frame Rear End Caps (pr)- 1026628</p> <p>26. Step Tube Left- 1026636 [1]
Step Tube Right- 1026637 [1]</p> <p>27. Brake Cable Guide- 1026639</p> <p>28. Cane Holder- 892</p> <p>29. Cross Brace Screw & Insert- 1026643</p> <p>30. Front Axle- 1026645 [4]</p> <p>31. Seat Rail Pad- STDS2A1807</p> |
|---|--|

[1] Serial # 2A140801025... & higher

[2] Serial # 2A1510302680 & higher

[3] Serial # 2A1606300234 & higher

[4] Serial # 2A1606301542 & higher

[5] RTL10266BK-T Serial # 2A1606301542 to 2A1909314949

RTL10266-T Serial # 2A1606301542 to 2A1909300101