


# OTHER GENUINE BIG JOHN PRODUCTS

## TOILET SEATS

The new era of stability, comfort and safety

Regular seats are too small and too flimsy. Perfect for Hospitals, Nursing Homes, Government Facilities, Stadiums, Airports, Theaters, Office Buildings and **For Those Aging in Place.**

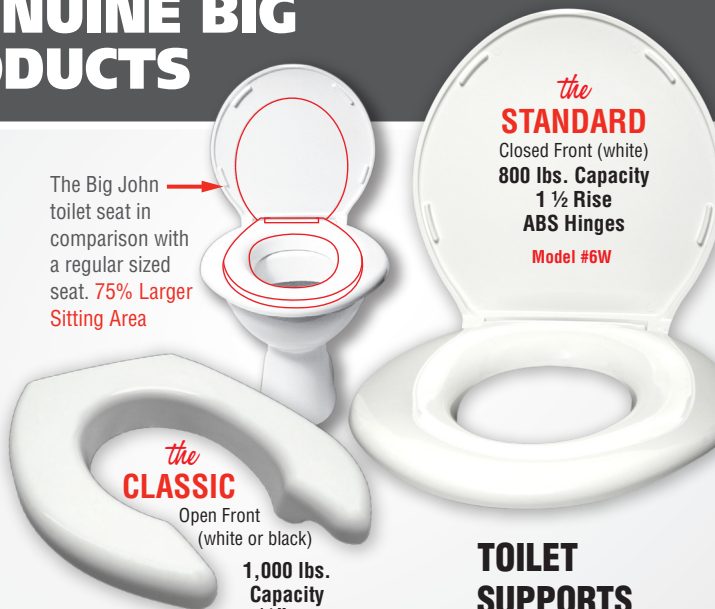

*the*  
**STANDARD**  
Closed Front (white)  
**800 lbs. Capacity**  
**1 1/2 Rise**  
**ABS Hinges**  
**Model #6W**

*the*  
**CLASSIC**  
Open Front  
(white or black)

**1,000 lbs. Capacity**  
**2 1/2" Rise**  
**Stainless Steel Hinges**  
**Model #7W**

## COMMODOE CHAIR

- 700 lb. capacity
- Bedside or over toilets
- Easy to assemble
- Adjustable height from 19"-23"

## BATH CHAIR

- 700 lb. capacity
- Adjustable height
- Extra wide seat and back
- Weighs only 3.13 lbs
- Non-slip feet
- Folds for easy storage
- Durable construction

## BARIATRIC STEP STOOL



The Short Swivel Support is a great alternative when the Regular Support doesn't fit.



Part # 612011919  
Part # 612011919-S

Height Adjustment From 7.5" to 14"  
Height Adjustment From 5" to 7.5"

# The Big John ORIGINAL



## Installation Instructions



ADA Compliant Registration Number 4.23 ADAAG

Meets or exceeds  
ASME / ANSI Z124.5  
CSA / IAPMO / UPC



**MADE IN THE U.S.A.**

**MADE IN THE U.S.A.**


# 9 Easy Step-by-Step Instructions for Installing the Big John Original Toilet Seat

## Step 1



Screw flanges all the way up so that there are no threads showing

## Step 2

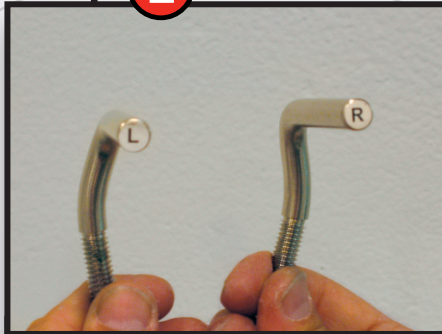

Hinges are marked right and left

## Step 3



Insert the Left Hinge through the seat and lid on the Left side\*

## Step 4




Insert Right Hinge\*

## Step 5




Put on the washers

## Step 6

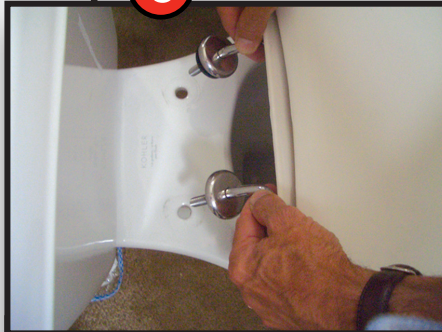

Mount on toilet and adjust flanges so bumpers are flat **See warning below\***

## Step 7

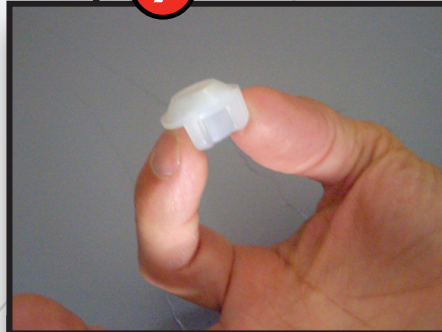

Be sure to put on the nuts with the beveled side up

## Step 8



Finger tighten the nuts...make sure that the seat is mounted properly

## Step 9



Secure the seat using pliers... DO NOT OVER TIGHTEN.

\*Left / Right Hinges - Refers to the left and right mounting holes on the toilet when facing it.

\*View installation video at ..... [WWW.BIGJOHNPRODUCTS.COM](http://WWW.BIGJOHNPRODUCTS.COM)



**WARNING: ADJUST FLANGES SO THE BUMPERS SIT FLAT ON THE TOILET BEFORE TIGHTENING NUTS. Cracking or breaking around the hinges is a result of improper installation and IS NOT covered by the warranty!**