



Homecraft Toilet Safety Frame

0815-61-901

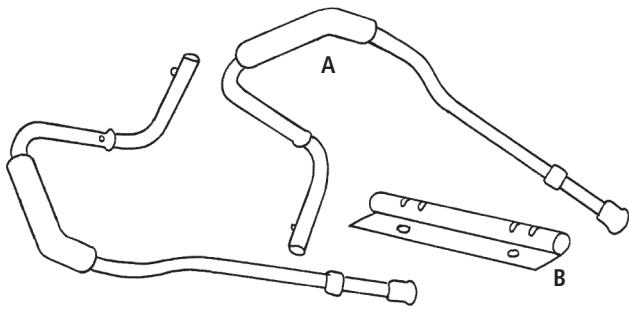

Illustrations

- A. Arm Support
- B. Mounting Bracket & Mounting Holes
- C. Inner Adjustment Holes

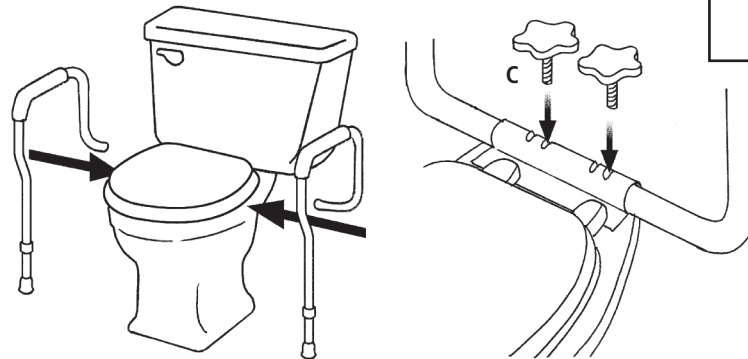

Before Assembly

Make sure all pieces—two arm supports and mounting bracket—are included in the package. Check that nothing has been damaged immediately after you open the package. Contact us if you find damage.

Read the following instructions, and if they are not clear, consult a professional caregiver or technician.

Assembly Instructions

Remove the toilet seat and lid, and place the hardware aside for later use. Place the holes on mounting bracket in the holes of toilet bowl. Return the seat and lid to position on top of the mounting bracket, and use existing hardware to attach and secure. Slide arm supports through both sides of the bracket and tighten thumb screws to secure.

Adjusting Height of Frame

The arm supports can be set at a height between 26" and 30". Press and hold the push button while sliding the leg up and down. When the desired hole is reached, release the button. Double check that the legs are set at same height and that buttons are fully engaged in the chosen hole.

Care

Use a mild cleaning product—no abrasive, scouring pads or solvents including lacquer thinner, turpentine and acetone.

Precautions

Do not exceed the user weight of 300 lbs.

Test for stability before use. Grip the supports securely and sit/stand. The frame serves as a stability aid; it does not support the total weight of an individual.

Assistance or supervision must be provided for those who are too weak or uncoordinated.

To ensure safety, before use: tighten thumb screws on bracket and check that all push buttons are locked in place.

Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



Patterson Medical

1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547

www.pattersonmedical.com

© June 2011 Patterson Medical Products, Inc.
081561901LIT

Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310



Authorized European Representative Patterson Medical, Ltd.

Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom
Phone: 08444 124 330
Fax: 08448 730 100
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784