#### Muñequera Ergodyne® ProFlex® (modelos 4000 y 4010):

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#### How to put on your Ergodyne® ProFlex® Wrist Support (Models 4000/4010):

1. Place around wrist (right or left, as stated) aligning the wrist to red line indicator.
2. Fasten hook-and-loop closure on back of wrist.
3. Place palm strap between thumb and index finger. Attach end of strap to hook-and-loop closure on back of wrist.
4. Wrap elastic strap across top of the wrist and fasten at desired tension.

#### How to put on the Ergodyne® ProFlex® Wrist Support (Model 4020):

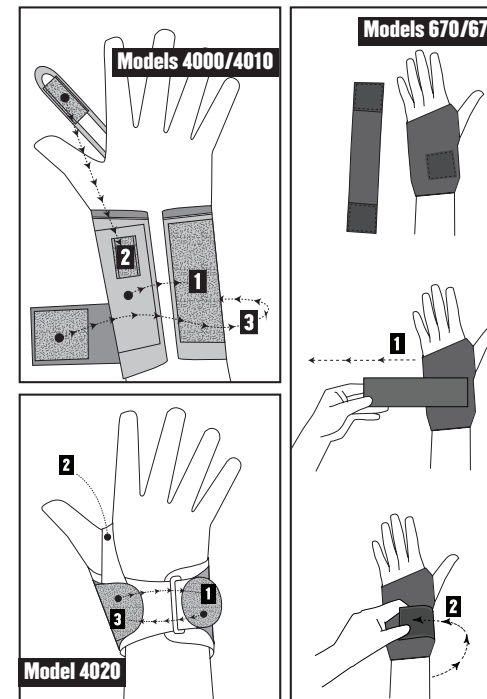
1. Loosen closure.
2. Slide hand (left or right, as stated) through bottom opening of wrist support. Position palm strap between thumb and index finger.
3. Pull closure tab up and toward the inside of the wrist to meet hook-and-loop closure. Fasten at desired tension.

#### How to put on the Ergodyne® ProFlex® Wrist Support (Model 670/675):

1. Slip hand and thumb through opening.
2. Place removable strap with loop to hook so the strap lays away from the body.
3. Pull strap underneath to wrist side and then over the back of the wrist.
4. Fasten hook-and-loop closure.
5. Repeat Step 2 for the additional strap on Model 675.

Care instructions: Machine wash cold; air dry.

- We cannot guarantee wearing personal protective equipment will prevent injury.

We stand behind everything we make and guarantee our products to be free of defects in materials and workmanship to the original purchaser for the lifetime of the product. Damage, abuse, and/or normal wear are not covered. Questions? Call Ergodyne at +1-651-642-9889 (8am-5pm CST).

1410 Energy Park Drive, Suite One, St. Paul, Minnesota, 55108 U.S.A.  
©2003-2005 Ergodyne. All Rights Reserved. Ergodyne, ProFlex, Be Tenacious, Tenacious Work Gear, Tenacious Work Badge, and the Ergoman logo are trademarks of Ergodyne.

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