SELECT

Cushioning Products
Operations Manual

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QUADTRO® SELECT®, CONTOUR SELECT™
Α-ΑΕÚ, %ÉÖ ÁÚÉÚÚÖÖ^a · ÍËÍ · ÚËÖÖ

QUADTRO® SELECT® and CONTOUR SELECT™ Cushions are manufactured in the U.S.A. by ROHO, Inc., 100 N. Florida Ave., Belleville, IL 62221-5429.

© The ROHO Group, Inc. 2003

The following are trademarks and registered trademarks of ROHO, Inc.:

ROHO®, AIR IN PLACE®, DRY FLOATATION®, ISOFLO™ Memory Control™, QUADTRO® SELECT®, CONTOUR SELECT™, Yellow Rope®, and shape fitting technology®.

SUPPLIER: This manual must be given to the user of this wheelchair cushion.

USER: Before using this wheelchair cushion, read this entire manual and save for future reference.

Table of Contents


ENGLISH	4
ESPAÑOL	11
FRANÇAIS	18
DEUTSCH	25
ITALIANO	32
NEDERLANDS	39
DANSK	46
SVENSKA	53
NORSK	60
SUOMI	67
PORTUGUÊS	74
EĪ Ī ĒVĒĪ ħ	81
Warranty.....	78
Disclaimer	88
EU MDD Representative Information	88

About this Manual

The QUADTRO SELECT and CONTOUR SELECT Cushions User Manual includes information regarding the proper use and care of your new cushion. Please read these instructions in full before using this product. Proper adjustment of QUADTRO SELECT or CONTOUR SELECT Cushions is critical for proper positioning and assisting in healing and protecting your skin from breakdown.


About the QUADTRO SELECT Cushion

The QUADTRO SELECT Cushion sets the standard in the industry for overall performance in wheelchair seating. Based on thirty years of research and client experience, our patented ROHO DRY FLOATATION® technology provides unparalleled performance in facilitating blood flow by fitting, matching and tracking the shape of a person. The revolutionary ISOFLO™ Memory Control™ Unit offers shape fitting capabilities while the user is seated, allowing quick and easy, on-demand adjustment to maximize function. Stability is enhanced by locking the air into place in each of the four quadrants through the simple use of the ISOFLO Memory Control Unit, minimizing side to side or front to back motion. With the built-in stability and simplicity of the QUADTRO SELECT Cushion, you get the best combination of skin protection, stability, positioning and convenience available.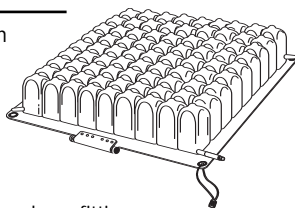

About the CONTOUR SELECT Cushion

The CONTOUR SELECT Cushion gives you all of the features of the QUADTRO SELECT and also, variable cell heights to further enhance seated stability. The ISOFLO Memory Control Unit is also featured on the CONTOUR SELECT Cushion.

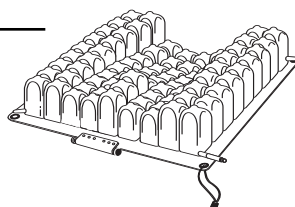The Concept Behind DRY FLOATATION

DRY FLOATATION supports your body like a fluid, allowing you to sit with minimal deformation to your tissue. Because DRY FLOATATION Cushions use air in which you “float”, they are lightweight and easy to adjust in order to accommodate your unique shape and posture. With the ISOFLO Memory Control Unit unlocked, the SELECT Cushions are dynamic, meaning that the air distribution in the cushion adjusts as you move around on it. Once you lock the ISOFLO Memory Control Unit, air distribution is minimized in order to provide stability while the user is seated in the cushion. Your ROHO Cushion should be properly adjusted, so you are immersed in the cushion as completely as possible, without bottoming out. Read this manual carefully to learn how to properly adjust your QUADTRO SELECT or CONTOUR SELECT Cushion.

CAUTION

To avoid the hazards identified below, it is your responsibility to check the cushion daily and to use the cushion appropriately.

DEFLATION: Loss of air in the cushion can result in bottoming out, eliminating (if not immediately fixed) the characteristics of the cushion that assist in healing and preventing skin tissue breakdown.

BOTTOMING OUT: Too much air being released from your cushion can cause your lowest bony prominence to touch the cushion base. This decreases or cancels any therapeutic value of the cushion and could result in skin tissue breakdown. The product should be adjusted so at least 1/2" (1.25 cm) of air is maintained between the QUADTRO SELECT or CONTOUR SELECT cushion and your lowest bony prominence.

OVER-INFLATION: Too much air in the cushion will reduce the therapeutic value of the cushion and could result in skin tissue breakdown or unstable seating position. Follow adjustment procedure on page 7.

DO NOT:


- expose cushion to pins or sharp objects.
- expose cushion to hot ashes or any open flame.
- use the air valve as a handle.
- use the ISOFLO Memory Control Unit as a handle.
- use oil based lotion or lanolin on your cushion, as they may compromise the integrity of the rubber.
- use petroleum-based products or materials containing plasticizers against the rubber, as they may cause discoloration and/or deterioration.
- use the cushion as a water flotation device.
- autoclave cushion because closed valves may increase internal pressure and cause the cushion to rupture.

Never use the ISOFLO® Memory Control™ as a handle. Pulling on the control mechanism may cause it to leak air. When necessary, use the handle strap on your cushion cover as a handle.

The U.S. Environmental Protection Agency, Health Canada, and other government agencies have issued warnings regarding the use of ozone generators. These devices may be injurious to your health and will degrade rubber, fabrics, plastics, and various other materials, including those used in the manufacture of your ROHO product. Prolonged exposure to ozone may affect the performance of your product and invalidate the product warranty.

Please contact The ROHO Group or your nearest rehabilitation technology supplier or distributor immediately if deflation or bottoming out that you cannot remedy is occurring for any reason.

U.S. & Canada: 1-800-851-3449
Outside the U.S. & Canada: 1-618-277-9150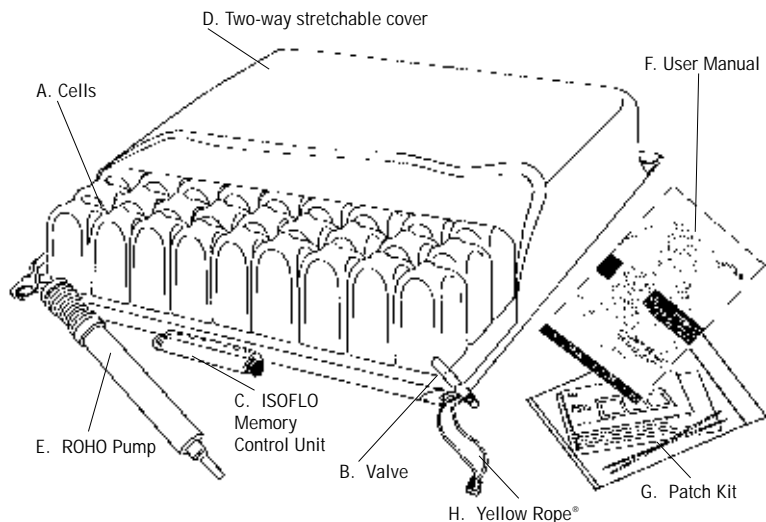

- A. Cells: ROHO's individual air pockets / bubbles.
- B. Valve: The valve is used to adjust the amount of air in the cushion. The valve should NOT be used for lifting or carrying.
- C. ISOFLO Memory Control Unit: The ISOFLO Memory Control Unit allows you to manage the air distribution in the cushion for positioning and stability by locking the air into four separate quadrants. The ISOFLO Memory Control Unit should NOT be used for lifting or carrying the cushion.
- D. Two-way stretchable cover: The cushion cover protects the cushion and allows for minimal interference with the therapeutic value of the cushion.
- E. ROHO Pump: The ROHO Pump is used to inflate your cushion.
- F. User Manual: Each ROHO Cushion includes a User Manual that provides instruction on adjustment, care, repairs, and warranty.
- G. Patch Kit: A Patch Kit is included to repair small holes.
- H. Handle: Use The Yellow Rope® for lifting or carrying.

Placing QUADTRO SELECT or CONTOUR SELECT Cushions On Your Wheelchair

The QUADTRO SELECT or CONTOUR SELECT Cushion should be placed on your wheelchair seat with the cells pointing upward. Place the cushion on your chair with the ISOFLO Memory Control Unit in front.


The cover should be placed on the cushion so the non-skid material is on the bottom. An opening is sewn into the cover to accommodate the ISOFLO Memory Control Unit.


Adjusting the QUADTRO SELECT or CONTOUR SELECT Cushion


IMPORTANT! Do Not Skip This Section!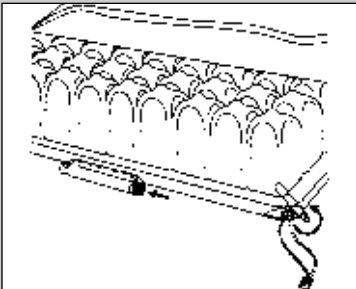


STEP 1: Open air valve by twisting it counter-clockwise. Check that the ISOFLO Memory Control Unit's green knob is pushed toward the center of the ISOFLO, in the UNLOCKED position.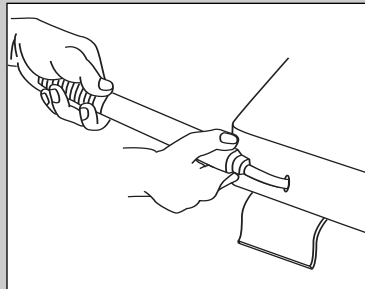

STEP 2: Attach the rubber hose on the ROHO Pump to the end of the valve. Inflate the cushion with the pump until the cushion begins to arch.

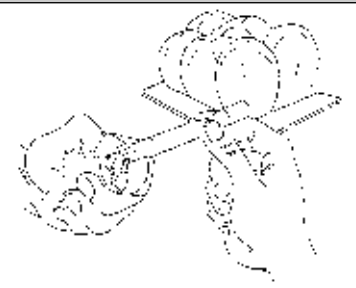
STEP 3: Remove the pump hose from the valve and close the valve by twisting it clockwise.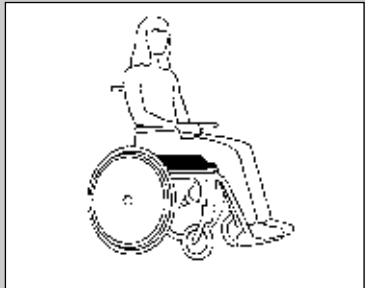


STEP 4: Sit on the cushion with armrests and footrests properly adjusted.


(continued on next page)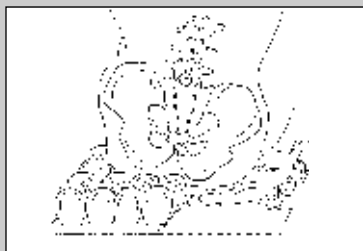


STEP 5: Slide your hand beneath your buttocks to locate your lowest bony prominence, usually the ischial tuberosities. Ask your clinician or caregiver to perform this step if you are unable to do so.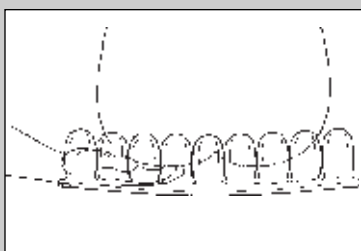

STEP 6: Leaving one hand in place beneath the lowest bony prominence, use the other hand to open the valve. As air is released from the cushion, you will begin to sink into it. When your fingertips touch the base, close the valve. Approximately $\frac{1}{2}$ " or 1.25 cm of air should now be between your lowest bony prominence and the base of the cushion.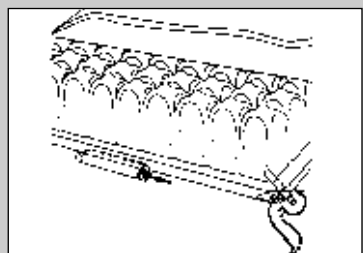

STEP 7*: Position yourself on the cushion in the posture you want to maintain. If necessary, your clinician or caregiver should assist you. Allow the air to transfer to accommodate your position. Once you are properly positioned, push the ISOFLO Control Unit's red knob toward the center, in the LOCKED position. This will isolate the flow of air to provide a more stable sitting position.

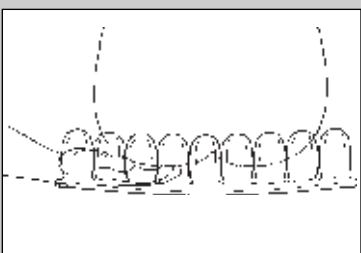STEP 8: Check your immersion by sliding your hand under your lowest bony prominence to ensure you have not bottomed out. Be sure to check your immersion in each quadrant of the cushion. If you wish to change your positioning for various activities, UNLOCK the ISOFLO Control Unit by pushing the green knob toward the center and repeat Steps 7 and 8.

* You have the option to skip Step 7 if you are not interested in using the extra stability and positioning features of the QUADTRO SELECT or CONTOUR SELECT Cushions.

IMPORTANT Adjustment Notes

- SELECT Cushioning Products are suitable for progressive positioning, allowing gradual changes to be made to your current posture. If you experience muscle fatigue or pain as a result of your new sitting position, readjust the cushion to a less stressful position, with the assistance of your clinician. Continue with gradual changes until the desired posture is achieved.
- You, your clinicians and / or caregivers should check the cushion daily to ensure that you have not bottomed out (i.e. lost air and sitting on the base of the cushion). If you bottom out, the therapeutic value of the cushion is lost. You should also check to be sure the cushion is not over-filled with air. The cushion will work best with approximately 1/2" or 1.25 cm of air between you and its base.
- You must adjust the air volume in a SELECT Cushioning Product with the ISOFLO Memory Control in the UNLOCKED (green knob pushed toward center) position. This is critical in allowing the cushion to fully conform to fit your unique shape. Once you have adjusted the air volume and closed the valve, you may proceed with the positioning functions described in Steps 7 and 8.
- SELECT Cushioning Products may be affected by changes in altitude. You may have to repeat Steps 1 - 8 above as you change altitudes.

Cleaning and Disinfecting Your SELECT Cushioning Product

Cleaning and disinfecting are two separate processes. Cleaning must precede disinfection.

To clean a SELECT Cushioning Product, deflate the cushion, close the valve, and place in a large sink. Gently scrub the air cells and the spaces between them with a soft plastic bristle brush, sponge, or washcloth. ROHO suggests using a liquid dishwashing or laundry detergent or other general-purpose cleaning product. For institutional cleaning, a general housekeeping disinfectant cleaner may be used.

To clean the cover, remove it from the cushion and machine wash in cold water using a liquid laundry detergent or other general purpose cleaning product. Air dry. Do not use heat or bleach.

To disinfect a SELECT Cushioning Product, use a germicidal product that meets infection control needs. ROHO suggests a disinfectant with tuberculocidal properties. Follow the manufacturer's dilution directions. For home disinfection, use solutions of 1/2 cup liquid household bleach per quart of water (125 ml bleach to 1 liter of water). Keep the clean cushion wet with bleach solution for 10 minutes. Rinse thoroughly, and dry.

To disinfect the cover, use a phenolic-based disinfectant, such as LYSOL® Disinfectant. Do not use heat or bleach. Do not autoclave.

Sterilization. ROHO does not recommend sterilization methods using temperatures greater than 200° F/93° C. High temperatures accelerate aging and may cause damage to the cushion. All valves must be opened before exposing the QUADTRO SELECT or CONTOUR SELECT Cushions to elevated temperatures.

DO NOT USE OIL-BASED LOTIONS OR LANOLIN ON YOUR CUSHION AS THEY MAY COMPROMISE THE INTEGRITY OF THE RUBBER.

REPAIRING LEAKS

You should protect your SELECT Cushioning Product from pins or other sharp objects and from exposure to hot ashes or any open flame. If your QUADTRO SELECT or CONTOUR SELECT Cushion is punctured, you can immediately repair it using the Patch Kit included with your cushion. Follow the directions with the kit to repair the leak. The patch should be monitored periodically to ensure its effectiveness. Check the cushion daily to ensure that you have not bottomed out. If your patch is not successful in addressing the problem, or if more extensive damage is present, contact your Rehabilitation Technology Supplier, distributor, or The ROHO Group for factory repair details.

The ROHO Group reserves the sole right to determine whether a cushion is repairable. A minimum repair charge plus shipping will be charged for all cushions not under warranty. Estimates for repair must be requested in writing and submitted with the cushion. If you fail to submit a written request for a repair estimate. The ROHO Group assumes authorization has been given and will proceed with the repair.

LOANER CUSHION PROGRAM (U.S.A. only):

The ROHO Group offers a loaner cushion while your cushion is being repaired. Loaner cushions require a deposit. Contact The ROHO Group for more information.

RETURN POLICY

All returns require prior authorization from The ROHO Group and are subject to a restocking charge. If you live outside the U.S.A., contact the distributor or dealer in your country or contact ROHO International to find a distributor near you.

CONTACT INFORMATION

If you have any questions concerning repairs, the loaner program, or the proper care and use of your ROHO product, contact:

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ROHO, Inc. 3-Year Limited Warranty

What is covered?

ROHO, Inc. covers any defects in materials or workmanship of the SELECT Cushioning Products (QUADTRO SELECT and CONTOUR SELECT only).

For how long?

Three years from the date the SELECT Cushioning Product was originally purchased as a new cushion, no matter who is the present owner.

What we will do.

Within a reasonable amount of time after you return the cushioning product to us, we will repair any defect in material and workmanship and ship it back to you, all free of charge within the warranty period.

What we will not do.

We will not repair a cushion free of charge if it has been misused or damaged by an accident. This also includes damages stemming from "acts of God", eg., flood, tornado, earthquake, or other catastrophic event, eg., fire.

What the customer must do.

The customer must send the SELECT Cushioning Product to us with the original sales ticket or other evidence of date of purchase, and he / she must pay all delivery costs. It must be sent to ROHO, Inc., 100 N. Florida Ave., Belleville, IL 62221-5429. We will not require the original sales ticket if you or the previous owner, if applicable, have completed and returned the warranty registration card, attached to this brochure, and we have it on file. Failure to submit the original sales ticket or failure to have the warranty registration card in our file voids this warranty.

Any other questions.

Call us toll free at 1-800-851-3449 or 1-618-277-9150.

Disclaimer

Any implied warranties, including those of MERCHANTABILITY and FITNESS FOR A PARTICULAR PURPOSE are also limited to three (3) years from the date of original purchase and any action for a breach of such warranties or any express warranties herein must be commenced within this time. ROHO, Inc. shall not be responsible for any incidental or consequential damages with respect to economic loss or injuries to property, whether as a result of a breach of express or implied warranties, or otherwise. ROHO, Inc. will not be responsible for the results of careless handling, unreasonable or improper use of this product.

Your State Law Rights and the Effect of this Warranty Upon Them

This warranty restricts the duration of all implied warranties to three (3) years and limits or excludes recovery for incidental or consequential damages. Some states do not allow limitations on how long implied warranties will last and some states do not allow the exclusion or limitation of consequential or incidental damages, so those provisions may not apply to you. While this warranty gives you specific legal rights, you may also have other rights which vary from state to state.

ROHO, Inc. and CROWN THERAPEUTICS, Inc. are wholly owned and operated subsidiaries of The ROHO Group, Inc. The ROHO Group, Inc. has a policy of continuous product improvement and reserves the right to amend specifications presented in this brochure.

REGISTRATION CARD 3-Year Limited Warranty

PLEASE COMPLETE, DETACH AND
MAIL FOR WARRANTY REGISTRATION

Name _____

Street No. _____

City, State _____

Zip & Country _____

Area Code & Telephone () _____

Email Address _____

User's Age? 18 and under 19-34 35-54 55-65 Over 65

PRODUCT INFORMATION

Purchased From _____

Date Purchased _____

Is this your first ROHO cushion? Yes No

Serial Number of cushion _____
(Printed on back of cushion along lip)

*Type of Product: QUADTRO® SELECT® CONTOUR SELECT™

* The 3-year limited warranty is only offered on the above SELECT Cushioning Products.

1. How did you learn about
DRY FLOATATION® cushions?

- therapist friend
 doctor advertisement
 dealer

2. Why the ROHO system?

- prescribed
 suggested by a ROHO product user
 unsatisfied with other brands

3. How was your ROHO® cushion
paid for?

- insurance Medicare
 self/family
 Other _____
(Please specify)


4. How many cushions do
you currently own?

- 1 3 more than 4
 2 4

signature

By signing this card, you agree that The ROHO
Group may contact you in the future with
additional product or promotional support.

LITG01010 ___ 10000 3/06

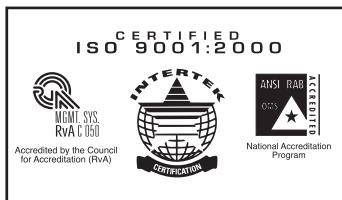NO POSTAGE
NECESSARY
IF MAILED
IN THE
UNITED STATES

BUSINESS REPLY MAIL

FIRST CLASS PERMIT NO. 846 BELLEVILLE, ILLINOIS, U.S.A.

POSTAGE WILL BE PAID BY ADDRESSEE

100 N. Florida Avenue
Belleville, Illinois 62221-5429 U.S.A.