



QuickStart Guide



Study the User Manual thoroughly before using the QuickStart Guide. The User Manual can be accessed online at http://www.quidel.com/immunoassays/sofia-tests-kits/sofia-2-analyzer

CUSTOMER SERVICE

If Sofia 2 does not perform as expected, contact Quidel Technical Support 800.874.1517 (in the U.S.), 858.552.1100 (outside the U.S.), technicalsupport@quidel.com or your local distributor.

UNPACK SOFIA 2

- Inspect the shipping container for obvious shipping damage prior to opening.
- Unpack the shipping container and inspect the unit for damage.



CONNECT TO POWER AND TURN ON

Place Sofia 2 on the counter top within reach of an electrical outlet. The unit is portable and can be moved to a suitable location for testing.



Plug the Power Adapter into the power port on the rear panel of Sofia 2 and appropriate electrical outlet.





Turn Sofia 2 on using the power switch located on the rear panel. A Power On Self-Test screen will appear on the front display when Sofia 2 is turned on.

Power Switch



ENTER ZIP CODE

When prompted, enter the ZIP Code where Sofia 2 is located and select \checkmark to confirm. Sofia 2 will then advance to the Run Test screen.



TEST VIRENA CONNECTION

From the Main Menu, select **Send Data**. Then select **Test connection** and press to test the Virena connection.



If the connection is successful, the message **Test connection success. Signal strength -##dBm.** will appear. Sofia 2 is now connected to Virena. If the connection is unsuccessful, contact Technical Support.



Continue to Run Calibration.

CUSTOMERS WITHOUT VIRENA ACTIVATED

For Sofia 2 customers without Virena activated, testing the connection will result in the message **Send To Virena Error. Unable to Acquire Cellular Connection.** or the Test Connection pop up window could continue for several minutes requiring the user to restart the instrument. To prevent this message from reappearing, select **Supervisor Menu** from the Main Menu. Then select **Network Settings**, then **Virena Settings**. Configure settings to the following:



Press \checkmark to confirm changes and return to the Run Test screen.

RUN CALIBRATION

- Enter the default Supervisor ID of 1234.
- Remove Calibration Cassette from the pouch.
- ▶ Press ▶ to advance to the next screen.





Following the prompts, insert the Calibration Cassette into Sofia 2 and gently close the drawer. Sofia 2 will perform the Calibration Check automatically.



The Sofia 2 display will indicate when the Calibration Check is completed.

NOTE: If Calibration Check cannot be completed successfully, notify the on-site Supervisor or contact Quidel Technical Support for assistance.

Once calibration is complete, remove and return Calibration Cassette to the storage pouch.

Important: Store the Calibration Cassette in the provided storage pouch between uses. Avoid touching the read window on the Calibration Cassette.

Run QC according to assay specific Package Insert.

MENU STRUCTURE





Quidel Corporation San Diego, CA 92121 USA quidel.com

1330303EN00 (07/18)