

Quick guide accustrip® URS Reader

Urine test procedure and daily maintenance

Test fresh urine samples within 2 hours of collection. Refrigerated urine samples must be brought to room temperature before testing (15-30 °C).

Read this guide before performing a test. Follow the instructions carefully.


Performing a urine test



Step 1: Turn ON the analyzer. Be sure that the strip holder is clean.



Step 2: Dip strip in the urine sample. Make sure to wet all test pads. Remove test strip from the urine after 1 second. Draw the long edge of the strip along the side of the container to remove excess urine. Use accustrip® URS 10 test strips only.

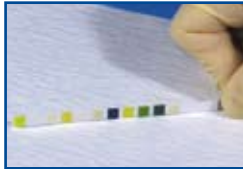

Step 3: Touch the long edge of the strip briefly (approx. 1 sec) to an absorbent paper towel to remove excess urine.



Step 4: Place the test strip on the strip holder. Make sure the test pads face upwards. The measurement starts automatically. The instrument will draw the strip in after 30 seconds. Do not push strip holder in manually. **IMPORTANT:** Do not remove the strip from the strip holder until the measurement is finished.



Step 5: When the measurement is complete, the result will be printed.



Step 6: Discard the used test strip. Wipe off any urine residues.

Daily Maintenance



Clean the strip holder with water and disinfectant once daily. Pull the strip holder to remove it from the instrument. Rinse the holder with water. Remove any crystalline deposits with a soft brush. Wipe down the strip holder with 70% alcohol or other suitable disinfectant. Dry with a soft cloth. Insert the holder into the instrument.



Changing printer paper



Open printer cover by pressing the small square button beside the printer paper slot. Remove the old paper roll. Insert the new roll of printer paper, unrolling from underneath. Unroll approx. 4 inches of paper. Close the cover.



Quality Control

Test negative and positive controls whenever a new bottle of strips is opened. Do not use water as negative control.

Technical Service

If you have any questions after reading the manual or if you need technical assistance, please contact

Jant Pharmacial Corporation
16255 Ventura Blvd.
Suite 505
Encino, California
91436 USA

Telephone: (800) 676-5565
E-mail: info@accutest.net



CAT. NO: UA850