drive™

www.drivemedical.com

Rev.1.10.04.14

Shower Chair With
Folding Back

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assembly instructions

Weight capacity 300 lbs (136 Kg)

1. Remove contents from carton.
2. Place seat upside down on a flat surface.
3. Insert leg into the leg receptacle by depressing push button and sliding into receptacle. Make sure the push button protrudes fully through the receptacle hole. Repeat for the remaining legs.
4. Turn bath bench over on its legs. Unfold the backrest frame so it is upright. Insert backrest over the top of the backrest frame.

ACCESSORIES (Sold Separately)

SUPPORT ARMS (RTL12606ARMS)

Insert arms into the receptacles on the sides of the seat, pushing down to secure in place.

SOAP DISH (RTL12606TRAY)

Insert soap dishes into the smaller receptacles located on the front corners of the seat by sliding them down into the receptacles.

adjusting leg height

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. To be sure legs are locked into place by pushing down on or by pulling legs.

additional precautions

1. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
2. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
3. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
4. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
5. DO NOT stand on chair.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

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lifetime limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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The logo for Drive Medical, featuring the word "drive" in a bold, blue, lowercase sans-serif font. A small yellow and orange square is positioned above the letter 'i'. A trademark symbol (TM) is located at the top right of the word.

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A large, stylized version of the Drive logo, oriented vertically. The word "drive" is written in white, lowercase letters against a dark blue background. A small yellow and orange square is positioned above the letter 'i'. A trademark symbol (TM) is located at the top right of the word.

www.drivemedical.com

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© 2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.