## Rolyan® Universal Sling: A549-1

### INDICATIONS

- For use following neurological injury to support a flaccid, weak, mildly subluxed, or painful shoulder
- Can also be used to support a casted arm

### CONTRAINDICATIONS

- Not in the presence of acute shoulder fracture unless immobilized by a cast
- Not in the presence of acute shoulder dislocation and other stages of severe shoulder trauma unless prescribed and fitted by a healthcare professional

### PRECAUTIONS

- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

### CARE AND CLEANING

Fasten hook-and-loop closures. Machine wash in warm water. Air dry for longer wear. Be sure sling is thoroughly dry before reapplication.

### APPLICATION

Sling can be worn on right or left arm. Orient sling so elbow cuff (larger one) is toward affected arm, and wrist cuff is toward unaffected arm. If necessary, unthread straps from buckles and rethread them with cuffs on appropriate sides.

1. Be sure all hook-and-loop closures are unfastened.
2. With elbow cuff toward affected arm and

A

wrist cuff toward unaffected arm, slide unaffected arm between the two straps attached to wrist cuff (A).

3. Bring elbow cuff around back and place shoulder strap over affected shoulder.
4. Place affected arm into elbow cuff. Slide wrist cuff onto affected arm (B). The back panel should be centered on upper back.
5. Adjust length of front and back straps until desired arm position is achieved.
6. Fasten strap across chest.
7. Bring thumb strap between thumb and index finger and fasten to wrist cuff (C).
8. Fasten strap that attaches wrist and elbow cuff together (D).

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Patterson Medical, Ltd.