

APPLICATION INSTRUCTIONS

Before applying the brace, please read these instructions completely and carefully. Correct application is vital to the proper functioning of the brace.

INDICATIONS:

Immobilization of wrist and thumb
Tendinitis
Carpal Tunnel Syndrome
deQuervian's Syndrome

- A** Loosen straps and slip hand into support with thumb resting in abduction stay.
- B** The palmar stay should be resting on palmar aspect of the hand. Stay is malleable and may be contoured to fit individual patient needs.
- C** Wrap and secure thumb strap.
- D** Starting at the hand, feed straps through D-rings and secure, working up the arm.

USE & CARE

Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY.
Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

WARNING: THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. THIS DEVICE IS NOT INTENDED FOR UNSUPERVISED PUBLIC USE. IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.