

## INSTRUCTIONS



### FUNCTIONAL POSITION HAND SPLINT

#### Apply Splint

Adjust straps and position pads for a comfortable fit

Plastic can be modified with a heat gun- Remove from patient · remove liner and straps  
Using extreme caution, gently heat plastic until soft · reshape · allow splint to cool completely before reapplying liner and straps



#### Cleaning

Plastics- Use damp cloth

Liner- Hand wash · air dry completely

Remove splint from patient to make adjustments

Avoid storing in extreme heat or cold.

Single Patient Use. This product is to be used under the supervision of a medical professional  

**Authorized Representative:** Medica Surgical Innovations Limited, Moorgate Street, Blackburn BB2 4PB, Lancs UK

## INSTRUCTIONS



### FUNCTIONAL POSITION HAND SPLINT

#### Apply Splint

Adjust straps and position pads for a comfortable fit

Plastic can be modified with a heat gun- Remove from patient · remove liner and straps  
Using extreme caution, gently heat plastic until soft · reshape · allow splint to cool completely before reapplying liner and straps

#### Cleaning

Plastics- Use damp cloth

Liner- Hand wash · air dry completely

Remove splint from patient to make adjustments

**Authorized Representative:** Medica Surgical Innovations Limited, Moorgate Street, Blackburn BB2 4PB, Lancs UK