Rolyan® Neutral Position Hand Splint:

A313-1 through A313-6

Rolyan® Deluxe Spasticity Hand Splint:

A484-1 through A484-6

A

B

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to both of the splints.

INDICATIONS

- To provide neutral positioning for spasticity

CONTRAINDICATIONS

- Not for the acute phase of most wrist or hand fractures
- Not for use directly over open wounds and infected areas

PRECAUTIONS

- **These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be**

discontinued and a healthcare professional consulted.

- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

PROCEDURE

Modification

These splints are made from either 1/8" (3.2mm) Polyform® splinting material (Rolyan

Neutral Position Hand Splint) or 1/8" (3.2mm) Polyflex II® splinting material (Rolyan Deluxe Spasticity Hand Splint). To trim or modify the splint, spot-heat it with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C) for one minute or until pliable. **Heating the entire splint may cause it to lose its preformed shape.** For better results when modifying the material, shape it with stroking movements.

Strapping

Apply straps as desired.

Rolyan Neutral Position Hand Splint

Includes straps—suggested strapping is shown in **A**.

Rolyan Deluxe Spasticity Hand Splint

Includes straps—suggested strapping is shown in **B**. This splint has a predrilled slot in the ulnar gutter at the MCPs to provide secure contact of the distal strap.

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Patterson Medical, Ltd.