
# Rolyan® Wrist and Thumb Spica Splint with IP Immobilization:

A164-1 and A164-2; 5503-91 and 5503-92 and 5514-02-01; 5503-92-01; 5664-25 through 27

A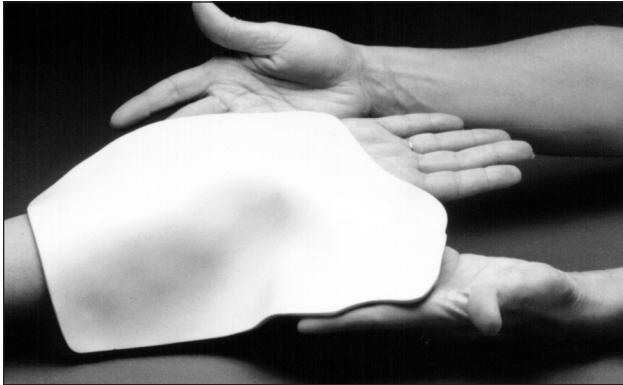

B

## INDICATIONS

- For immobilizing the wrist and joints of the thumb
- May be used for deQuervain's syndrome, gamekeeper's thumb, trapeziectomy and MCP fractures, and following MCP reconstruction

## CONTRAINDICATIONS

- Not in the presence of severe edema
- May not be recommended for the acute phase of wrist and thumb fractures

## PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**

- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

## CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

## PROCEDURE

The splint blank can be applied to either hand— simply invert the splint blank.

1. Heat splint blank in water heated to approximately 160°F (70°C) until pliable. (Refer to the instructions for Rolyan splinting materials for information about working with specific splinting materials.)
2. Remove from heat pan and place on a towel on a flat surface. Blot excess water.

3. If possible, place patient's hand in supination to achieve a gravity-assisted position.
4. Place splint blank in palm with the long portion of the splint blank over the thumb (A). Wrap the thumb piece circumferentially around the proximal phalanx of the thumb (B).
5. Mold the splint to the forearm.
6. The thumb piece can be left free to accommodate fluctuation in edema or bonded to the splint base as follows:
  - If the splinting material is coated to prevent accidental bonding, remove the coating by scraping the surface or brushing it with an acetone material.
  - The best bond can be achieved by dry heating the material (such as with a heat gun).
  - While still warm, place the pieces together carefully—they will bond instantly.
7. The splint can be trimmed and the edges flared if necessary.
8. Apply appropriate straps.  
(Design courtesy of Betty Osterhout, OTR/L, and Loyola University Medical Center.)

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Patterson Medical, Ltd.**