

Safe-t mate®

Model SM-016 Floor Mat with Alarm
Model FM-2436 Replacement Floor Mat

210 Innovations, LLC
210 Leonard Drive
Groton, CT 06340
800-210-2298

Floor mat and alarm box warranty—1 Year

Model SMABX Alarm Box Features and Operation


Install 9 Volt Battery (included).

Plug cord from floor mat into side of alarm box.

Turn on alarm box by fully depressing button on silicone jacket. LED will glow blue to indicate power is ON.

Test floor mat. Alarm should sound when pressure is applied to mat and automatically silence when pressure is removed.

Test Fail-safe circuit. With alarm box ON, remove cord and alarm should sound.

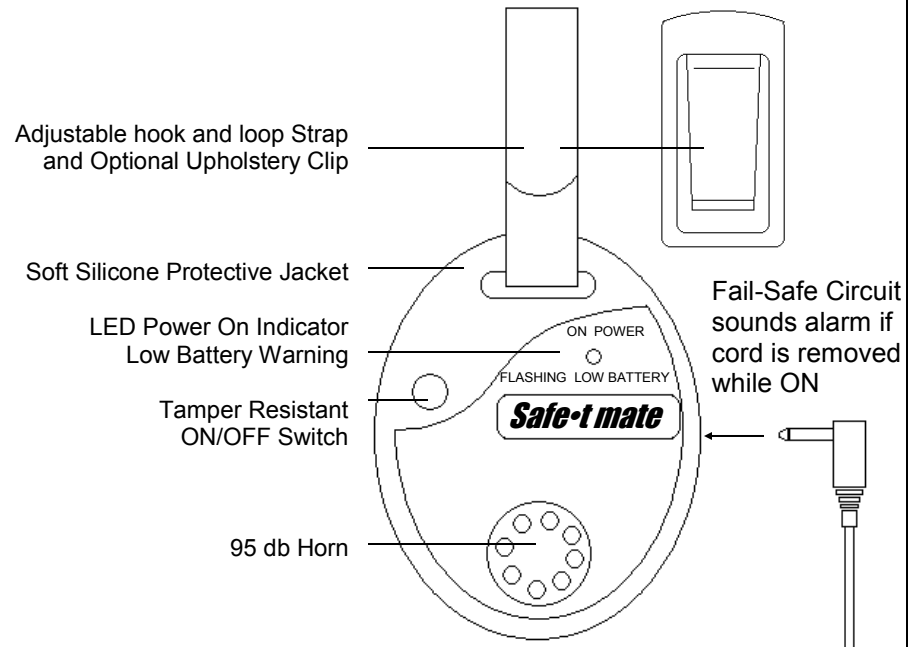

Floor Mat Positioning

Observe where patient's feet normally meet the floor when rising from bed. This is usually mid-way between top and bottom of mattress.

Position center of floor mat at the determined location.

TIP...

It's possible that the mat may occasionally shift out of position. It may be beneficial to mark the floor to indicate where the corners of the mat should be located.

Cleaning Mat

We recommend the use of a mild bleach-free household cleaner on a soft cloth.

