



Item# SS7677

- Place the button end of the pneumatic cylinder into the opening of the metal plate on the bottom of the seat. (Fig.1) Use minimal force to press the cylinder into the opening.
- 2. Turn the 5-point base upside down. Press one (1) caster into each of the ends of the 5-point base. (Fig.2)
- Turn the 5-point base over and place it on level ground so it is resting on all 5 casters. Place the bottom end of the pneumatic cylinder into the center opening of the 5-point base. (Fig. 3) Use minimal force to press the cylinder into the opening.
- 4. Carefully sit down on the seat* until the cylinder locks into place.
- 5. Using the lever under the seat, you can adjust its height as needed.

CAUTION:

- Maximum Weight Rating 300 lbs.
- If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts.
- \ast You need to weigh at least 90 lbs. to adjust the height of the seat.

Fig. 1







Fig. 3

