

Benefits

For the Fitter:

- easy to fit (headrest can be shaped by hand)
- attaches to standard three point hardware
- optional use of headband (included) for additional anterior support
- the pads are easily reshaped by hand when the occupant is repositioned

For the Wearer and/or Caregiver:

- a cool and comfortable headrest, the modular design of the cushions allows for air circulation
- low heat retention
- easy to put on and remove
- low profile; unobtrusive appearance
- easy to keep clean (sponge clean with soap and water, then air dry)
- total control for the head

Sizes and Colours



- Available in Adult and Small/Pediatric sizes
- Currently Available in Black only - check with Symmetric Designs for additional colours

Accessories

Side Pad

Part # SAVANT-SP

- for extra lateral support



This support can also be used as a switch mount

Whitmyer Adaptor

Part # SAVANT-WA

- This adaptor allows the Savant to be fitted to a 3/4" ball / Whitmyer mounting hardware



Netti Adaptor

Part # SAVANT-NA

- This adaptor allows the Savant to be fitted to Netti mounting hardware



Headrest Mounting Hardware

Part #HRHCO

- Incorporates a 1" (25mm) steel ball allowing universal adjustment
- Coated with a durable nylon finish that keeps its appearance longer



More Fine Products from Symmetric Designs

Knee & Elbow
Orthoses



TOT
Collar



Headmaster
Collar



www.symmetric-designs.com

The Savant Headrest™



A Wheelchair Headrest for Total Head Control

The Savant Headrest™ is a new and unique support for the head. It is cool, light-weight and can easily be shaped by hand. It provides support at three levels on the head to control lateral flexion and rotation.

A headband can be used for anterior support.



Symmetric Designs

125 Knott Place, Salt Spring Island, BC V8K 2M4, CANADA

email: sales@symmetric-designs.com

www.symmetric-designs.com

Tel: 800-537-1724 Fax: 250-537-1998

Fitting Instructions

Choosing the Correct Size

The Savant is available in two sizes:

- Adult
- Small / Pediatric



Features

The Savant Headrest™ is a unique new support for the head. It is cool, light-weight and can easily be shaped by hand. It provides support at three levels on the head to control lateral flexion and rotation.

A headband can be used for anterior support.

- Low profile headband gives anterior support
- Quick release clamps hold the Headband in place. Easy release for fine adjustment



- High Lateral Pads give effective lateral flexion control
- Mid-Lateral Pads are shaped for comfort behind the ears
- Sub-Occipital Pads form extra support below the mandible.
- The pads are filled with a soft gel for comfort.
- The Savant is covered with a soft and durable nylon/lycra knit fabric.

Sizing

Adult

On the client:	
Circumference at forehead	22" - 24" 56 - 61 cm
1/2 Circumference behind ears	8" - 9" 20 - 30 cm

On the Adult Headrest:	
Inside distance along top support	14" 36 cm
Inside distance along middle support	8" 20 cm
Overall height	6" 15 cm

Small/Pediatric

On the client:	
Circumference at forehead	18" - 22" 46 - 56 cm
1/2 Circumference behind ears	6" - 7" 15 - 18 cm

On the Small Headrest:	
Inside distance along top support	12" 30 cm
Inside distance along middle support	7" 18 cm
Overall height	5" 13 cm



The Savant is constructed with a unique steel core that allows selective shaping without tools.



The mounting disk accommodates a 1" (25mm) ball. The disk can be removed to attach custom hardware using M5 screws.

- optional Netti and Whitmyer hardware adapters are available



The Savant comes with a headband that can be used for anterior support.

The Savant can be worn with or without the headband.

Symmetric Designs accepts no liability resulting from the misapplication of this product.