

4850 Split Top Over bed Table

Operating Instructions

Tools required

Phillips Head Screw Driver

Operating Instructions

Note: Do not remove shipping screws until steps 1-4 are complete



- 1 Remove table from box with tops on the bottom tray.
- 2 Insert the casters, tie wrapped to base, into the holes in the bottom of the base.
- 3 Remove tape from drop lock paddle and table top. (Fig. 3)
- 4 Stand Table upright on the Casters.
- 5 Remove cardboard inserts from between tabletops. (Fig. 2)
- 6 Remove shipping screw. (Fig. 2)
- 7 To raise and lower the top, lift the Trip Handle upward and push down on the top to desired height.
- 8 To slide out the Lower Top, lift upward on the Top Lock Release tabs. Slide upper top back and forth several times as far as it will go. This is necessary after shipping to assure smooth operation.

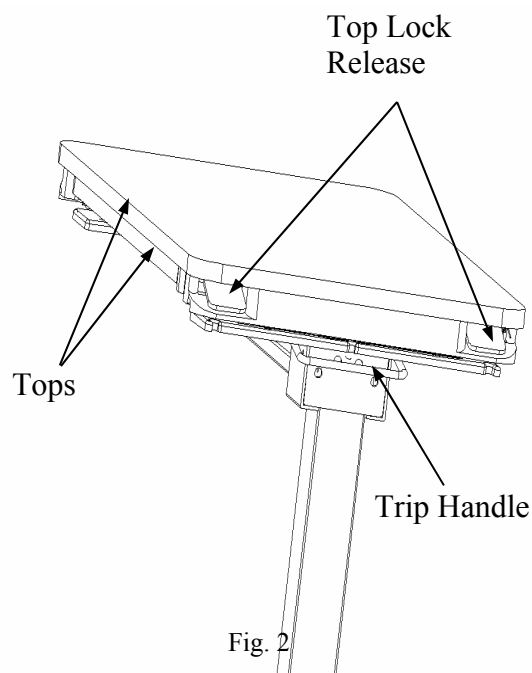


Fig. 1

STEP 3: REMOVE TAPE

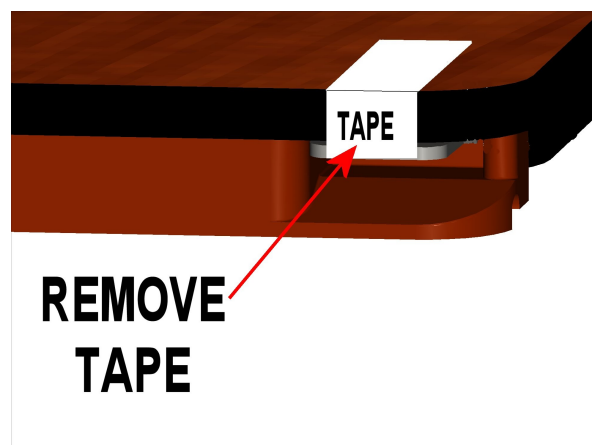


Fig. 3

4850 Split Top Over bed Table
Operating Instructions