

SURGIPRO™ II ~
SURGIPRO™
SUTURE

SURGIPRO™ II~SURGIPRO™ SUTURE

Description

SURGIPRO™ II~SURGIPRO™ sutures (clear or pigmented) are inert, nonabsorbable, sterile sutures composed of an isotactic, crystalline stereoisomer of polypropylene and contain polyethylene. The suture is pigmented blue to enhance visibility.

Characteristics

The advanced extrusion process of the polypropylene molecule gives the suture:

- Uniform diameter
- Maximum flexibility of the strand
- Excellent security with snug and flattened knots
- Minimal memory and reduced “pig-tailing”
- Consistent knotting strength

Indications

SURGIPRO™ II~SURGIPRO™ polypropylene sutures are indicated for use in general soft tissue approximation and/or ligation, including use in cardiovascular, ophthalmic and neurological surgery.

Contraindications

None known.

Actions

SURGIPRO™ II~SURGIPRO™ polypropylene sutures elicit a minimal acute inflammatory reaction in tissues, which is followed by gradual encapsulation of the suture by fibrous connective tissue.

Tensile Strength

SURGIPRO™ II~SURGIPRO™ polypropylene sutures are nonabsorbable and no significant change in strength retention is known to occur in vivo.

Sterilization

Ethylene oxide






Packaging

SURGIPRO™ II~SURGIPRO™ sutures are available in USP and EP sizes 2 (5 metric) through 8-0 (0.4 metric) and 10-0 (0.2 metric).





SURGIPRO™ II~SURGIPRO™ polypropylene sutures are available either undyed (clear) or dyed with Copper Phthalocyanine Blue. The sutures are supplied sterile affixed to needles using both permanent and removable needle attachment techniques; in one, two and three dozen box quantities.





REVERSE CUTTING ▼	Length	Color	Units Per Box	USP	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2
				M	0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5
3/8 Circle 16mm  C-12	18	Blue	12						SP-660-G							
	18	Blue	36						SP-660							
	30	Blue	36								SP-631					
3/8 Circle 19mm  C-13	18	Blue	12							SP-661-G	SP-683-G	SP-665-G				
	18	Blue	36							SP-661	SP-683	SP-665				
3/8 Circle 24mm  C-14	18	Blue	12									SP-684-G				
	18	Blue	36								SP-629	SP-684				
3/8 Circle 26mm  C-15	18	Blue	12										SP-868-G			
	18	Blue	36										SP-868			
	30	Blue	36										SP-685			
3/8 Circle 30mm  C-16	30	Blue	36								SP-670	SP-675				

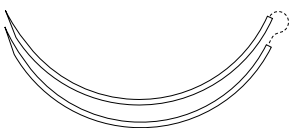

REVERSE CUTTING ▼	Length	Color	Units Per Box	USP	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2
				M	0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5
3/8 Circle 39mm  C-17	30	Blue	36									SP-649	SP-689	SP-690		
3/8 Circle 77mm  GS-18	3x24	Blue	12													3-CP-360
3/8 Circle 77mm  GS-18	30	Blue	24													CP-460
1/2 Circle 26mm  GS-10	30	Blue	36										CP-443			







PREMIUM REVERSE CUTTING ▼	Length	Color	Units Per Box	USP	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2		
				M	0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5		
3/8 Circle 11mm P-10	18	Blue	12					❖ SP-1696-G	❖ SP-5697-G	❖ SP-2698-G								
	18	Clear	36						❖ SP-1606									
	18	Blue	36					❖ SP-1696	❖ SP-1697									
3/8 Circle 13mm P-13	18	Blue	12						❖ SP-5695-G	❖ SP-5698-G	❖ SP-5699-G							
	18	Blue	36						❖ SP-5695	❖ SP-5698	❖ SP-5699							
3/8 Circle 16mm P-11	18	Blue	12						❖ SP-5680-G	❖ SP-5681-G	❖ SP-5692-G							
	18	Blue	36							❖ SP-5681								
3/8 Circle 19mm P-12	18	Blue	12							❖ SP-5686-G	❖ SP-5682-G	❖ SP-5687-G						
	18	Blue	36							❖ SP-5686	❖ SP-5682	❖ SP-5687						
3/8 Circle 24mm P-14	18	Blue	12									❖ SP-5663-G						
	18	Blue	36									❖ SP-5663						



PREMIUM X-CUTTING	Length	Color	Units Per Box	USP	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2
				M	0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5
3/8 Circle 11mm DX-11	18	Blue	12					✦ XP-8696								
3/8 Circle 13mm DX-13	18	Blue	12						✦ XP-8617	✦ XP-8618	✦ XP-8619					
3/8 Circle 16mm DX-16	18	Blue	12								✦ XP-8634					
3/8 Circle 19mm DX-19	18	Blue	12							✦ XP-8630		✦ XP-8632				



CONVENTIONAL CUTTING ▲	Length	Color	Units Per Box	USP	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2
				M	0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5
3/8 Circle 11mm  PC-13	18	Blue	36						SP-1617	SP-1618						
3/8 Circle 16mm  PC-11	18	Blue	36						SP-1636	SP-1635	SP-1634					
3/8 Circle 19mm  PC-12	18	Blue	36								SP-1644	SP-1643				
CUTTING ▼																
Straight 60mm  SC-2	30	Blue	36								SP-621	SP-622	SP-623			
	30	Blue	24									SP-622-BC	SP-623-BC			







PREMIUM SPATULA	Length	Color	Units Per Box	USP																
				10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2					
3/8 Circle 6.3mm Curvature 140° Radius 2.64mm Diameter 0.008" SE-140-8	12	Blue	12	◆ P-2771K																
1/2 Circle 5.5mm Curvature 160° Radius 2.11mm Diameter 0.006" SE-160-6	12	Blue	12	◆ P-2756K																
TAPER POINT 																				
3/8 Circle 11mm CVL	SURGALLOY™ OptiVis™ 2x24	Blue	12					VP-652-MX												
	SURGALLOY™ OptiVis™ 5x30	Blue	12					VP-650-MX												
3/8 Circle 13mm CVL-11	SURGALLOY™ OptiVis™ 5x30	Blue	12					VP-655-M												
3/8 Circle 6mm MV-135-5	18	Blue	36					◆ VP-900-X												
	2x18	Blue	12					◆ VP-900-MX												
	5x18	Blue	12					◆ VP-901-MX												
	24	Blue	36					◆ VP-902-X	◆ VP-904-X											
	2x24	Blue	12					◆ VP-902-MX	◆ VP-904-MX											
	5x24	Blue	12					◆ VP-903-MX												
	30	Blue	12					◆ VP-973-X												








TAPER POINT	Length	Color	Units Per Box	USP															
				M	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2			
					0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5			
3/8 Circle 8mm MV-175-8	18	Blue	36				❖ VP-744-X	❖ VP-628											
	2x18	Blue	12				❖ VP-744-MX												
	5x18	Blue	12					❖ VP-734-MX											
	24	Blue	36				❖ VP-745-X	❖ VP-735-X											
	2x24	Blue	12				❖ VP-743-MX	❖ VP-737-MX											
	5x24	Blue	12					❖ VP-735-MX											
	30	Blue	12					❖ VP-747-X											
	2x30	Blue	12				❖ VP-753-MX	❖ VP-747-MX											
3/8 Circle 8mm CV-351	2X24	Blue	12				❖ VP-738-MX												
3/8 Circle 9mm CV-1	18	Blue	36					❖ VP-701-X	❖ VP-746-X										
	2x18	Blue	12						❖ VP-746-MX										
	5x18	Blue	12					❖ VP-701-MX											
	24	Blue	36					❖ VP-702-X	❖ VP-703-X										
	SURGALLOY™ OptiVis 24	Blue	36					VP-72-X	VP-73-X										
	5x24	Blue	12					❖ VP-702-MX	❖ VP-703-MX										
	SURGALLOY™ OptiVis 5x24	Blue	12					❖ VP-72-MX	❖ VP-73-MX										
	30	Blue	36					❖ VP-713-X	❖ VP-709-X										
	5x30	Blue	12					❖ VP-713-MX	❖ VP-709-MX										
	SURGALLOY™ OptiVis 5x30	Blue	12					❖ VP-37-MX											
	36	Blue	36					❖ VP-366-X	❖ VP-365-X										







TAPER POINT 	Length	Color	Units Per Box	USP															
				10-0 0.2	9-0 0.3	8-0 0.4	7-0 0.5	6-0 0.7	5-0 1	4-0 1.5	3-0 2	2-0 3	0 3.5	1 4	2 5				
3/8 Circle 10mm  MV-175-9	24	Blue	36			VP-763X													
	5x24	Blue	12				VP-759-MX												
	30	Blue	36				VP-875-X												
	5x30	Blue	12				VP-875-MX												
3/8 Circle 11mm  CV	SURGALLOY™ OptiVis™ 24	Blue	36					VP-74-X											
	5x24	Blue	12				VP-765-MX	VP-764-MX											
	30	Blue	36					VP-776-X											
	5x30	Blue	12					VP-776-MX											
3/8 Circle 13mm  CV-11	5x18	Blue	12					VP-718-MX	VP-719-MX										
	18	Blue	36				VP-800-X	VP-718-X	VP-719-X										
	SURGALLOY™ OptiVis™ 18	Blue	36					VP-78-X											
	5x24	Blue	12					VP-726-MX	VP-725-MX										
	24	Blue	36				VP-727-X	VP-726-X	VP-725-X										
	SURGALLOY™ OptiVis™ 24	Blue	36						VP-75-X										
	5x30	Blue	12					VP-706-MX	VP-705-MX										
	30	Blue	36				VP-812	VP-706-X	VP-705-X	VP-813-X									
	SURGALLOY™ OptiVis™ 30	Blue	36					VP-76-X											
	SURGALLOY™ OptiVis™ 5x30	Blue	12					VP-76-MX											
	36	Blue	36					VP-363-X	VP-720-X										
	SURGALLOY™ OptiVis™ 36	Blue	36						VP-70-X										
	SURGALLOY™ OptiVis™ 5x36	Blue	12						VP-70-MX										
	5x36	Blue	12						VP-720-MX										




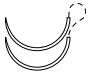


TAPER POINT 	Length	Color	Units Per Box	USP	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2
				M	0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5
3/8 Circle 13mm  CV-11	30	Blue	36						❖ VP-889-X	❖ VP-890-X						
3/8 Circle 17mm  CV-15	24	Blue	36								❖ VP-584-X					
	36	Blue	36							❖ VP-580-X	❖ VP-581-X	❖ VP-585-X				
	++ Pledget 36	Blue	36								❖ VP-582-X					
	48	Blue	36								❖ VP-583-X					
3/8 Circle 17mm  CV-15	30	Blue	36								❖ VP-881-X					
3/8 Circle 22mm  CV-13	24	Blue	36							❖ VP-579-X						
	36	Blue	36								❖ VP-851-X	❖ VP-532-X	VP-533			







TAPER POINT	Length	Color	Units Per Box	USP	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2
				M	0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5
3/8 Circle 6mm  MVF-135-5	2x24	Blue	12				❖ VP-209-MX									
3/8 Circle 8mm  MVF-175-8	24	Blue	36					❖ VPF-735-X								
	2x24	Blue	12					❖ VPF-737-MX								
3/8 Circle 9mm  CVF-1	24	Blue	36					❖ VPF-702-X	❖ VPF-703-X							
	5x24	Blue	12					❖ VP-102-MX	❖ VPF-703-MX							
	30	Blue	36					❖ VPF-713-X	❖ VPF-709-X							
3/8 Circle 11mm  CVF	30	Blue	36						VPF-776-X							





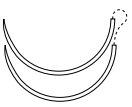

TAPER POINT	Length	Color	Units Per Box	USP																
				M	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2				
					0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5				
3/8 Circle 13mm  CVF-11	24	Blue	36						❖ VPF-726-X											
	5x24	Blue	12						❖ VP-126-MX											
	30	Blue	36						❖ VPF-706-X											
	5x30	Blue	12						❖ VPF-706-MX											
	36	Blue	36							❖ VPF-720-X										
	5x36	Blue	12							❖ VP-120-MX										
3/8 Circle 17mm  CVF-15	5x36	Blue	36										❖ VPF-612-X							
1/2 Circle 10mm  CV-20	24	Blue	36					❖ VP-772-X	❖ VP-754-X	❖ VP-755-X										
1/2 Circle 13mm  CV-22	18	Blue	36						❖ VP-714-X	❖ VP-715-X										
	24	Blue	36						❖ VP-711-X	❖ VP-205-X	❖ VP-204-X									
	30	Blue	36						❖ VP-733-X	❖ VP-710-X										
	5x30	Blue	12						❖ VP-733-MX	❖ VP-710-MX										



TAPER POINT	Length	Color	Units Per Box	USP															
				M	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2			
				0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5				
1/2 Circle 17mm  CV-23	18	Blue	36							❖ VP-756-X	❖ VP-757-X								
	24	Blue	36							❖ VP-566-X									
	36	Blue	36						❖ VP-555-X	❖ VP-556-X	❖ VP-557-X	❖ VP-558-X	❖ VP-559						
	++ Pledget 36	Blue	36							❖ VPP-557-X									
	5x36	Blue	12							❖ VP-556-MX	❖ VP-557-MX								
1/2 Circle 17mm  CV-23	30	Blue	36						❖ VP-870-X	❖ VP-871-X	❖ VP-872-X								
1/2 Circle 22mm  CV-25	30	Blue	36								❖ VP-760-X								
	36	Blue	36							❖ VP-761-X	❖ VP-762-X								
	5x36	Blue	12							❖ VP-761-MX									
1/2 Circle 22mm  CV-25	30	Blue	36							❖ VP-771-X									



TAPER POINT	Length	Color	Units Per Box	USP																
				M	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2				
				0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5					
1/2 Circle 12mm  CVF-21	24	Blue	36								❖ VPF-610-X									
1/2 Circle 13mm  CVF-22	30	Blue	36						❖ VPF-733-X	❖ VPF-710-X										
	5x30	Blue	12							❖ VPF-710-MX										
	36	Blue	36							❖ VPF-711-X										
1/2 Circle 17mm  CVF-23	5x36	Blue	12								❖ VPF-557-MX									
	36	Blue	36								❖ VPF-557-X									
1/2 Circle 27mm  GS-22	^Δ D-TACH [®] 5x18	Blue	12											CP-16-M	CP-17-M					
	30	Blue	36											CP-411	CP-412					



TAPER POINT	Length	Color	Units Per Box	USP												
				10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2	
				M 0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5	
1/2 Circle 37mm GS-21	48	Blue	24										CP-533			
1/2 Circle 37mm GS-21	5x18	Blue	12											CP-10-M		
	30	Blue	36										CP-422	CP-423	CP-424	CP-425
1/2 Circle 40mm GS-24	30	Blue	36										CP-433	CP-434	CP-435	
	40	Blue	36												CP-535	
1/2 Circle 48mm GS-25	5x18	Blue	12											CP-30-M		
	30	Blue	36											CP-454	CP-455	






TAPER POINT	Length	Color	Units Per Box	USP															
				M	10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2			
				0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5				
1/2 Circle 65mm GS-26	48 ²⁴	Blue	24													CP-824-L			
	96 ⁴⁸	Blue	24													CP-834-L			
1/2 Circle 65mm GS-26	60	Blue	24											CP-823	CP-824	CP-825			
1/2 Circle 22mm HGS-23	^Δ D-TACH [®] 5x18	Blue	12											CP-92-M					
1/2 Circle 27mm HGS-22	^Δ D-TACH [®] 5x18	Blue	12											CP-90-M					
	30	Blue	36											CP-414					
	36	Blue	36											CP-415					
1/2 Circle 26mm V-20	30	Blue	36								❖ VP-831-X	❖ VP-832-X	❖ VP-833	❖ VP-834					



TAPER POINT	Length	Color	Units Per Box	USP															
				10-0 0.2	9-0 0.3	8-0 0.4	7-0 0.5	6-0 0.7	5-0 1	4-0 1.5	3-0 2	2-0 3	0 3.5	1 4	2 5				
1/2 Circle 26mm V-20	30	Blue	36							❖ VP-510-X	❖ VP-511-X								
	++ Pledget 36	Blue	36								❖ VP-421-X	❖ VP-422-X							
	36	Blue	36							❖ VP-520-X	❖ VP-521-X	❖ VP-522-X	❖ VP-523						
	48	Blue	36								❖ VP-541-X	❖ VP-542-X	❖ VP-543						
	* Pledget 36	Blue	36								❖ VP-621-X	❖ VP-622-X							
1/2 Circle 26mm VF-20	48	Blue	36											VP-151					
1/2 Circle 26mm VF-20	36	Blue	36								VPF-521-X	VPF-522-X							
	48	Blue	36										VPF-543						
	54	Blue	36										VPF-354-X						
1/2 Circle 30mm V-30	30	Blue	36											VP-945					

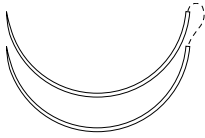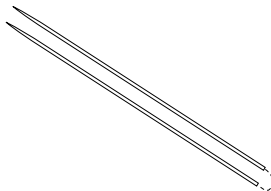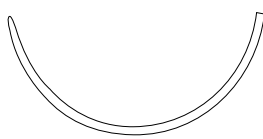

TAPER POINT	Length	Color	Units Per Box	USP															
				10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2				
				0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5				
1/2 Circle 37mm  V-26	36	Blue	36									VP-842-X	VP-843						
	48	Blue	36										VP-853						
Straight 51mm  ST-1	36	Blue	36										GP-282						
BLUNT TAPER POINT 																			
1/2 Circle 48mm  BGS-25	40	Blue	36													CP-836			



TAPERCUTTING	Length	Color	Units Per Box	USP															
				10-0 0.2	9-0 0.3	8-0 0.4	7-0 0.5	6-0 0.7	5-0 1	4-0 1.5	3-0 2	2-0 3	0 3.5	1 4	2 5				
3/8 Circle 9mm KV-1	18	Blue	36				❖ VP-721-X												
	5x18	Blue	12				❖ VP-721-MX												
	24	Blue	36				❖ VP-704-X	❖ VP-712-X											
	5x24	Blue	12				❖ VP-704-MX	❖ VP-712-MX											
3/8 Circle 13mm KV-11	18	Blue	36					❖ VP-723-X											
	4x18	Blue	12					❖ VP-723-MX											
	24	Blue	36					❖ VP-707-X											
	30	Blue	36					❖ VP-728-X	❖ VP-708-X										
	5x30	Blue	12					❖ VP-728-MX											
	36	Blue	36						❖ VP-751-X										
	5x36	Blue	36						❖ VP-751-MX										
3/8 Circle 17mm KV-15	30	Blue	36							❖ VP-914-X									
	36	Blue	36							❖ VP-924-X	❖ VP-925-X	❖ VP-926-X							
3/8 Circle 17mm KV-15	30	Blue	36							❖ VP-941-X									



TAPERCUTTING	Length	Color	Units Per Box	USP															
				10-0	9-0	8-0	7-0	6-0	5-0	4-0	3-0	2-0	0	1	2				
				0.2	0.3	0.4	0.5	0.7	1	1.5	2	3	3.5	4	5				
3/8 Circle 9mm KVF-1	24	Blue	36				❖ VPF-704-X												
	30	Blue	36					❖ VPF-807-X											
3/8 Circle 13mm KVF-11	30	Blue	36					❖ VPF-728-X											
1/2 Circle 17mm KV-5	++ Pledget 36	Blue	36								❖ VP-435-X								
	36	Blue	36							❖ VP-934-X	❖ VP-935-X	❖ VP-936-X							
1/2 Circle 26mm KV-7	36	Blue	36								❖ VP-975-X	❖ VP-976-X	❖ VP-977	❖ VP-978					
1/2 Circle 37mm KV-34	30	Blue	36											CP-844					