

TECHNICAL DATA SHEET



FEATURES & BENEFITS

- Thoughtful designs offer ease of movement, as well as secure closure and easy removal.
- Premium materials provide superior protection, reliability and durability.
- Lightweight, soft, supple, fabrics keep wearers cool and comfortable.
- SafeWear™ protective apparel complies with recommended OSHA best practices.

RECOMMENDED USE

- Procedure SMS Jacket designed to protect health professionals and patients from cross contamination. Single use and disposable gowns made of polypropylene. Material meets AAMI/PB70 Level 2 requirements for impact penetration and hydrostatic pressure. Latex free.

PRODUCT DETAILS

Product Name:	Medicom SafeWear™ Hipster Jacket™
Product Type:	Single-use, non-sterile
Color:	See Appendix 1
Sizes:	See Appendix 1
Quantity per Bag:	12
Country of Origin:	China

DONNING INSTRUCTIONS

- Perform hand hygiene before removing item from package.
- Select appropriate garment for intended use (length, coverage, fluid resistance, etc.)
- Fasten correctly: Front opening/closure snaps to ensure comfortable fit, adequate coverage and freedom of movement.
- Protective apparel should not be worn in restrooms, personal offices or break rooms.

DOFFING INSTRUCTIONS

- Remove apparel immediately after use. Do not re-use.
- Perform proper hand hygiene after removing apparel to prevent cross-contamination.
- Turn garment inside out when removing to keep infectious material inside.
- Designate a safe receptacle in which to dispose of protective apparel after use.

CERTIFICATION & STANDARDS

ISO 13485 certified manufacturer.
Class 1 Medical Device.
Health Canada registered medical device manufacturer.


STORAGE CONDITIONS

These gowns should be disposed of after each use.
Store under ambient temperature and dry condition.



GOWN SPECIFICATIONS

CHARACTERISTIC		SPECIFICATION						
Material Composition		Spunbond Melt-down Spunbond (SMS) polypropylene						
Description		Jacket made of non-woven material with front closure and snap buttons. Features knitted collar and cuffs. Refer to Appendix 1 for specific attributes.						
Part of Coat		Dimension (cm)						
		X-Small	Small	Medium	Large	X-Large	2X-Large	3X-Large
A	Coat Length	66	71	75	82	86	89	91
B	Underarm Width	50	55	60	65	70	78	86
C	Sleeve Length*	50	54	54	57	58	59	62
D	Arm Height	27	28	29	30	30	33	34
E	Shoulder Width	16	17	17	21	22	26	27
F	Knitted Cuff Length	6	6	6	6	6	6	6
L	Sleeve Opening Width (Stretched)	15	15	15	15	15	15	17
M	Knitted Cuff Width	6	6	6	6	6	6	6
G	Button Spacing	11.5	11.5	11.5	11.5	11.5	11.5	11.5
I	Lower Front Pocket - Width	18	18	18	18	18	18	18
H	Lower Front Pocket - Height	18	18	18	18	18	18	18
K	Upper Pocket - Width	14	14	14	14	14	14	14
J	Upper Pocket - Height	14	14	14	14	14	14	14
N	Upper Pocket Distance from Opening Edge	5	8	9	10	11	13	15
P	Lower Pocket Distance from Opening Edge	5	5	5	7	9	10	12
S	Distance between Upper and Lower Pocket	10	12	14	16	20	23	23
O	Lower Pocket Distance from Bottom Edge	4	5	7	13	13	13	13
	Number of Buttons	5	5	5	5	5	5	5

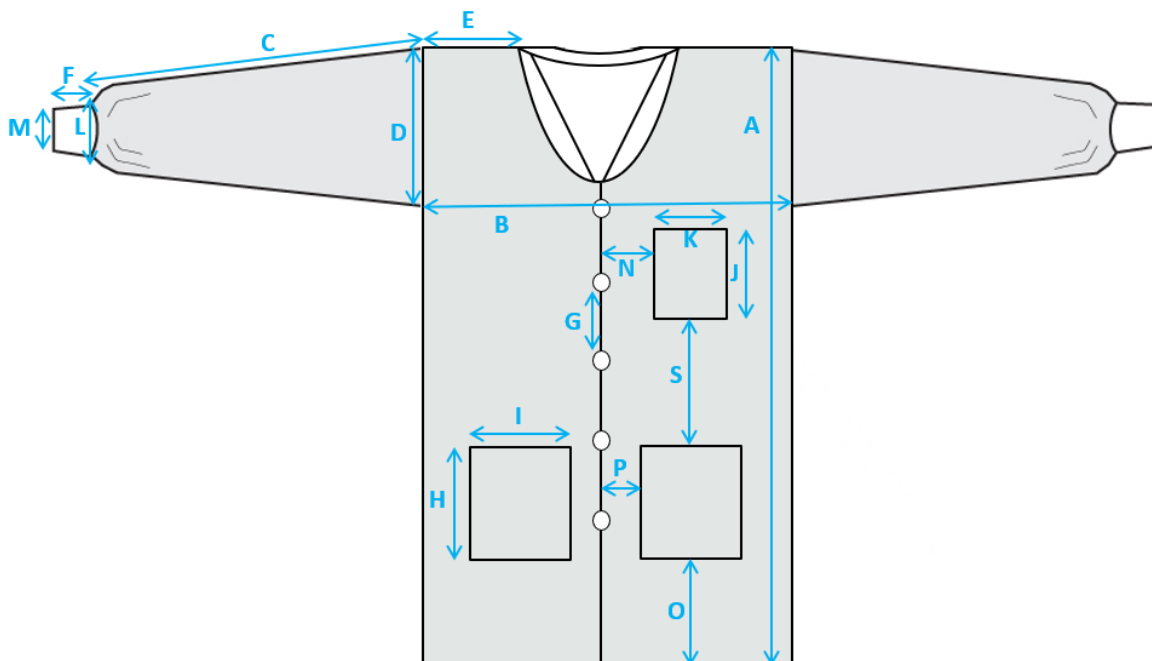



Appendix 1 (List of items)

Item Code	Description	Packaging Information	Size	Color	Specific Attribute	Treatment	Marketed in USA as	Marketed in Canada as
8101-A	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Small	Deep Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8101-B	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Medium	Deep Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8101-C	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Large	Deep Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8101-D	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	X-Large	Deep Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8101-E	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	2X-Large	Deep Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1

Appendix 1 (List of items)

Item Code	Description	Packaging Information	Size	Color	Specific Attribute	Treatment	Marketed in USA as	Marketed in Canada as
8101-F	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	3X-Large	Deep Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8102-A	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Small	Pretty Pink	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8102-B	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Medium	Pretty Pink	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8102-C	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Large	Pretty Pink	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8102-D	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	X-Large	Pretty Pink	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8103-A	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Small	Plum Purple	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1

Appendix 1 (List of items)

Item Code	Description	Packaging Information	Size	Color	Specific Attribute	Treatment	Marketed in USA as	Marketed in Canada as
8103-B	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Medium	Plum Purple	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8103-C	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Large	Plum Purple	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8103-D	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	X-Large	Plum Purple	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8103-XS	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	X-Small	Plum Purple	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8104-A	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Small	Soft Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8104-B	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Medium	Soft Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1

Appendix 1 (List of items)

Item Code	Description	Packaging Information	Size	Color	Specific Attribute	Treatment	Marketed in USA as	Marketed in Canada as
8104-C	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Large	Soft Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8104-D	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	X-Large	Soft Blue	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8105-A	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Small	White	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8105-B	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Medium	White	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8105-C	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Large	White	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8105-D	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	X-Large	White	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1

Appendix 1 (List of items)

Item Code	Description	Packaging Information	Size	Color	Specific Attribute	Treatment	Marketed in USA as	Marketed in Canada as
8115-A	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Small	Tropical Teal	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8115-B	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Medium	Tropical Teal	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8115-C	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Large	Tropical Teal	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8115-D	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	X-Large	Tropical Teal	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8116-A	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Small	Poppy Pink	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8116-B	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Medium	Poppy Pink	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1

Appendix 1 (List of items)

Item Code	Description	Packaging Information	Size	Color	Specific Attribute	Treatment	Marketed in USA as	Marketed in Canada as
8116-C	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Large	Poppy Pink	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8116-D	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	X-Large	Poppy Pink	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8118-A	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Small	Jet Black	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8118-B	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Medium	Jet Black	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8118-C	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	Large	Jet Black	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1
8118-D	Medicom SafeWear Hipster Jacket	12 Jackets/Bag, 5 Bags/Case, 60 Jackets/Case	X-Large	Jet Black	Matching Sewing Thread Color, 3 Pockets	Non-Sterile	Medical Device Class 1	Medical Device Class 1

AMD Medicom Inc. | 2555 Chemin de l'Aviation, Montreal, Quebec, Canada H9P 2Z2